


Member View

As a manager or editor of an org, sometimes you want to test how your Quizzes work, or want to complete a Quiz made for you by another user.

To do this you can enter 'Member View'

First click your initials in the top right

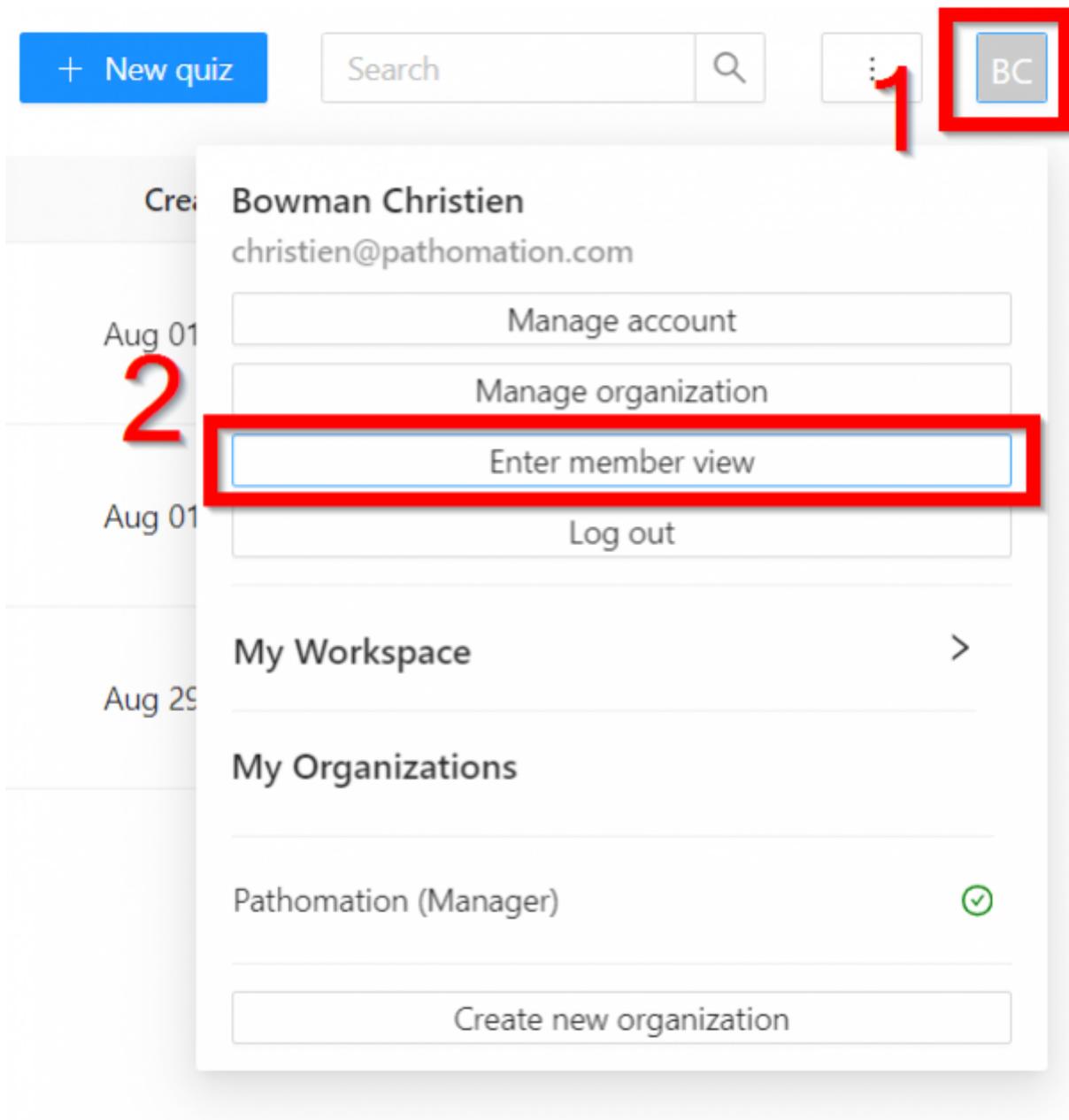

Then Enter Member View

The screenshot shows the 'My Pathomation' interface. On the left is a navigation menu with 'Quizzes' selected. The main area displays a table of quizzes. A notification at the top right says 'You are currently in Member view!' with an 'Exit member view' button. A red arrow points to this notification. Below the notification is a search bar. The table below has the following data:

Title	# of questions	Quiz scored	Quiz scorable	Created on	Actions
Demo1	2	✓	✓	Aug 01, 2024, 09:09	ⓘ
Demo2	0	✗	✓	Aug 01, 2024, 09:19	ⓘ

Now experience your content as your users do



From:
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:
https://docs.pathomation.com/my/doku.php?id=member_view&rev=1725184974

Last update: **2024/09/01 13:02**

