

Shares

My Pathomation lets you share your [slides](#), [cases](#), [collections](#) and [slideboxes](#) with whoever you want! People receiving your shares will see exactly what you want them to. The whole slide/collection/slidebox is shared by default, but by configuring the advanced options when you hit the “share” button you can add or remove features and information including the filename, barcode, and individual zoom levels. You can view your current shares by going to “Shares” in the sidebar.

Here you can also see the number of views (1), edit (2) and delete (3) your shares. A note on viewing statistics: The first time an individual opens your shared slide/case/collection/slidebox, it will be counted as a view.

Pathomation demo **Active shares** 99+ Expired shares 0 Deleted shares 0

Type	Title	URL	Created	Views	Actions
Course	Biology year 1 plant histology - Root structure	https://my.pathomation.com/share/course/course_1	Created on Jul 13, 2023, 15:24 by Hovelinck Rudy	38 Views	1, 2, 3
Slide	Elodea root.mrxs	https://my.pathomation.com/share/slide/3tDsss0YzZn23oTEpfcH	Created on Mar 24, 2023, 09:14 by Hovelinck Rudy	8 Views	2, 3
Slidebox	Test demo slidebox	https://my.pathomation.com/share/slidebox/slidebox_150	Created on Nov 08, 2023, 11:21 by Hovelinck Rudy	2 Views	2, 3
Slide	Breast case 1 HER2.mrxs	https://my.pathomation.com/share/slide/odtGqAL6T3iMmT2uQOT	Created on Jan 18, 2023, 15:17 by Hovelinck Rudy	22 Views	2, 3
Slide	slide_05_HER2.mrxs	https://my.pathomation.com/share/slide/CbdmTeJJ751GxRBDXEvh	Created on Oct 20, 2023, 18:21 by Hovelinck Rudy	14 Views	2, 3

From: <https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link: https://docs.pathomation.com/my/doku.php?id=my_shares

Last update: **2023/11/09 16:23**

