


Shares

My Pathomation lets you share your [slides](#), [cases](#), [collections](#) and [slideboxes](#) with whoever you want! People receiving your shares will see exactly what you want them to. The whole slide/collection/slidebox is shared by default, but by configuring the advanced options when you hit the “share” button you can add or remove features and information including the filename, barcode, and individual zoom levels. You can view your current shares by going to “Shares” in the sidebar.

You can view your current shares by going to “My shares” in the sidebar.

Here you can also see the number of views: The first time an individual opens your shared slide/case/collection/slidebox, it will be counted as a view.

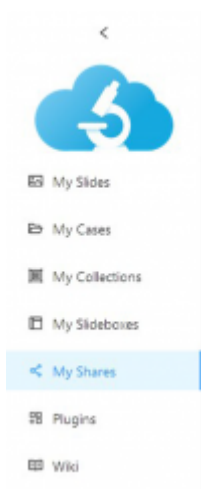

From:
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:
https://docs.pathomation.com/my/doku.php?id=my_shares&rev=1699535236

Last update: **2023/11/09 16:07**

