

Release Notes

Current release Build #3.0.2

Error occurred while sending invitations to users to join an organisation

The bug was fixed and the invitation function was restored

- Bug fix

When selecting a free algorithm with zero credits on an account the algorithm did not start and displays a message that the credits are insufficient.

The credit rule was correctly configured

- Bug fix

The Z-stack slider is not shown.

The bug in the Z-stack slider feature was fixed

- Bug fix

The free pilot WSI algorithms do not return a result.

The bug fix is ongoing

- Known bug

Previous releases

Build #3.0.1

Top up for use of algorithms in the AI plan

Users can pay for third party AI algorithms by putting credit in their account. Only available in Organizations with a Case Sharing and AI plan

- New

Enforce web caching refresh

Users could experience lack of features if they had caching requiring hard refresh to make new functions visible, My Pathomation now enforces this

- Enhancement

Separate viewport settings for user level & organisation view versus viewport settings for shares

The settings for an individual user define how the user interacts with images across all viewer interfaces in My Pathomation, shares are controlled separately

- Enhancement

Users with inactive accounts that are added to an organisation are asked to take action

Upon opening the invitation link users with an inactive account get a message "Your access to the

platform is limited. Please contact the organization's manager(s)''

- Enhancement

Backend monitoring and performance of My Pathomation platform

Minor improvements of dashboard for Pathomation service, Identity provider HTTPS

- Enhancement

Removed unintended AI menu option in Education and collaboration plan

In the education and collaboration plan the AI icon appeared as an option under the '...' menu and was not functional

- Bug fix

Aligned phrasing of RUO AI descriptive text

When launching an AI algorithm that has the research use only designation the wrong text was shown. Only available in the Case Sharing and AI plan.

- Bug fix

Collections and Slideboxes retain slides and cases when renaming image source folders

Slideboxes and slide collections contains WSI from the slide repository. When changing the folder names in the slide repository, the collections and slideboxes broke.

- Bug fix

Build#3.0.0

Case sharing and AI plan

Users can now access third party AI algorithms and define a case workflow. Only available in Organizations with a Case Sharing and AI plan

- New

Free plan

Differentiation of users with a free plan that can Upload up to a 10GB ceiling in a cumulative count, share 3 times per month versus with a paid plan that enjoys non cumulative storage counting, upload and download, unlimited sharing

- New

Updated terms of use, privacy policy

Text was changed to be in line with regulatory requirements

- Update

Data Processor Agreement

Document was created to be in line with GDPR and HIPAA regulation

- New

Build#2.0.0.383d3d5**Member view**

Only available in organizations

New

As a manager or an editor of an organization, you can now enable Member view in order to be able to have an overview of what members of your organization have access to, and also be able to score quizzes.

To access Member view click on your profile icon in the top right corner of the screen and click on the button from the dropdown menu.

You should add your account to the organization groups you want to be member of before entering member view!

Multiple scores in quizzes

Only available in organizations

Change

Members of a group can now score multiple times in a single quiz. To enable this feature for a specific quiz, check the Users can score multiple times switch in the quiz's edit page.

If a member of a group scores a quiz multiple times, the previous scores will be lost and only the last score will be saved!

Groups changes Only available in organizations

Change

While managing a group, you can now reorder collections, slideboxes, quizzes and courses available in a group to your liking.

Annotation tools

Change

Annotation tools are now available on all locations with annotation editing capabilities and can have predefined dimensions.

To define annotation tools, you can visit your organization's page and click on the button.

Build#7b2ef543**Quizzes**

New

Quizzes are now supported by My Pathomation. A quiz is a set of different questions that can be assigned to a group of members or be part of a course. Currently available question types are:

- Multiple choice (with one or more correct answers)
- Essay
- Numerical
- No answer

To create a quiz, you can visit new Quizzes menu item and click on button. After defining a name for the quiz, you can change available options and create new questions.

From:

<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:

https://docs.pathomation.com/my/doku.php?id=release_notes&rev=1754907018

Last update: **2025/08/11 13:10**

