

## Release Notes

### Current release Build #3.0.3

#### **in a smaller browser window the upload button disappears**

The bug was fixed and the upload button remains visible

- Bug fix

#### **In the confirmation step of the plan for AI and case sharing 'courses and quizzes' was mistakenly shown as included**

The text was removed

- Bug fix

#### **The annotations made by AI analysis were not visible in collections**

The annotations are now visible by default

- Bug fix

#### **Text is overlapping AI made annotations making interpretation by pathologists sometimes impossible**

The amount of visible text detail scales with the zoom level improving visibility on the annotations

- Bug fix

#### **My Pathomation Blank Page Appears When Re-Editing Annotations on KFB Slides**

KFB slides annotation editing does not result in a blank page \* **Bug fix**

Channels are not visible in my pathomation organisation

#### **Channel settings in the viewport are retained in the organisation**

- Bug fix

#### **Folder containing failed uploads cannot be deleted in My Pathomation**

Failed uploads are purged from the directory unblocking deletion \* **Bug fix**

Moving WSI file message does not disappear

#### **Moving file message is part of a new Status menu placed on top right of software and shows on prompt only**

- Bug fix

#### **AI task bar field remains visible throughout all slides in the viewer, hiding the overview**

AI task bar is now part of a new Status menu placed on top right of software and shows on prompt only \* **Bug fix**

The tracking function was hidden by the status screen

#### **All task bars are now part of a new Status menu placed on top right of software and shows on prompt only**

- Bug fix

## Previous releases

### Build #3.0.2

#### **Error occurred while sending invitations to users to join an organisation**

The bug was fixed and the invitation function was restored

- Bug fix

#### **When selecting a free algorithm with zero credits on an account the algorithm did not start and displays a message that the credits are insufficient.**

The credit rule was correctly configured

- Bug fix

#### **The Z-stack slider is not shown.**

The bug in the Z-stack slider feature was fixed

- Bug fix

#### **The free pilot WSI algorithms do not return a result.**

The bug fix is ongoing

- Known bug

### Build #3.0.1

#### **Top up for use of algorithms in the AI plan**

Users can pay for third party AI algorithms by putting credit in their account. Only available in Organizations with a Case Sharing and AI plan

- New

#### **Enforce web caching refresh**

Users could experience lack of features if they had caching requiring hard refresh to make new functions visible, My Pathomation now enforces this

- Enhancement

#### **Separate viewport settings for user level & organisation view versus viewport settings for shares**

The settings for an individual user define how the user interacts with images across all viewer interfaces in My Pathomation, shares are controlled separately

- Enhancement

#### **Users with inactive accounts that are added to an organisation are asked to take action**

Upon opening the invitation link users with an inactive account get a message "Your access to the platform is limited. Please contact the organization's manager(s)"

- Enhancement

## **Backend monitoring and performance of My Pathomation platform**

Minor improvements of dashboard for Pathomation service, Identity provider HTTPS

- Enhancement

## **Removed unintended AI menu option in Education and collaboration plan**

In the education and collaboration plan the AI icon appeared as an option under the '...' menu and was not functional

- Bug fix

## **Aligned phrasing of RUO AI descriptive text**

When launching an AI algorithm that has the research use only designation the wrong text was shown. Only available in the Case Sharing and AI plan.

- Bug fix

## **Collections and Slideboxes retain slides and cases when renaming image source folders**

Slideboxes and slide collections contains WSI from the slide repository. When changing the folder names in the slide repository, the collections and slideboxes broke.

- Bug fix

## **Build#3.0.0**

### **Case sharing and AI plan**

Users can now access third party AI algorithms and define a case workflow. Only available in Organizations with a Case Sharing and AI plan

- New

### **Free plan**

Differentiation of users with a free plan that can Upload up to a 10GB ceiling in a cumulative count, share 3 times per month versus with a paid plan that enjoys non cumulative storage counting, upload and download, unlimited sharing

- New

### **Updated terms of use, privacy policy**

Text was changed to be in line with regulatory requirements

- Update

### **Data Processor Agreement**

Document was created to be in line with GDPR and HIPAA regulation

- New

## **Build#2.0.0.383d3d5**

## **Member view**

Only available in organizations

New

As a manager or an editor of an organization, you can now enable Member view in order to be able to have an overview of what members of your organization have access to, and also be able to score quizzes.

To access Member view click on your profile icon in the top right corner of the screen and click on the button from the dropdown menu.

You should add your account to the organization groups you want to be member of before entering member view!

### **Multiple scores in quizzes**

Only available in organizations

Change

Members of a group can now score multiple times in a single quiz. To enable this feature for a specific quiz, check the Users can score multiple times switch in the quiz's edit page.

If a member of a group scores a quiz multiple times, the previous scores will be lost and only the last score will be saved!

**Groups changes** Only available in organizations

Change

While managing a group, you can now reorder collections, slideboxes, quizzes and courses available in a group to your liking.

### **Annotation tools**

Change

Annotation tools are now available on all locations with annotation editing capabilities and can have predefined dimensions.

To define annotation tools, you can visit your organization's page and click on the button.

## **Build#7b2ef543**

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
### **Quizzes**

New

Quizzes are now supported by My Pathomation. A quiz is a set of different questions that can be assigned to a group of members or be part of a course. Currently available question types are:

- Multiple choice (with one or more correct answers)
- Essay
- Numerical
- No answer

To create a quiz, you can visit new Quizzes menu item and click on button. After defining a name for the quiz, you can change available options and create new questions.



From:  
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:  
[https://docs.pathomation.com/my/doku.php?id=release\\_notes&rev=1762444234](https://docs.pathomation.com/my/doku.php?id=release_notes&rev=1762444234)

Last update: **2025/11/06 18:50**

