

# My Pathomation Start Page

My Pathomation is a cloud platform offering services for digital pathology applications.







## Features

With My Pathomation you can store and share images for all scanner file formats. It comes with advanced features like annotation tools, collection organisation, access management and student course building. Registered users can store whole slide images (WSIs) and share them in an easy and universal way.

Whole slide images can be organised in folders but also be part of Collections, which in turn can be part of Slideboxes.

The tag features provides an additional layer of data for identification.

				
<b>Register</b>	<b>Upload</b>	<b>Share</b>	<b>Teach</b>	<b>Learn</b>
an individual or organisation account	your digital pathology slides from your drives to the My Pathomation cloud	individual slides and slide collections with others either private or public	with live digital slide microscopy in your presentations or classes	at your own pace in on-line courses for students that have text instructions and interactive digital pathology slides

## Organisations


Organisations in My Pathomation offer many interesting new features on top of annotations, hosting and organizing your slides. The most exciting features for educators are **courses** and **quizzes**. With these powerful tools you can create your own content which integrate your fully interactive slides with course material, your own logos / branding, and many different question types including multiple choice, essay open field, number input and more.

Organisations have multiple user profiles including managers, content editors and members, each with different access rights.

My Pathomation also comes with a course builder tool and easy setup and management of groups and access right to all content.

## Storage Options and Pricing

My Pathomation offers free image storage up to 10GB. For storing larger volumes of images, a monthly, volume-based subscription fee is required starting at 30 EUR for 25GB storage. Users can increase and decrease their required storage volume as required per organisation or personal workspace. There is no upper limit for the storage volume.

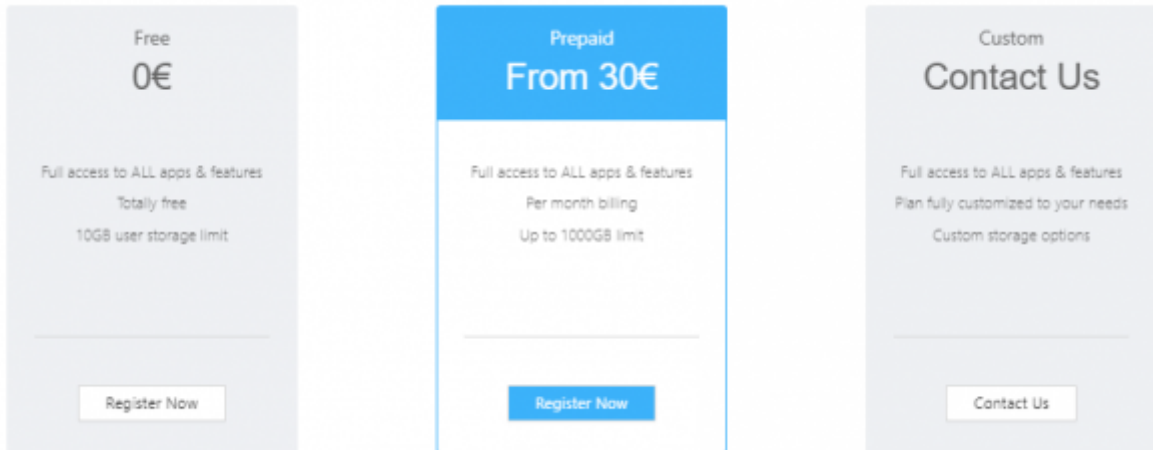

## We Support You(r file types)

The My Pathomation cloud platform is compatible with all routine whole slide image formats. If you find a filetype that's not included that you'd like to use, we'd love to hear from you so we can integrate it into our platform!

**BRIGHTFIELD**

3DHistech MRXS  
 Aperio / Leica CWS  
 Aperio LEICA SCN  
 Aperio LEICA SVS  
 GE Omnyx / Inspirata RTS  
 GE Omnyx JP2  
 Generic JPG  
 Generic PNG  
 Generic TIFF  
 Hamamatsu NDPI  
 Hamamatsu VMS  
 Huron Technologies TIFF  
 JPEG 2000 encoded slides  
 Jpeg XR JXR  
 KFBio KFB  
 Menarini DSight Raw INI  
 Microsoft Deep Zoom .DZI  
 Motic MDS  
 Motic MDSX  
 Nikon ND2  
 Philips iSyntax  
 Philips TIFF  
 Olympus VSI  
 Olympus Webview  
 Open Microscopy OME-TIFF  
 Objective Imaging (Glissando)  
 Omero ZARR  
 Sakura SVSLIDE  
 Smartzoom SZI  
 Unic Tech TMAP  
 Ventana Roche BIF  
 (with and without TIFF)  
 Zeiss ZVI  
 Zeiss CZI  
 Zoomify ZIF

**FLUORESCENT**

3DHistech MRXS  
 Aperio AFI  
 Aperio LEICA SCN  
 Aperio LEICA SVS  
 Hamamatsu NDPIS  
 Nikon ND2  
 Nikon TIFF  
 Leica LIF  
 Olympus OIR  
 Olympus VSI  
 Open Microscopy OME-TIFF  
 Perkin Elmer QPTIFF  
 Zeiss CZI  
 Zeiss LSM  
 Zeiss ZVI

**Z-STACKS**

3DHistech MRXS  
 Aperio LEICA SVS  
 Aperio LEICA SCN  
 Hamamatsu NDPIS  
 Huron Technologies TIFF  
 Leica LIF  
 Nikon ND2  
 Objective Imaging (Glissando)  
 Open Microscopy OME-TIFF  
 Olympus OIR  
 Olympus VSI  
 Sakura SVSLIDE  
 Zeiss ZVI  
 Zeiss LSM  
 Zeiss CZI



From:  
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:  
<https://docs.pathomation.com/my/doku.php?id=start&rev=1706543839>

Last update: **2024/01/29 18:57**

