

Whats new?

Build#3.0.0

Case sharing and AI plan

Previous Versions

Build#2.0.0.383d3d5

Member view

Only available in organizations

New

As a manager or an editor of an organization, you can now enable Member view in order to be able to have an overview of what members of your organization have access to, and also be able to score quizzes.

To access Member view click on your profile icon in the top right corner of the screen and click on the button from the dropdown menu.

You should add your account to the organization groups you want to be member of before entering member view!

Multiple scores in quizzes

Only available in organizations

Change

Members of a group can now score multiple times in a single quiz. To enable this feature for a specific quiz, check the Users can score multiple times switch in the quiz's edit page.

If a member of a group scores a quiz multiple times, the previous scores will be lost and only the last score will be saved!

Groups changes

Only available in organizations

Change

While managing a group, you can now reorder collections, slideboxes, quizzes and courses available in a group to your liking.

Annotation tools

Change

Annotation tools are now available on all locations with annotation editing capabilities and can have predefined dimensions.

To define annotation tools, you can visit your organization's page and click on the button.

Build#7b2ef543

Quizzes

New

Quizzes are now supported by My Pathomation. A quiz is a set of different questions that can by

assigned to a group of members or be part of a course. Currently available question types are:

- Multiple choice (with one or more correct answers)
- Essay
- Numerical
- No answer

To create a quiz, you can visit new Quizzes menu item and click on button. After defining a name for the quiz, you can change available options and create new questions.

From:

<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:

https://docs.pathomation.com/my/doku.php?id=what_s_new&rev=1752237039

Last update: **2025/07/11 15:30**

