

## Whats new?

### Build#3.0.0

#### Case sharing and AI plan

Users can now access third party AI algorithms and define a case workflow. Only available in Organizations with a Case Sharing and AI plan

- New

#### Free plan

Differentiation of users with a free plan that can Upload up to a 10GB ceiling in a cumulative count, share 3 times per month versus with a paid plan that enjoys non cumulative storage counting, upload and download, unlimited sharing

- New

#### Updated terms of use, privacy policy

Text was changed to be in line with regulatory requirements

- Update

#### Data Processor Agreement

Document was created to be in line with GDPR and HIPAA regulation

- New

### Previous Versions

#### Build#2.0.0.383d3d5

#### Member view

Only available in organizations

New

As a manager or an editor of an organization, you can now enable Member view in order to be able to have an overview of what members of your organization have access to, and also be able to score quizzes.

To access Member view click on your profile icon in the top right corner of the screen and click on the button from the dropdown menu.

You should add your account to the organization groups you want to be member of before entering member view!

#### Multiple scores in quizzes

Only available in organizations

Change

Members of a group can now score multiple times in a single quiz. To enable this feature for a specific quiz, check the Users can score multiple times switch in the quiz's edit page.

If a member of a group scores a quiz multiple times, the previous scores will be lost and only the last

score will be saved!

### **Groups changes** Only available in organizations

Change

While managing a group, you can now reorder collections, slideboxes, quizzes and courses available in a group to your liking.

### **Annotation tools**

Change

Annotation tools are now available on all locations with annotation editing capabilities and can have predefined dimensions.

To define annotation tools, you can visit your organization's page and click on the button.

### **Build#7b2ef543**

### **Quizzes**

New

Quizzes are now supported by My Pathomation. A quiz is a set of different questions that can be assigned to a group of members or be part of a course. Currently available question types are:

- Multiple choice (with one or more correct answers)
- Essay
- Numerical
- No answer

To create a quiz, you can visit new Quizzes menu item and click on button. After defining a name for the quiz, you can change available options and create new questions.

From:

<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:

[https://docs.pathomation.com/my/doku.php?id=what\\_s\\_new&rev=1752566559](https://docs.pathomation.com/my/doku.php?id=what_s_new&rev=1752566559)

Last update: **2025/07/15 11:02**

