


# Adding Cases and Case Collections to a Project

Case collections are how you organize your content to be shown in courses.

## Creating a case collection in a project

You can create a Case Collection via the [slides and images](#) option in the sidebar of Pathotrainer and assign that case collection to a project during setup.

Or you can directly add a case collection to a project from within the project details window (shown below). If you have not added any case collections to your project, you will see the following bluetext (1) prompting you to do so:

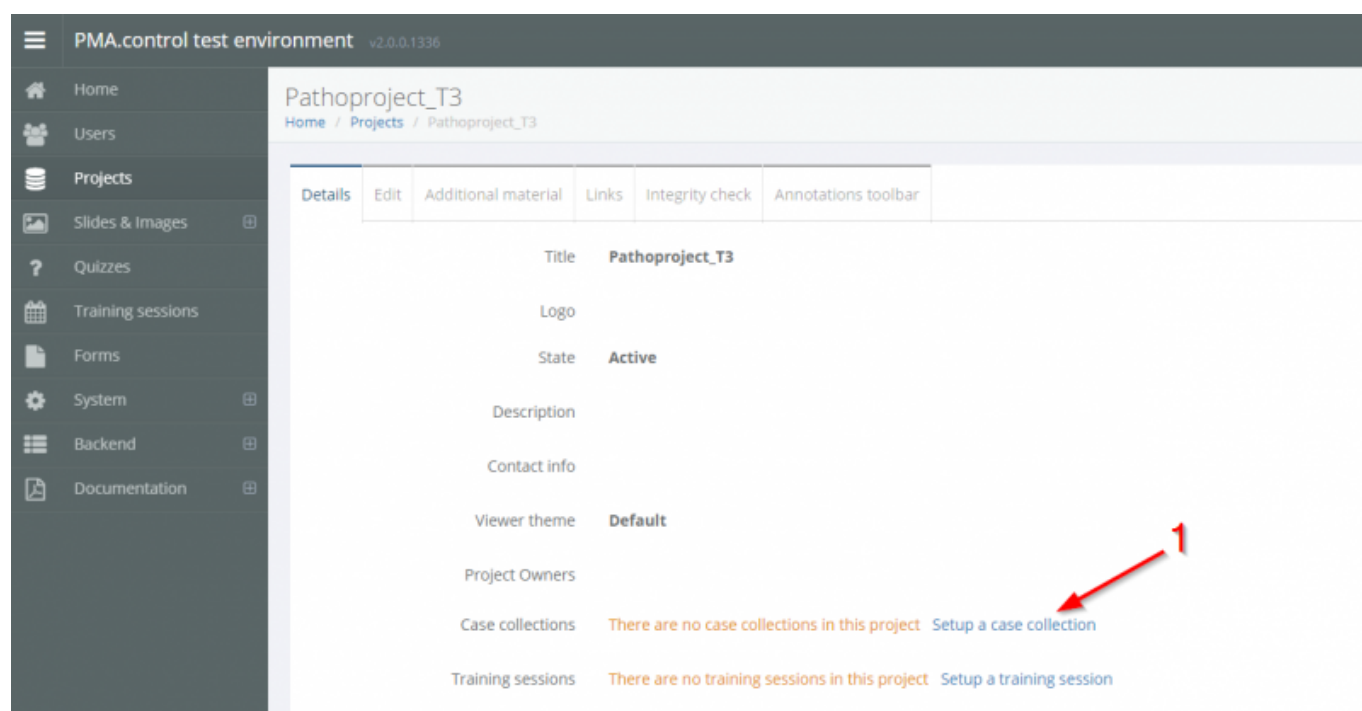

If you have already added some case collections, you can click on the bluetext name of the case collection(s) (1) to view the case collection:

Patho\_Project\_Multicase

Home / Projects / Patho\_Project\_Multicase

Details Edit Additional material Links Integrity check Annotations toolbar

Title Patho\_Project\_Multicase

Logo

State Active

Description A project to add multiple case collections

Contact info

Viewer theme Default

Project Owners

Case collections [2]

Title	# of cases	Scoring Form
X	0	Test form
Y	1	Test form

Training sessions There are no training sessions in this project. Setup a training session

then click Case Collections (2),

PMA.control test environment v2.0.0.1336

Home / Projects / Patho\_Project / Case collections / Test

Case collections

Details Edit Consensus data Cases Graph Integrity check Audit Trail

Logo No file selected

Max file size: 10Mb

Project Patho\_Project

Title Test

Scoring Form Test form

Clinical information form Test form

Show slide labels ☒


Show measurements ☒

Case comments templates Nothing selected

Published Nothing selected

Save Back to list

and then Create (3).

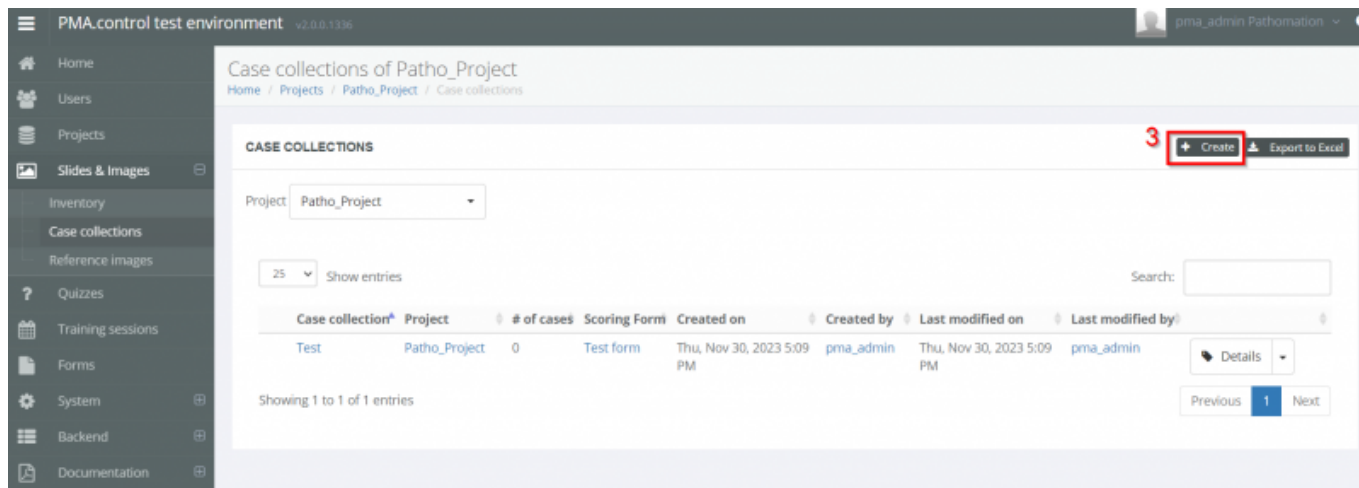

## Creating a New Case Collection

After hitting the “Create” button, you will be presented with the option to define some details about your new collection:

1. Uploading a logo image
2. Assigning the case to a project
3. Giving the case a name
4. Assigning a scoring form for your trainees - how they will complete slide-based exam questions
5. Assigning a clinical information form for your trainees
6. Options to toggle slide label and measurement visibility
7. Additional comment templates

The screenshot shows the 'Create new case collection' form. The form fields are numbered 1 through 7, corresponding to the steps in the list above:


1. Logo: A file upload button with 'No file selected' and 'Max file size: 10Mb'.
2. Project: A dropdown menu with 'Patho\_Project' selected.
3. Title: A text input field with 'Test' entered.
4. Scoring Form: A dropdown menu with 'Test form' selected.
5. Clinical information form: A dropdown menu with 'Test form' selected.
6. Show slide labels: A checkbox that is checked.
6. Show measurements: A checkbox that is checked.
7. Case comments templates: A dropdown menu with 'Nothing selected'.

At the bottom of the form are two buttons: 'Create' and 'Back to list'.

## Adding Cases


Now that you've created a case, its time to add some cases.

To do this, navigate to the “Cases” tab and click the “Create”

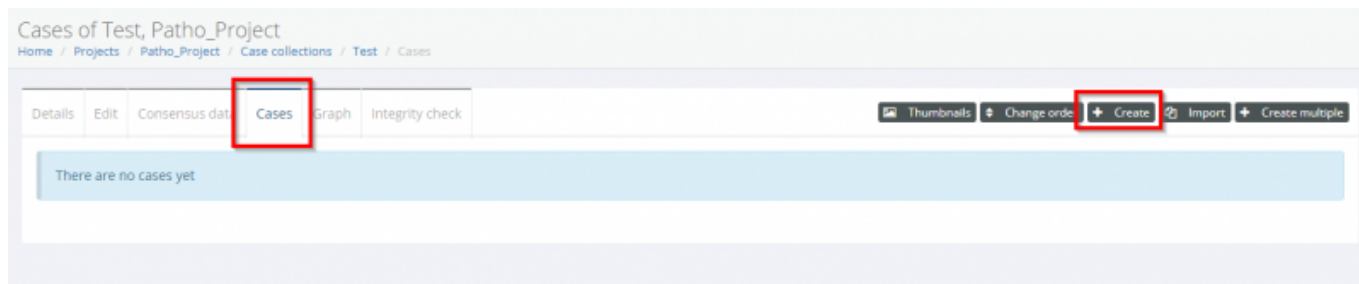


You will find yourself in the Editing Cases tab.  
Here you can set up various case properties:

1. A name for the case
2. Whether to display the slide filename, caption, or no label to the participants
3. Comments
4. Select your slide(s) from your PMA core inventory via the file treeview
5. Save your slides

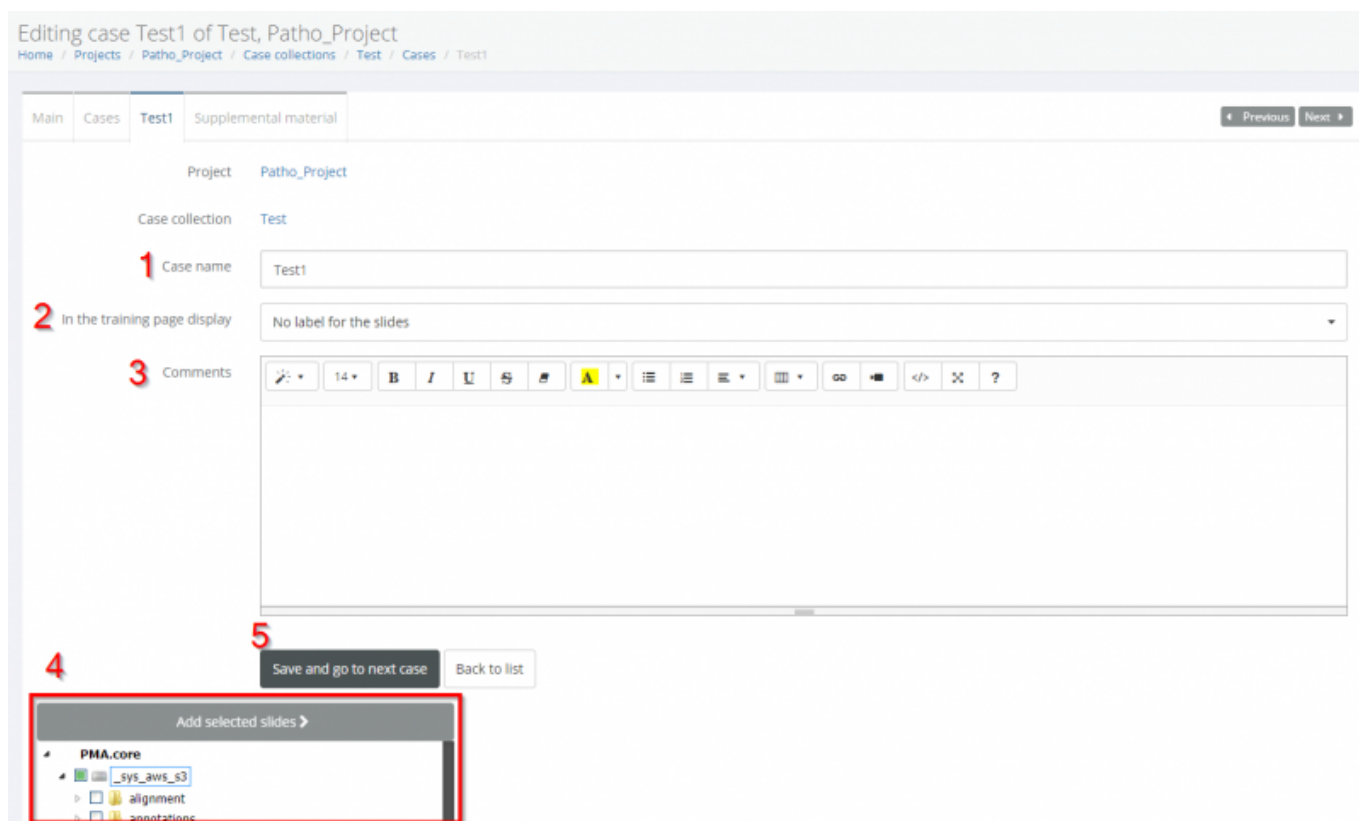

## Selecting slides to add


To select slides, navigate the treeview and click the checkmark next to your desired slides (1). These can be from several different folders, PMA.cores, or servers. You can hover your mouse over the filename to preview a thumbnail of the image. When you've picked them, click 'Add selected slides' (2).

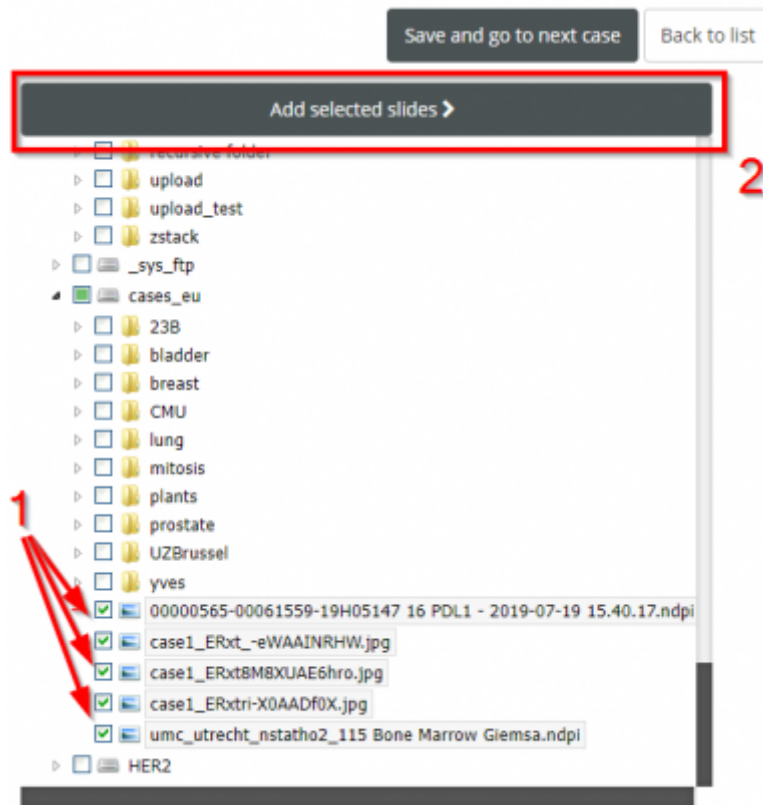

Once the slides are added you can check and change several details about them

1. The filename
2. The path
3. The caption - you can edit this which is useful if you want to give different labels to your participants and only display the caption
4. Add comments
5. See and modify annotations on the slide

You can also click the 'X' button to remove a slide from the collection

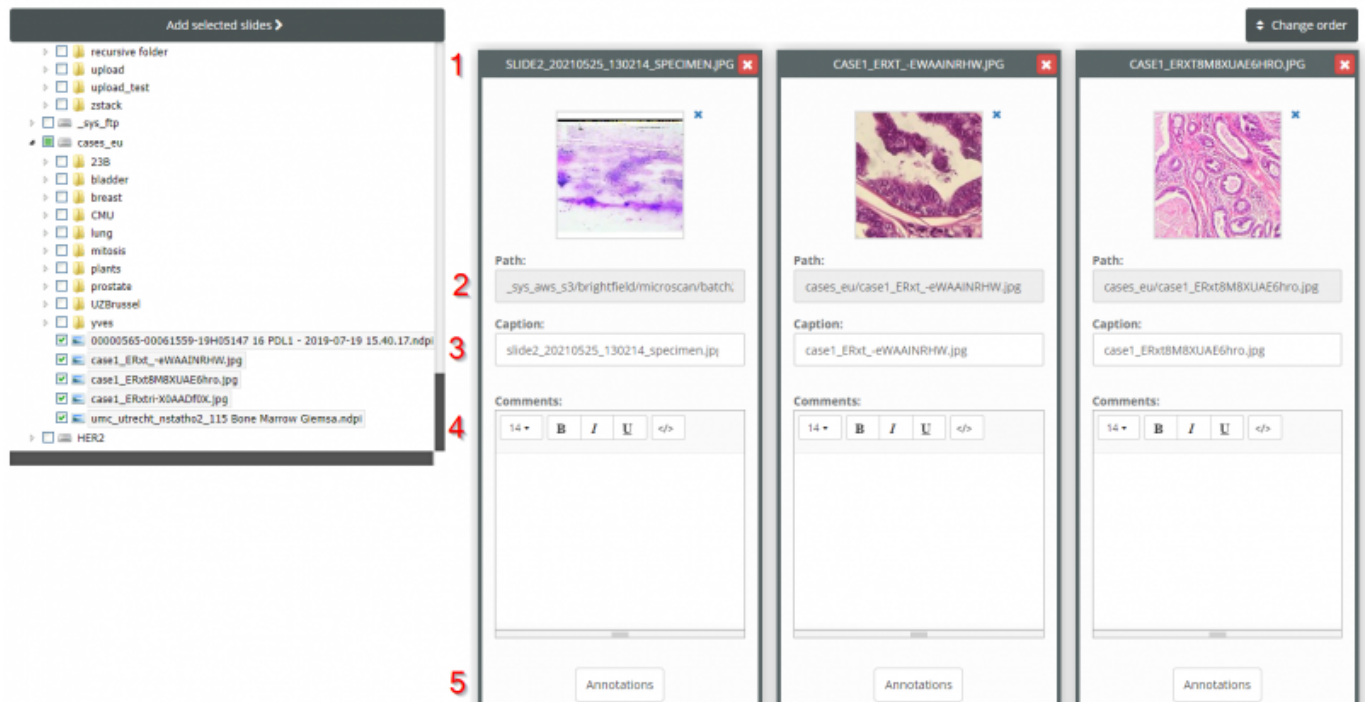

## Adding Multiple Cases

To add multiple cases at once that each have their own folder, you can click the 'Create multiple' button in the Cases tab. Then you will be prompted to select a directory containing all the Case subfolders (1), click to preview what case folders are detected (2), Create (3).

The screenshot displays the 'Create new case(s)' page in the PathoProject application. At the top, there's a breadcrumb trail: Home / Projects / Patho\_Project / Case collections / Test 2 / Cases / Create Multiple. Below this, two tabs are visible: 'Select directory' (active) and 'Pattern search'. The main heading reads 'Create new case(s) for case collection Test 2, project Patho\_Project'. A grey instruction box states: 'Select a directory to create multiple cases. Each sub-directory of the selected directory, becomes a new case and all the slides, directly in it, are added.'

A dark sidebar on the left shows a file tree under 'PMA.core'. Red arrows and numbers indicate steps: Arrow 1 points to the 'bladder' folder; Arrow 2 points to the 'Select & Preview >' button at the top of the sidebar.

On the right, a light blue panel titled '6 cases found' contains a 'Create' button, indicated by red arrow 3. Below this, a preview of the selected 'bladder' directory shows six microarray files (HE\_..., PanCK-CD8\_..., PDL1\_..., PDL1NC\_...) and their associated slide thumbnails.

## A Note on Adding Content

You can use any slides from your [slide inventory](#) in Pathotrainer case collections. The slide inventory is managed by PMA.core - for guides on how to import content there, you might find these links useful:

[https://docs.pathomation.com/pma.core/3.0.2/doku.php?id=minimal\\_user\\_management](https://docs.pathomation.com/pma.core/3.0.2/doku.php?id=minimal_user_management)  
<https://docs.pathomation.com/pma.core/3.0.2/doku.php?id=upload>

From:  
<https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link:  
[https://docs.pathomation.com/pathotrainer/doku.php?id=cases\\_and\\_case\\_collections](https://docs.pathomation.com/pathotrainer/doku.php?id=cases_and_case_collections)

Last update: **2023/12/05 16:49**

