


## Creating Cases and Case Collections

Case collections are how organize your content to be shown in courses.

Navigating to the Slides and Images tab on the sidebar, and then selecting Case Collections will bring you to this tab. Here you can see an overview of existing case collections (if any) and can click the create button to add a new one.

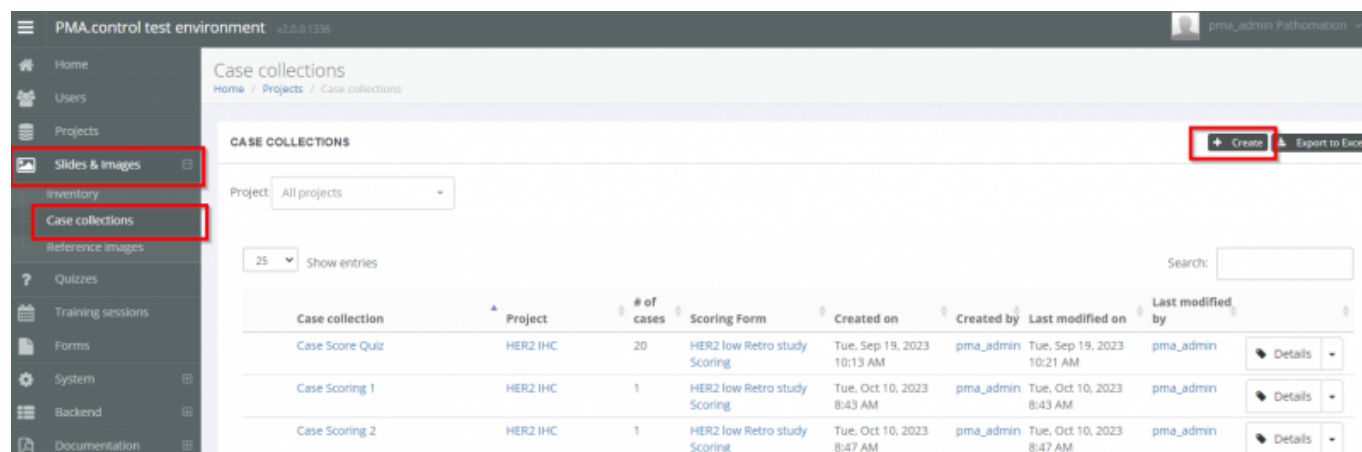

The screenshot displays the 'Case collections' page in the PMA control test environment. The sidebar on the left contains a list of navigation items: Home, Users, Projects, Slides & Images (highlighted with a red box), Inventory, Case collections (highlighted with a red box), Reference images, Quizzes, Training sessions, Forms, System, Backend, and Documentation. The main content area is titled 'Case collections' and includes a 'Project' dropdown menu set to 'All projects'. A '+ Create' button is highlighted with a red box in the top right corner. Below the button is a table listing existing case collections. The table has columns for Case collection, Project, # of cases, Scoring Form, Created on, Created by, Last modified on, and Last modified by. The table contains three entries: 'Case Score Quiz' (20 cases, HER2 low Retro study Scoring), 'Case Scoring 1' (1 case, HER2 low Retro study Scoring), and 'Case Scoring 2' (1 case, HER2 low Retro study Scoring). Each entry has a 'Details' link.

Case collection	Project	# of cases	Scoring Form	Created on	Created by	Last modified on	Last modified by
Case Score Quiz	HER2 IHC	20	HER2 low Retro study Scoring	Tue, Sep 19, 2023 10:13 AM	pma_admin	Tue, Sep 19, 2023 10:21 AM	pma_admin
Case Scoring 1	HER2 IHC	1	HER2 low Retro study Scoring	Tue, Oct 10, 2023 8:43 AM	pma_admin	Tue, Oct 10, 2023 8:43 AM	pma_admin
Case Scoring 2	HER2 IHC	1	HER2 low Retro study Scoring	Tue, Oct 10, 2023 8:47 AM	pma_admin	Tue, Oct 10, 2023 8:47 AM	pma_admin

### A Note on Adding Content

You can use any slides from your [slide inventory](#) in Pathotrainer case collections. The slide inventory is managed by PMA.core - for guides on how to import content there, you might find these links useful:

[https://docs.pathomation.com/pma.core/3.0.2/doku.php?id=minimal\\_user\\_management](https://docs.pathomation.com/pma.core/3.0.2/doku.php?id=minimal_user_management)

<https://docs.pathomation.com/pma.core/3.0.2/doku.php?id=upload>

### Creating a New Case Collection

After hitting the “Create” button, you will be presented with the option to define some details about your new collection:

1. Uploading a logo image
2. Assigning the case to a project
3. Giving the case a name
4. Assigning a scoring form for your trainees
5. Assigning a clinical information form for your trainees
6. Options to toggle slide label and measurement visibility
7. Additional comment templates

### Create new case collection

[Home](#) / [Projects](#) / [Create](#)

- 1 Logo   
Max file size: 10Mb
- 2 Project
- 3 Title
- 4 Scoring Form
- 5 Clinical information form
- 6 ☒ Show slide labels  
☒ Show measurements
- 7 Case comments templates

From:  
<https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link:  
[https://docs.pathomation.com/pathotrainer/doku.php?id=cases\\_and\\_case\\_collections&rev=1701679073](https://docs.pathomation.com/pathotrainer/doku.php?id=cases_and_case_collections&rev=1701679073)

Last update: **2023/12/04 11:37**

