


Adding Cases and Case Collections to a Project

Case collections are how you organize your content to be shown in courses.


Creating a case collection in a project

You can create a Case Collection via the [slides and images](#) option in the sidebar of Pathotrainer and assign that case collection to a project during setup.


Or you can directly add a case collection to a project from within the project details window (shown below). If you have not added any case collections to your project, you will see the following bluetext (1) prompting you to do so:

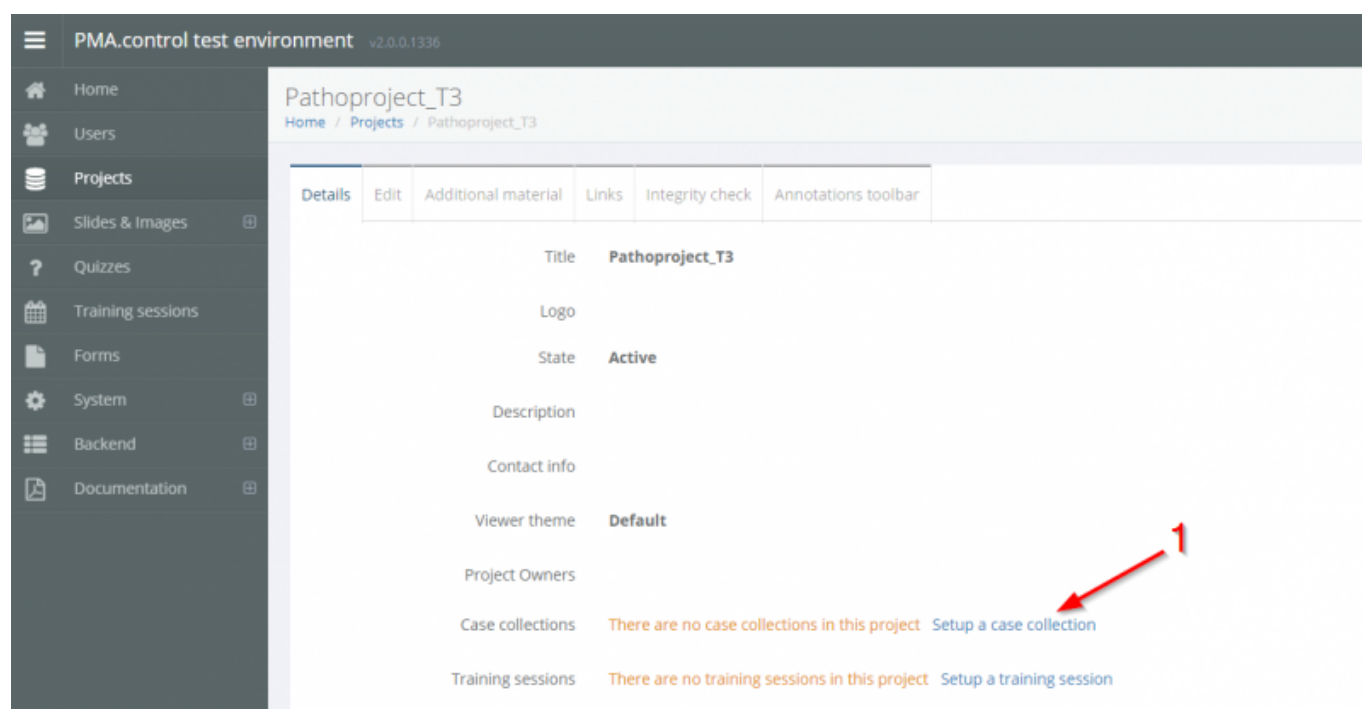


If you have already added some case collections, you can click on the bluetext name of the case collection(s) (1) to view the case collection:

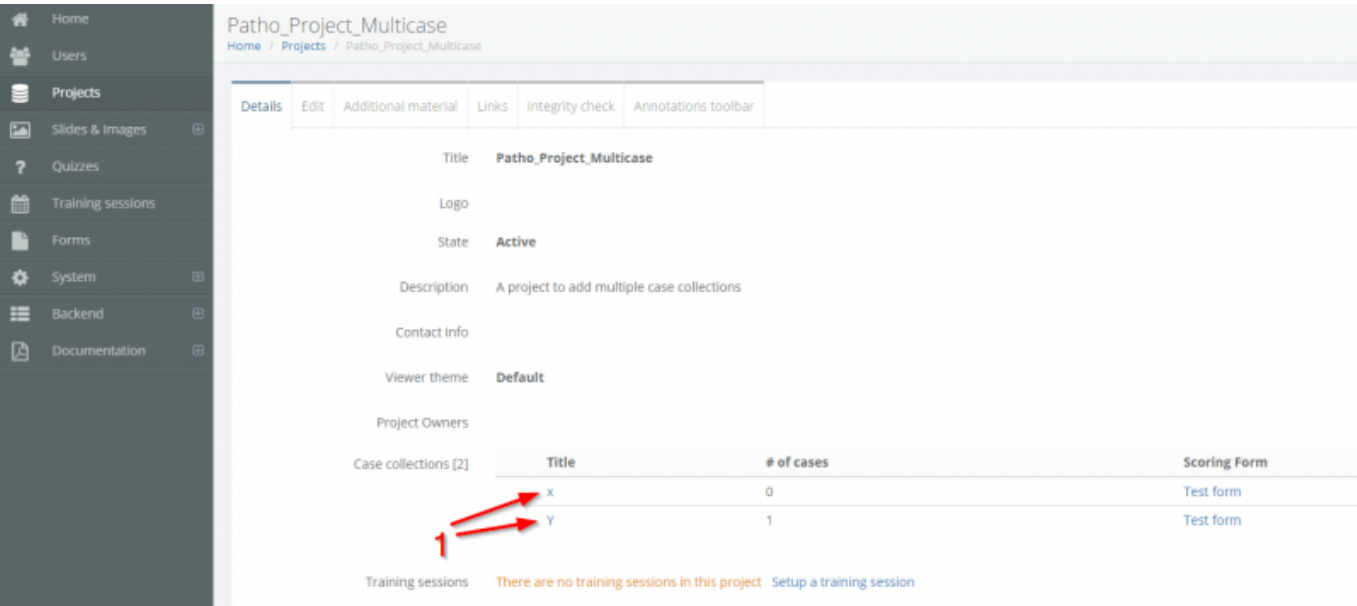

then click Case Collections (2),

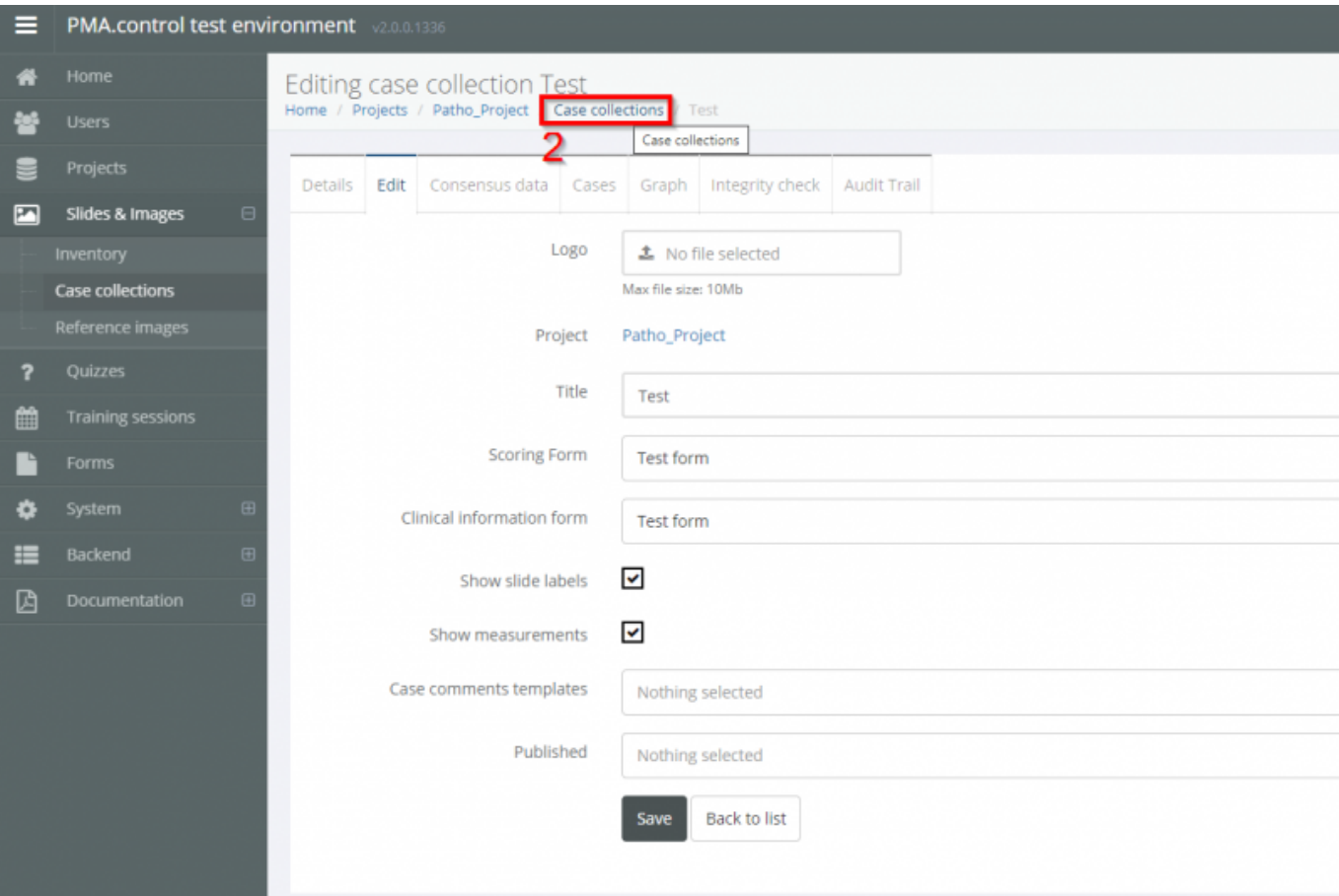

and then Create (3).

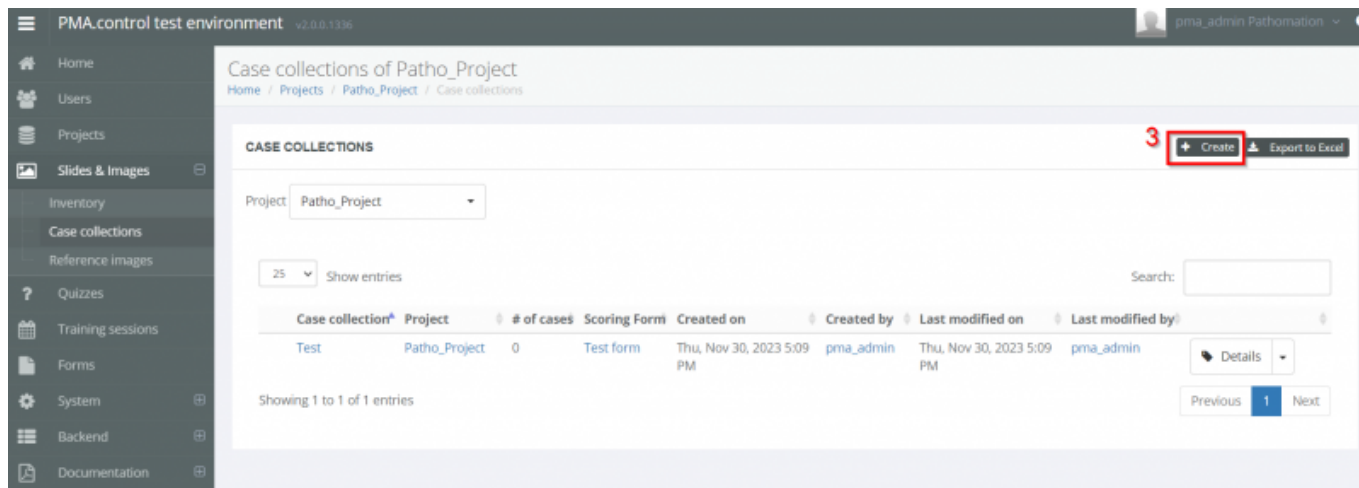

Creating a New Case Collection

After hitting the “Create” button, you will be presented with the option to define some details about your new collection:

1. Uploading a logo image
2. Assigning the case to a project
3. Giving the case a name
4. Assigning a scoring form for your trainees
5. Assigning a clinical information form for your trainees
6. Options to toggle slide label and measurement visibility
7. Additional comment templates

The screenshot shows the 'Create new case collection' form. The form has the following fields and options:


- 1. Logo: A file upload button with the text 'No file selected' and a note 'Max file size: 10Mb'.
- 2. Project: A dropdown menu with 'Patho_Project' selected.
- 3. Title: A text input field with 'Test' entered.
- 4. Scoring Form: A dropdown menu with 'Test form' selected.
- 5. Clinical information form: A dropdown menu with 'Test form' selected.
- 6. Show slide labels: A checkbox that is checked.
- 6. Show measurements: A checkbox that is checked.
- 7. Case comments templates: A dropdown menu with 'Nothing selected'.

At the bottom of the form are two buttons: 'Create' and 'Back to list'.

Adding Cases


Now that you've created a case, its time to add some cases.

To do this, navigate to the “Cases” tab and click the “Create”

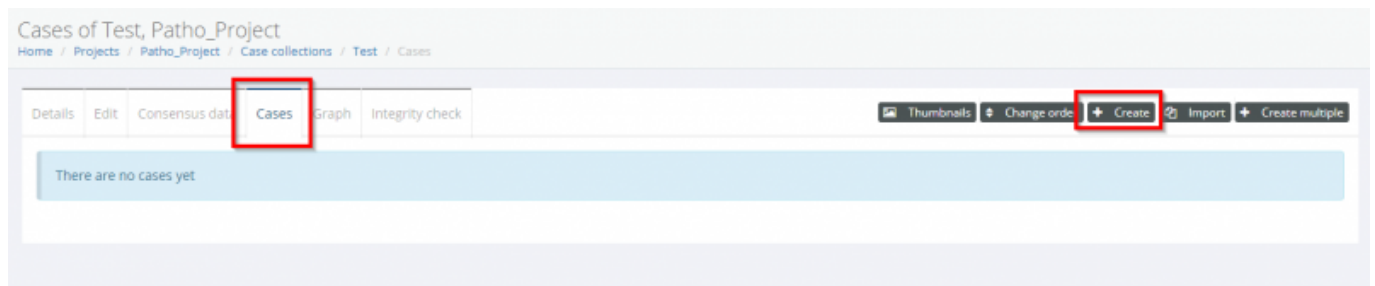

You will find yourself in the Editing Cases tab.
Here you can set up various case properties:

- A name for the case
- What slide information will be displayed: no labels, filenames, the caption
- Comments
- Select your slide(s) from your PMA core inventory
- Save your slides

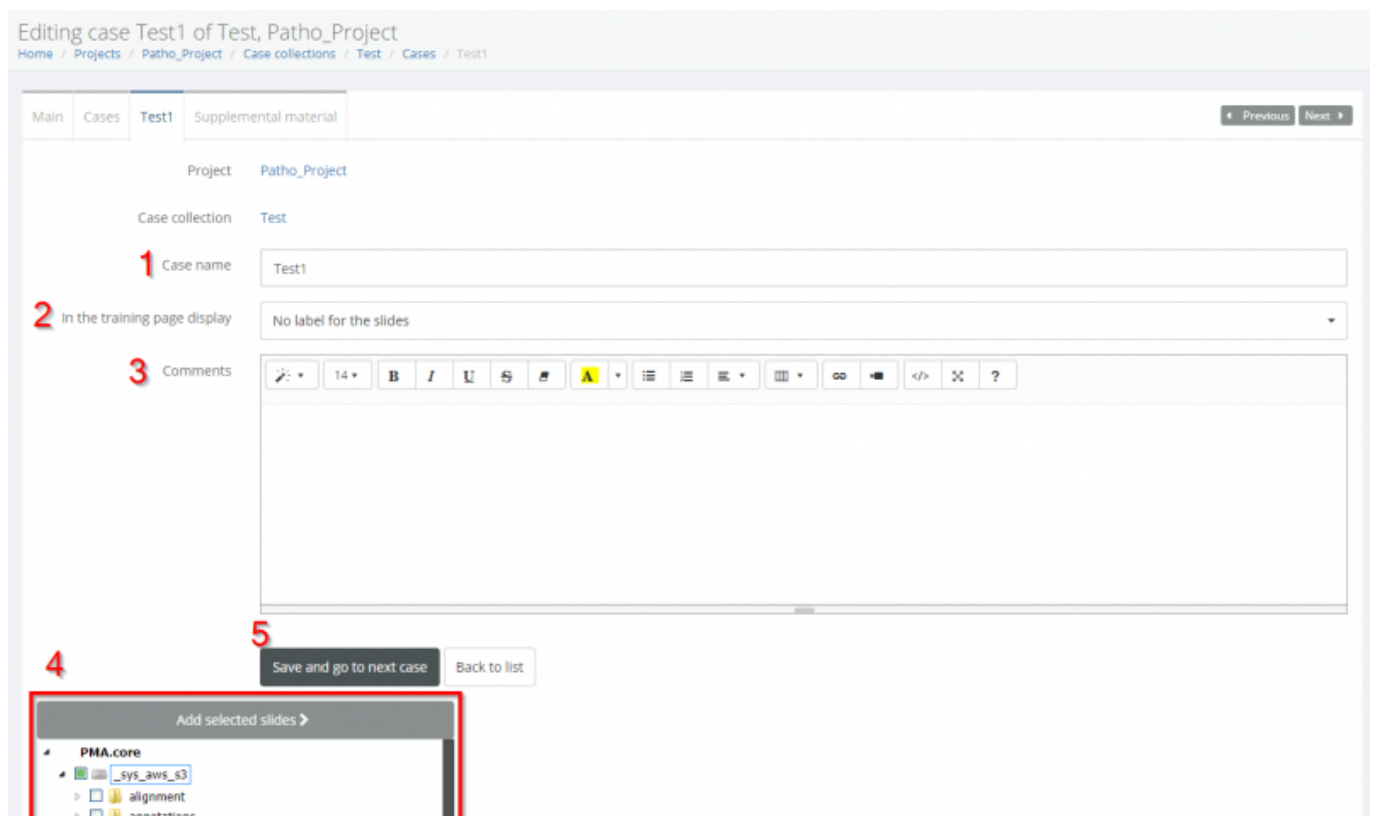

A Note on Adding Content

You can use any slides from your [slide inventory](#) in Pathotrainer case collections. The slide inventory is managed by PMA.core - for guides on how to import content there, you might find these links useful:

https://docs.pathomation.com/pma.core/3.0.2/doku.php?id=minimal_user_management
<https://docs.pathomation.com/pma.core/3.0.2/doku.php?id=upload>

From:
<https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link:
https://docs.pathomation.com/pathotrainer/doku.php?id=cases_and_case_collections&rev=1701688161

Last update: **2023/12/04 14:09**

