


Viewer Settings


In the Comment Fields Template page of the System section in the administrators sidebar, you can see existing templates or create new ones using the “Create” button in the upper right of the window.

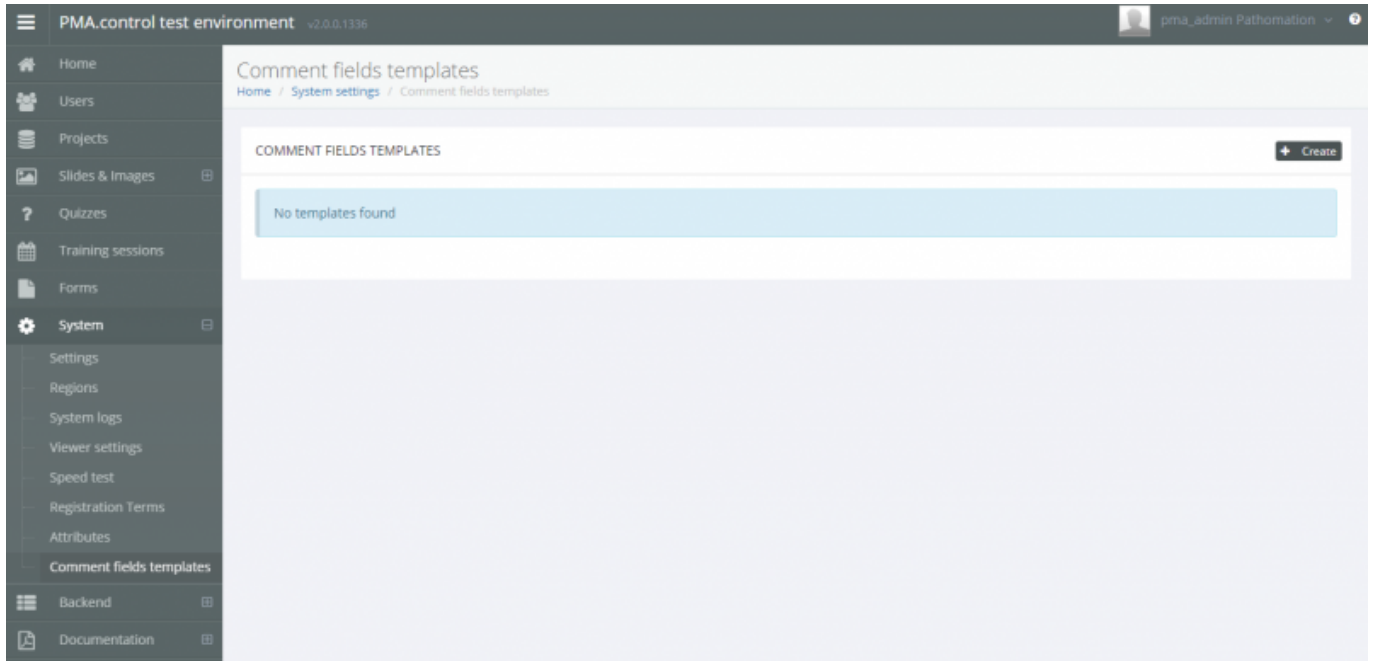

After clicking “Create” you can easily define the:

- Language
- Template Name
- Define and add field names and whether their score and/or report will be visible

When you're finished with your template, just click save

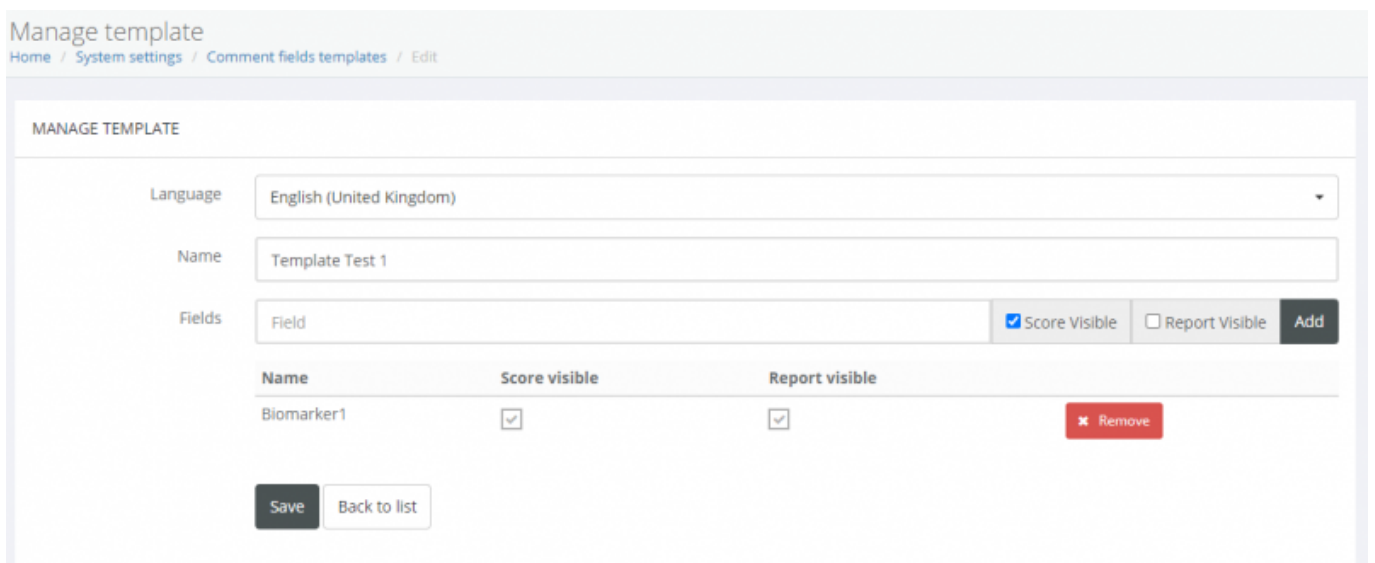

Last update: 2024/01/03 17:39 comment_fields_templates https://docs.pathomation.com/pathotrainer/doku.php?id=comment_fields_templates&rev=1704292793

From: <https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link: https://docs.pathomation.com/pathotrainer/doku.php?id=comment_fields_templates&rev=1704292793

Last update: **2024/01/03 17:39**

