


# Create a project

As a project owner you decide on the name, the content and the way participants/trainees can interact with the slides.

[pma.control\\_2.0\\_create\\_a\\_project.mp4](#)

First navigate to the Projects tab in the sidebar, and select “Create”

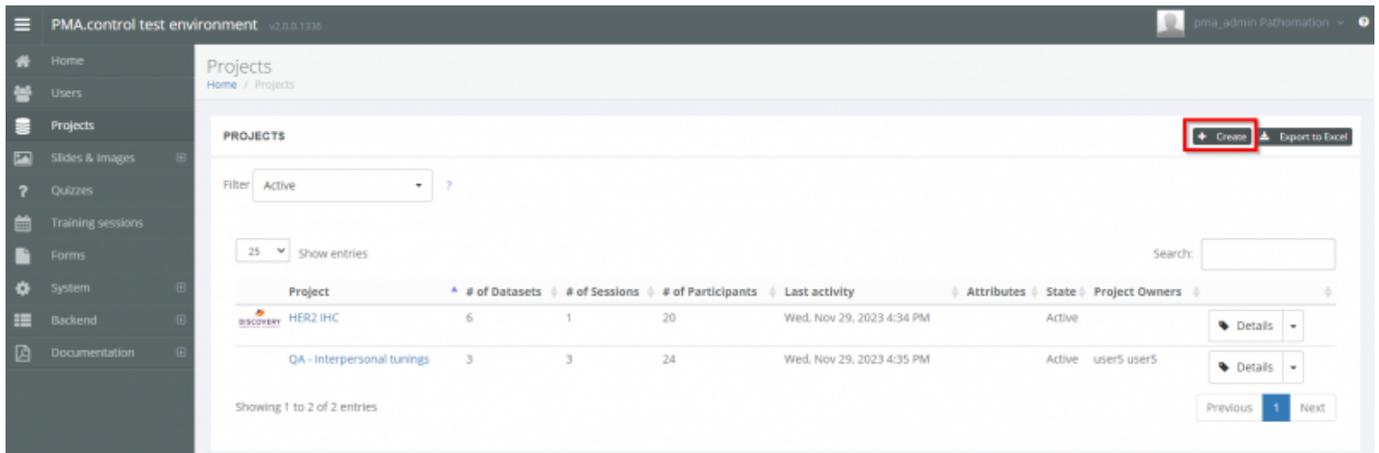

In this screen you can:

1. Upload a logo image
2. Give your project a title
3. Describe your project
4. Give contact information for the project owner
5. Choose between Modern, Default and Classic themes
6. Toggle what permissions users will have e.g. being able to edit clinical information - you can hover the mouse over the '?' icon to get more information about each option
7. Toggle attributes e.g. is this a billable project

The screenshot shows a web form titled "Create new project" with a breadcrumb "Home / Projects / Create". The form contains several fields and a list of checkboxes. Red numbers 1 through 8 are overlaid on the form to highlight specific elements:

- 1: Logo field (upload icon, "No file selected", "Max file size: 10Mb")
- 2: Title field (text: "Patho\_Project")
- 3: Description field (text: "A project to show how easy it is to start a project!")
- 4: Contact info field (empty)
- 5: Viewer theme field (text: "Modern")
- 6: Allowed interaction modes field (checkboxes: Board, Browse, Clinical information edit, Consensus score edit, Consensus view, Hidden, Locked, Review, Self review, Self test, Test active)
- 7: Attributes field (checkboxes: Billable, attribute nr 1, attribute nr 2)
- 8: "Create" button (highlighted with a red box)

Here you can see the popup menu when you mouseover the '?' next to (6):

**Board:** Collaboration mode is enabled with chat functionality  
**Browse:** Only the slides are displayed  
**Clinical information edit:** Enter the clinical information about the case collection  
**Consensus score edit:** The user is presented with a form where they can enter the consensus score values. Also the values of other users can be reviewed.  
**Consensus view:** The consensus score is displayed to everyone  
**Hidden:** The case collection is not visible  
**Locked:** The case collection is locked  
**Review:** The consensus score, as well as each user's individual score is displayed to everyone  
**Self review:** The consensus score, as well as the user's own score is displayed.  
**Self test:** A form is displayed to the user, to score for each case. The correct answers are immediately displayed upon submitting the form for each case.  
**Test active:** A form is displayed to the user, to score for each case.

Allowed interaction modes 

- Board
- Browse

From:  
<https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link:  
[https://docs.pathomation.com/pathotrainer/doku.php?id=create\\_a\\_project&rev=1701346752](https://docs.pathomation.com/pathotrainer/doku.php?id=create_a_project&rev=1701346752)

Last update: **2023/11/30 15:19**

