


# Create A Project

As a project owner you decide on the name, the content and the way participants/trainees can interact with the slides.

[pma.control\\_2.0\\_create\\_a\\_project.mp4](#)

First navigate to the Projects tab in the sidebar, and select “Create”

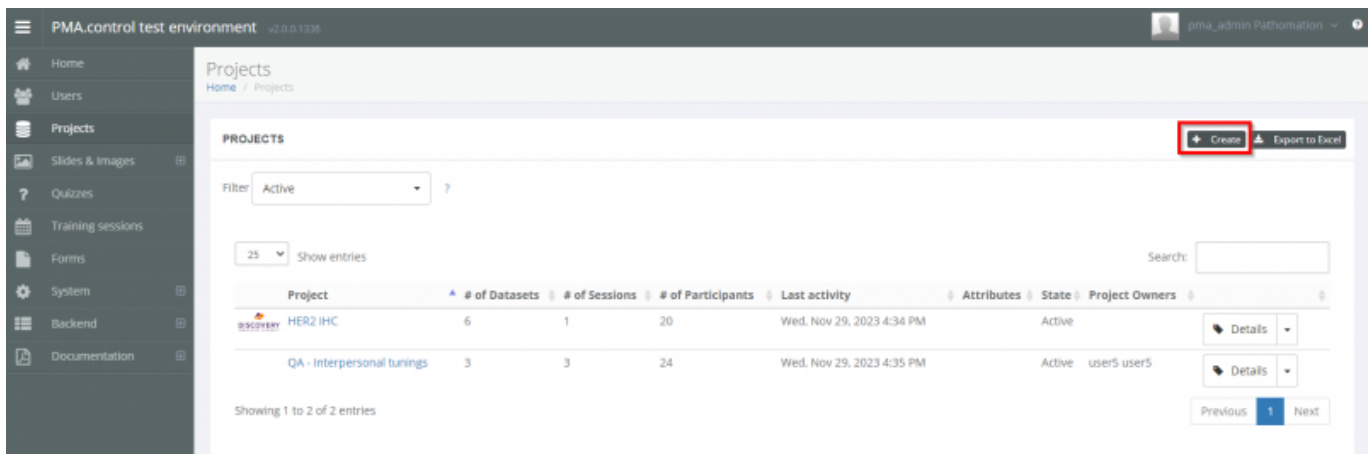

In this screen you can:

1. Upload a logo image
2. Give your project a title
3. Describe your project
4. Give contact information for the project owner
5. Choose between Modern, Default and Classic themes
6. Toggle what permissions users will have e.g. being able to edit clinical information - you can hover the mouse over the '?' icon to get more information about each option (see end of page)
7. Toggle attributes e.g. is this a billable project
8. Once you're done, hit this to create the project - you will be taken to the editing page

The screenshot shows a 'Create new project' form with the following fields and annotations:

- 1** Logo: A file upload field with the text 'No file selected' and 'Max file size: 10Mb'.
- 2** Title: A text input field containing 'Patho\_Project'.
- 3** Description: A text area containing 'A project to show how easy it is to start a project!'.
- 4** Contact info: An empty text area.
- 5** Viewer theme: A dropdown menu showing 'Modern'.
- 6** Allowed interaction modes: A list of checkboxes, all of which are checked:
  - Board
  - Browse
  - Clinical information edit
  - Consensus score edit
  - Consensus view
  - Hidden
  - Locked
  - Review
  - Self review
  - Self test
  - Test active
- 7** Attributes: A list of checkboxes, all of which are unchecked:
  - Billable
  - attribute nr 1
  - attribute nr 2
- 8** Create: A button labeled 'Create' next to a 'Back to list' button. The 'Create' button is highlighted with a red box.

Here you can see the popup menu when you mouseover the '?' next to (6):

**Board:** Collaboration mode is enabled with chat functionality  
**Browse:** Only the slides are displayed  
**Clinical information edit:** Enter the clinical information about the case collection  
**Consensus score edit:** The user is presented with a form where they can enter the consensus score values. Also the values of other users can be reviewed.  
**Consensus view:** The consensus score is displayed to everyone  
**Hidden:** The case collection is not visible  
**Locked:** The case collection is locked  
**Review:** The consensus score, as well as each user's individual score is displayed to everyone  
**Self review:** The consensus score, as well as the user's own score is displayed.  
**Self test:** A form is displayed to the user, to score for each case. The correct answers are immediately displayed upon submitting the form for each case.  
**Test active:** A form is displayed to the user, to score for each case.

Allowed interaction modes ?

- Board
- Browse

From:  
<https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link:  
[https://docs.pathomation.com/pathotrainer/doku.php?id=create\\_a\\_project&rev=1701347577](https://docs.pathomation.com/pathotrainer/doku.php?id=create_a_project&rev=1701347577)

Last update: **2023/11/30 15:32**

