


Create A Project

As a project owner you decide on the name, the content and the way participants/trainees can interact with the slides.

[pma.control_2.0_create_a_project.mp4](#)

First navigate to the Projects tab in the sidebar, and select “Create”

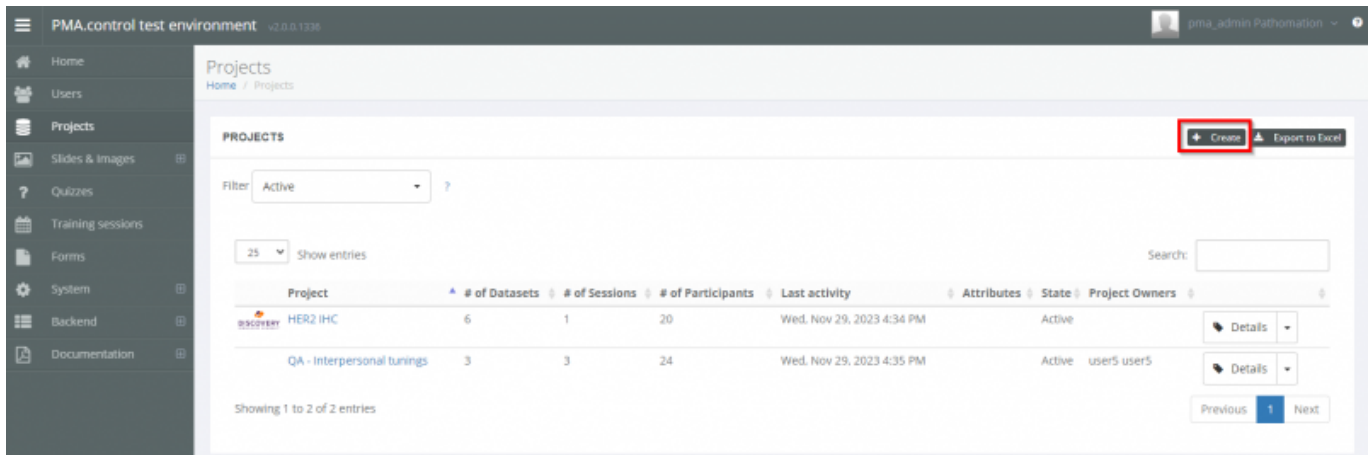

In the Create New Project screen you can:


1. Upload a logo image
2. Give your project a title
3. Describe your project
4. Give contact information for the project owner
5. Choose between Modern, Default and Classic themes
6. Toggle what permissions users will have e.g. being able to edit clinical information - you can hover the mouse over the '?' icon to get more information about each option (see end of page)
7. Toggle attributes e.g. is this a billable project
8. Once you're done, hit this to create the project - you will be taken to the editing page

The screenshot shows a web form titled "Create new project" with a breadcrumb "Home / Projects / Create". The form contains several fields and a list of checkboxes, each annotated with a red number:

- 1** Logo: A file upload field with "No file selected" and "Max file size: 10Mb".
- 2** Title: A text input field containing "Patho_Project".
- 3** Description: A text area containing "A project to show how easy it is to start a project!".
- 4** Contact info: An empty text area.
- 5** Viewer theme: A dropdown menu showing "Modern".
- 6** Allowed interaction modes: A list of checkboxes, all of which are checked. A question mark icon is next to the label.
- 7** Attributes: A list of checkboxes, all of which are unchecked.
- 8** Create: A button labeled "Create" is highlighted with a red box. A "Back to list" button is also visible.

Here you can see the popup menu when you mouseover the '?' next to (6):

Board: Collaboration mode is enabled with chat functionality
Browse: Only the slides are displayed
Clinical information edit: Enter the clinical information about the case collection
Consensus score edit: The user is presented with a form where they can enter the consensus score values. Also the values of other users can be reviewed.
Consensus view: The consensus score is displayed to everyone
Hidden: The case collection is not visible
Locked: The case collection is locked
Review: The consensus score, as well as each user's individual score is displayed to everyone
Self review: The consensus score, as well as the user's own score is displayed.
Self test: A form is displayed to the user, to score for each case. The correct answers are immediately displayed upon submitting the form for each case.
Test active: A form is displayed to the user, to score for each case.

Allowed interaction modes 

- Board
- Browse

From:
<https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link:
https://docs.pathomation.com/pathotrainer/doku.php?id=create_a_project&rev=1701347623

Last update: **2023/11/30 15:33**

