


Creating Cases and Case Collections

Case collections are how organize your content to be shown in courses.

Navigating to the Slides and Images tab on the sidebar, and then selecting Case Collections will bring you to this tab. Here you can see an overview of existing case collections (if any) and can click the create button to add a new one.

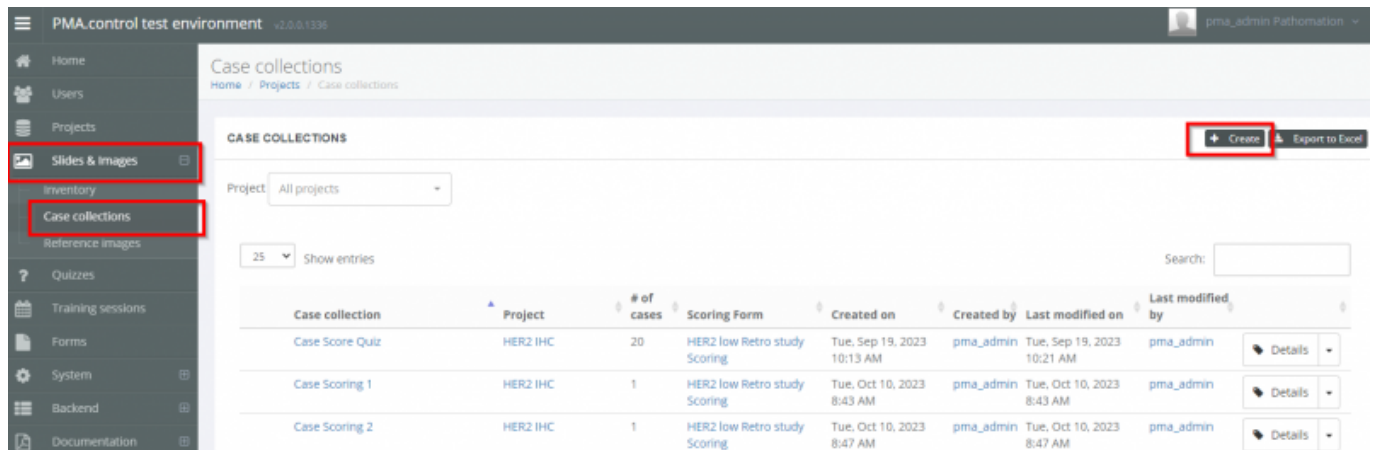

Case collection	Project	# of cases	Scoring Form	Created on	Created by	Last modified on	Last modified by	
Case Score Quiz	HER2 IHC	20	HER2 low Retro study Scoring	Tue, Sep 19, 2023 10:13 AM	pma_admin	Tue, Sep 19, 2023 10:21 AM	pma_admin	Details
Case Scoring 1	HER2 IHC	1	HER2 low Retro study Scoring	Tue, Oct 10, 2023 8:43 AM	pma_admin	Tue, Oct 10, 2023 8:43 AM	pma_admin	Details
Case Scoring 2	HER2 IHC	1	HER2 low Retro study Scoring	Tue, Oct 10, 2023 8:47 AM	pma_admin	Tue, Oct 10, 2023 8:47 AM	pma_admin	Details

A Note on Adding Content

You can use any slides from your [slide inventory](#) in Pathotrainer case collections. The slide inventory is managed by PMA.core - for guides on how to import content there, you might find these links useful:

https://docs.pathomation.com/pma.core/3.0.2/doku.php?id=minimal_user_management

<https://docs.pathomation.com/pma.core/3.0.2/doku.php?id=upload>

Creating a New Case Collection

After hitting the “Create” button, you will be presented with the option to define some details about your new collection:

1. Uploading a logo image
2. Assigning the case to a project
3. Giving the case a name
4. Assigning a scoring form for your trainees
5. Assigning a clinical information form for your trainees
6. Options to toggle slide label and measurement visibility
7. Additional comment templates

Create new case collection

[Home](#) / [Projects](#) / [Create](#)

- 1 Logo
Max file size: 10Mb
- 2 Project
- 3 Title
- 4 Scoring Form
- 5 Clinical information form
- 6 Show slide labels
 Show measurements
- 7 Case comments templates

From: <https://docs.pathomation.com/pathotrainer/> - Pathotrainer

Permanent link: https://docs.pathomation.com/pathotrainer/doku.php?id=create_cases_and_case_collections&rev=1701446247

Last update: 2023/12/01 18:57

