

Creating Cases and Case Collections

Case collections are how organize your content to be shown in courses.

Navigating to the Slides and Images tab on the sidebar, and then selecting Case Collections will bring you to this tab. Here you can see an overview of existing case collections (if any) and can click the create button to add a new one.

Case collection	Project	# of cases	Scoring Form	Created on	Created by	Last modified on	Last modified by	
Case Score Quiz	HER2 IHC	20	HER2 low Retro study Scoring	Tue, Sep 19, 2023 10:13 AM	pma_admin	Tue, Sep 19, 2023 10:21 AM	pma_admin	Details
Case Scoring 1	HER2 IHC	1	HER2 low Retro study Scoring	Tue, Oct 10, 2023 8:43 AM	pma_admin	Tue, Oct 10, 2023 8:43 AM	pma_admin	Details
Case Scoring 2	HER2 IHC	1	HER2 low Retro study Scoring	Tue, Oct 10, 2023 8:47 AM	pma_admin	Tue, Oct 10, 2023 8:47 AM	pma_admin	Details

Creating a New Case Collection

After hitting the “Create” button, you will be presented with the option to define some details about your new collection:

1. Uploading a logo image
2. Assigning the case to a project
3. Giving the case a name
4. Assigning a scoring form for your trainees
5. Assigning a clinical information form for your trainees
6. Options to toggle slide label and measurement visibility
7. Additional comment templates

Create new case collection

Home / Projects / Create

1 Logo
Max file size: 10Mb

2 Project

3 Title

4 Scoring Form

5 Clinical information form

6 Show slide labels


6 Show measurements

7 Case comments templates

Adding Cases

Now that you've created a case, its time to add some cases.


To do this, navigate to the “Cases” tab and click the “Create”

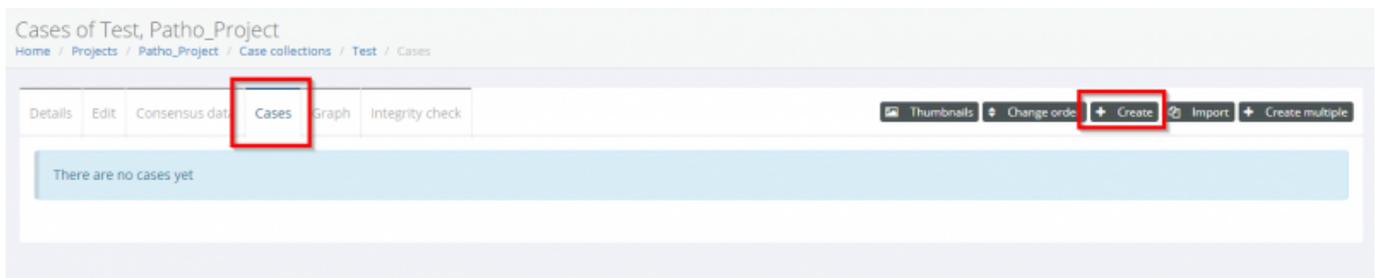

You will find yourself in the Editing Cases tab.

Here you can set up various case properties:


1. A name for the case
2. Whether to display the slide filename, caption, or no label to the participants
3. Comments
4. Select your slide(s) from your PMA core inventory via the file treeview
5. Save your slides

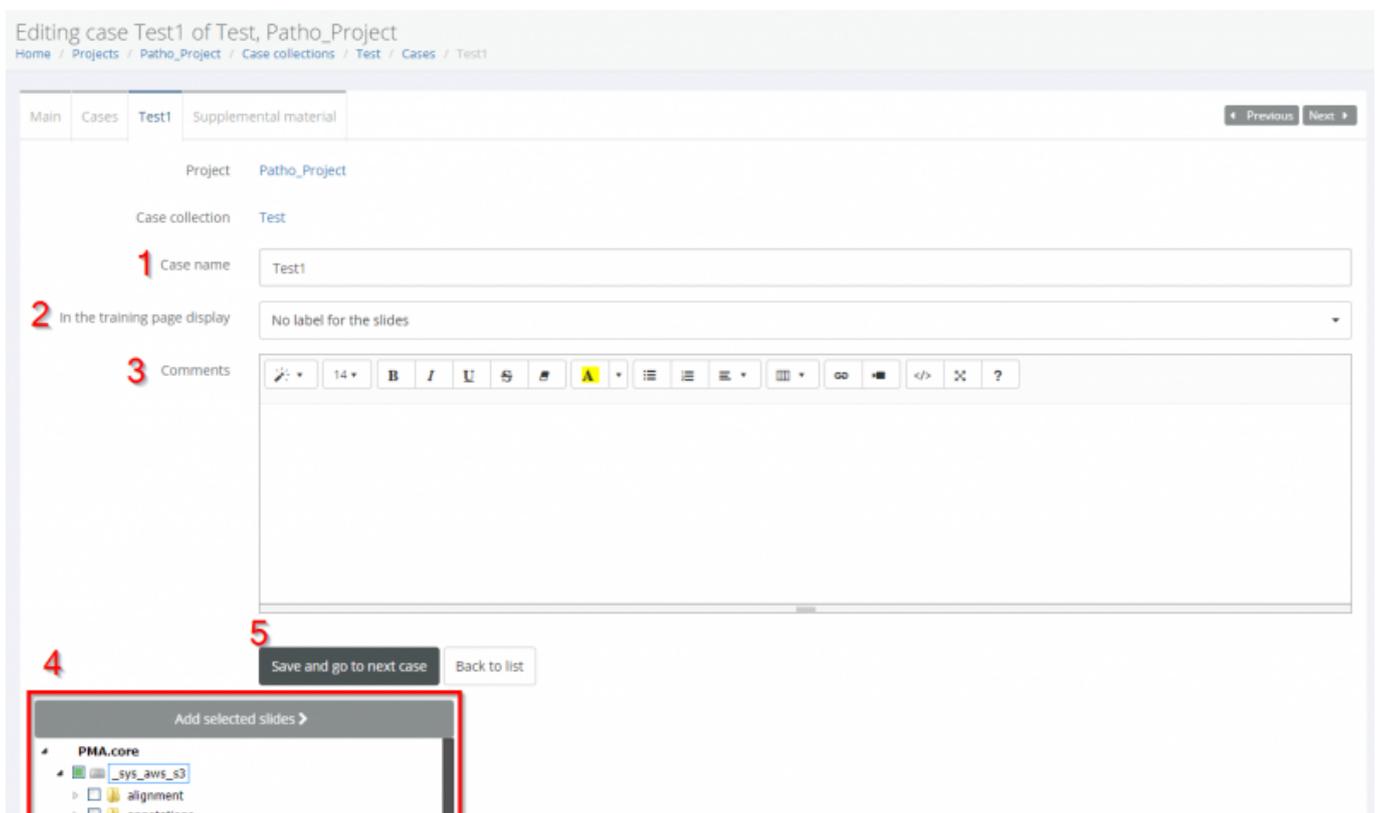

Selecting slides to add

To select slides, navigate the treeview and click the checkmark next to your desired slides (1). These can be from several different folders, PMA.cores, or servers. You can hover your mouse over the

filename to preview a thumbnail of the image. When you've picked them, click 'Add selected slides' (2).

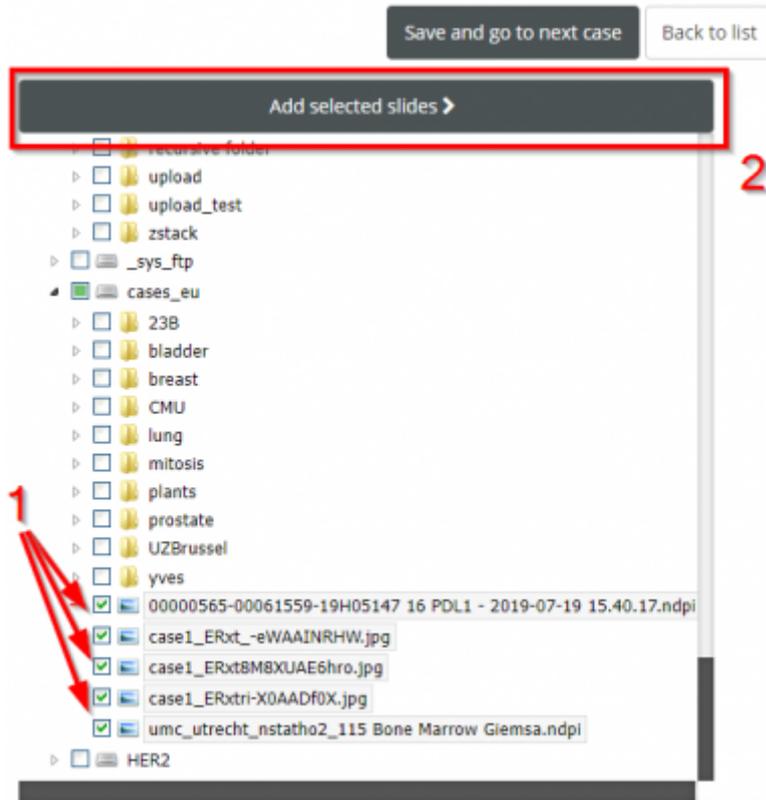

Once the slides are added you can check and change several details about them

1. The filename
2. The path
3. The caption - you can edit this which is useful if you want to give different labels to your participants and only display the caption
4. Add comments
5. See and modify annotations on the slide

You can also click the 'X' button to remove a slide from the collection

The screenshot displays the Pathotrainer interface. On the left, a slide inventory is shown with a tree view. The 'cases_eu' folder is expanded, showing several slide files. Three files are selected: '00000565-00061559-19H05147 16 PDL1 - 2019-07-19 15.40.17.ndpi', 'case1_ERxt_eWAAINRHW.jpg', 'case1_ERxt8MBXUAE6hro.jpg', and 'case1_ERxt8MBXUAE6hro.jpg'. On the right, three slide preview panels are shown, each with a histology image and a form for Path, Caption, and Comments. The panels are titled 'SLIDE2_20210525_130214_SPECIMEN.JPG', 'CASE1_ERXT_eWAAINRHW.JPG', and 'CASE1_ERXT8MBXUAE6hro.JPG'. Red numbers 1-5 are overlaid on the interface: 1 points to the inventory, 2 to the Path field, 3 to the Caption field, 4 to the Comments field, and 5 to the Annotations button.

A Note on Adding Content

You can use any slides from your [slide inventory](#) in Pathotrainer case collections. The slide inventory is managed by PMA.core - for guides on how to import content there, you might find these links useful:

https://docs.pathomation.com/pma.core/3.0.2/doku.php?id=minimal_user_management
<https://docs.pathomation.com/pma.core/3.0.2/doku.php?id=upload>

From: <https://docs.pathomation.com/pathotrainer/> - Pathotrainer

Permanent link: https://docs.pathomation.com/pathotrainer/doku.php?id=create_cases_and_case_collections&rev=1701706854

Last update: 2023/12/04 19:20

