

Creating Cases and Case Collections

Case collections are how organize your content to be shown in courses.

Navigating to the Slides and Images tab on the sidebar, and then selecting Case Collections will bring you to this tab. Here you can see an overview of existing case collections (if any) and can click the create button to add a new one.

The screenshot shows the 'Case collections' page in the PMA control test environment. The sidebar on the left has 'Slides & Images' and 'Case collections' highlighted with red boxes. The main area displays a table of existing case collections. A '+ Create' button is highlighted with a red box in the top right corner.

Case collection	Project	# of cases	Scoring Form	Created on	Created by	Last modified on	Last modified by
Case Score Quiz	HER2 IHC	20	HER2 low Retro study Scoring	Tue, Sep 19, 2023 10:13 AM	pma_admin	Tue, Sep 19, 2023 10:21 AM	pma_admin
Case Scoring 1	HER2 IHC	1	HER2 low Retro study Scoring	Tue, Oct 10, 2023 8:43 AM	pma_admin	Tue, Oct 10, 2023 8:43 AM	pma_admin
Case Scoring 2	HER2 IHC	1	HER2 low Retro study Scoring	Tue, Oct 10, 2023 8:47 AM	pma_admin	Tue, Oct 10, 2023 8:47 AM	pma_admin

Creating a New Case Collection

After hitting the “Create” button, you will be presented with the option to define some details about your new collection:


1. Uploading a logo image
2. Assigning the case to a project
3. Giving the case a name
4. Assigning a scoring form for your trainees
5. Assigning a clinical information form for your trainees
6. Options to toggle slide label and measurement visibility
7. Additional comment templates

The screenshot shows the 'Create new case collection' form. The form has seven numbered steps: 1. Logo (upload button), 2. Project (dropdown), 3. Title (text input), 4. Scoring Form (dropdown), 5. Clinical information form (dropdown), 6. Show slide labels and Show measurements (checkboxes), and 7. Case comments templates (dropdown). The 'Create' button is at the bottom.

Adding Cases

Now that you've created a case, its time to add some cases.


To do this, navigate to the “Cases” tab and click the “Create”

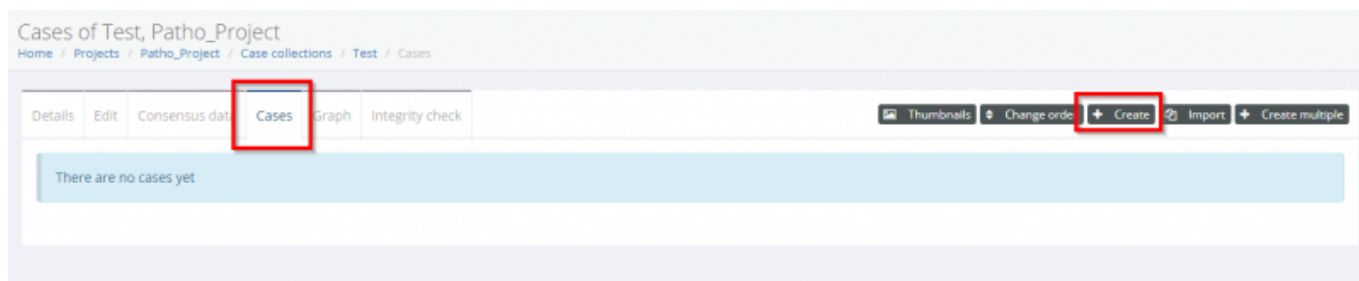

You will find yourself in the Editing Cases tab.

Here you can set up various case properties:


1. A name for the case
2. Whether to display the slide filename, caption, or no label to the participants
3. Comments
4. Select your slide(s) from your PMA core inventory via the file treeview
5. Save your slides

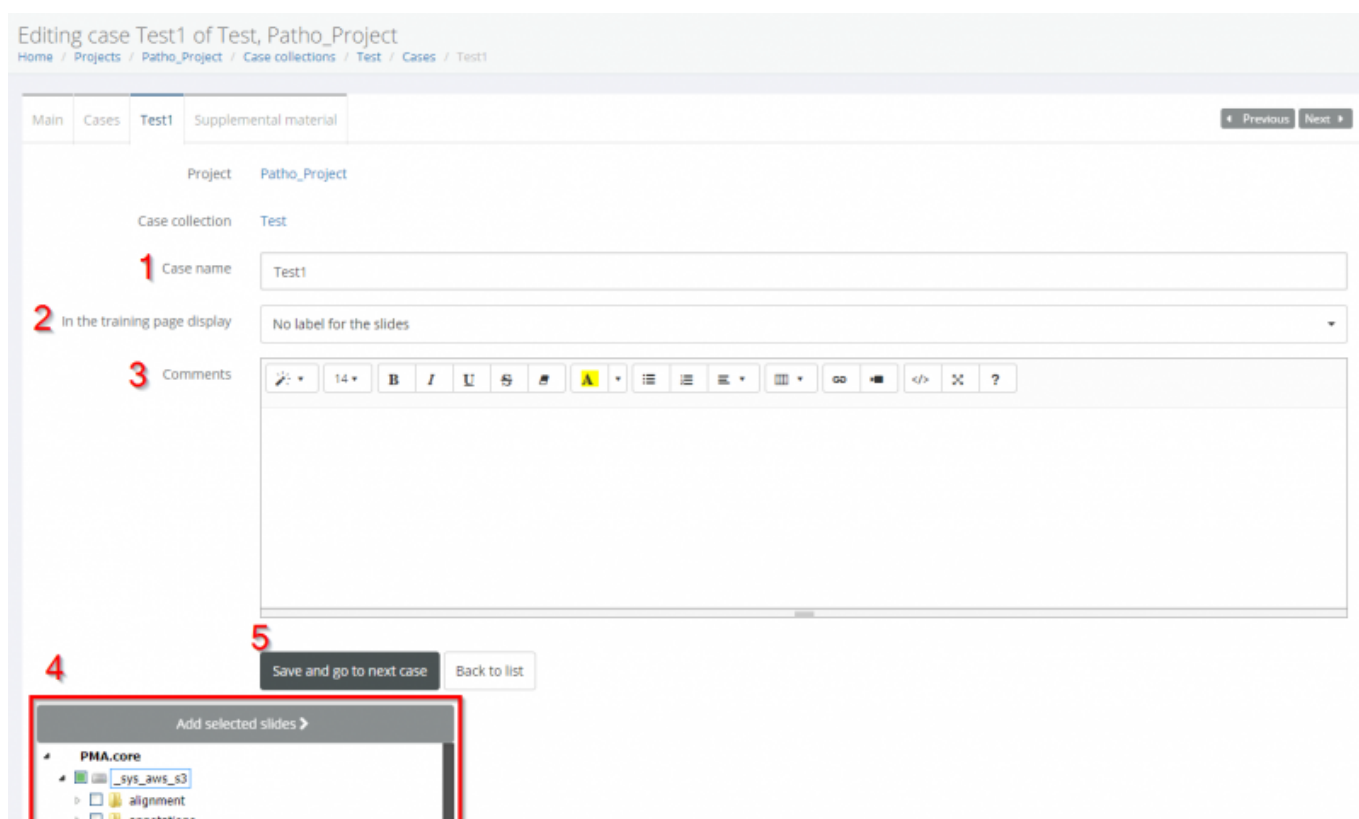

Selecting slides to add

To select slides, navigate the treeview and click the checkmark next to your desired slides (1). These can be from several different folders, PMA.cores, or servers. You can hover your mouse over the


filename to preview a thumbnail of the image. When you've picked them, click 'Add selected slides' (2).

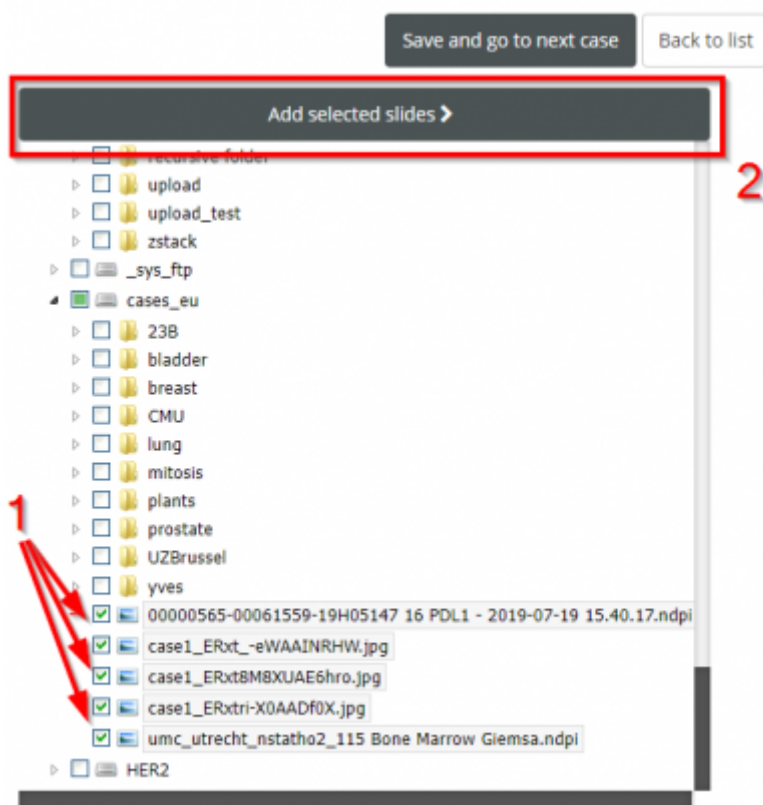

Once the slides are added you can check and change several details about them

1. The filename
2. The path
3. The caption - you can edit this which is useful if you want to give different labels to your participants and only display the caption
4. Add comments
5. See and modify annotations on the slide

You can also click the 'X' button to remove a slide from the collection

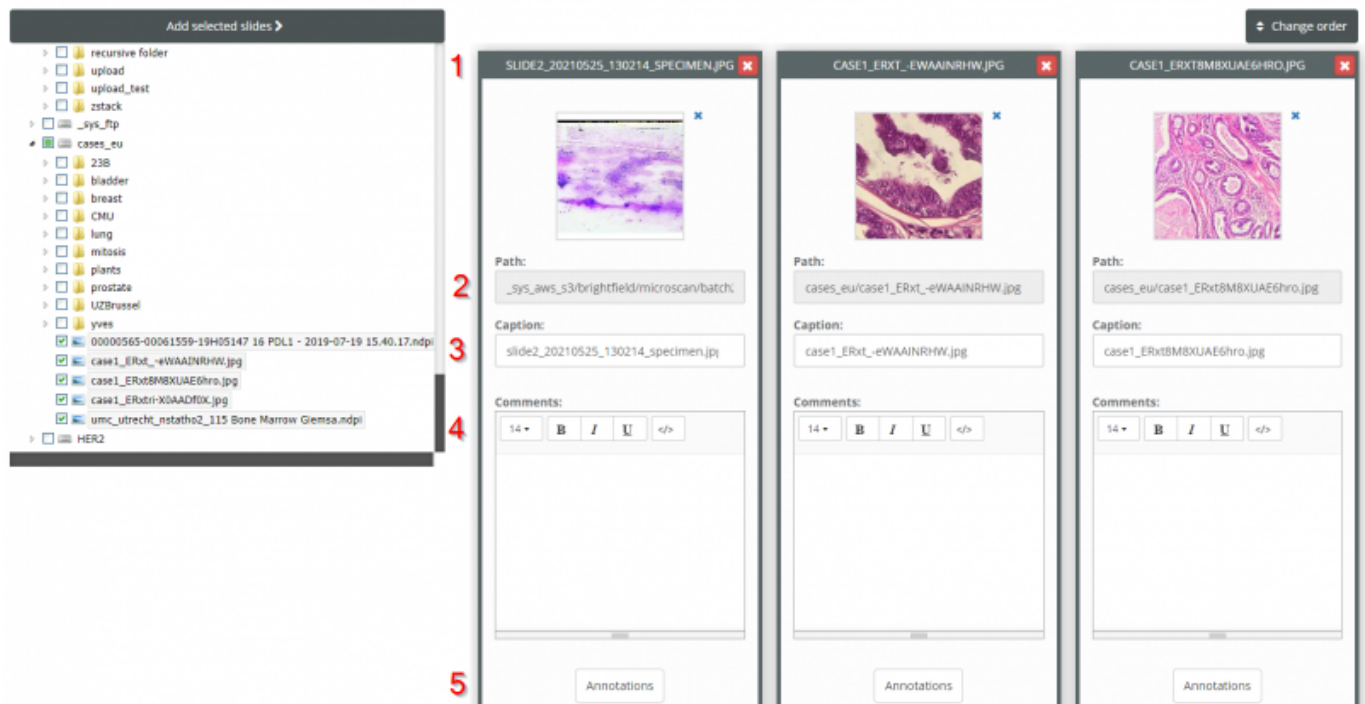

A Note on Adding Content

You can use any slides from your [slide inventory](#) in Pathotrainer case collections. The slide inventory is managed by PMA.core - for guides on how to import content there, you might find these links useful:

https://docs.pathomation.com/pma.core/3.0.2/doku.php?id=minimal_user_management

<https://docs.pathomation.com/pma.core/3.0.2/doku.php?id=upload>

From:
<https://docs.pathomation.com/pathotrainer/> - Pathotrainer

Permanent link:
https://docs.pathomation.com/pathotrainer/doku.php?id=create_cases_and_case_collections&rev=1704288411

Last update: 2024/01/03 16:26

