


Custom Annotation Tools


After selecting the Annotations toolbar tab when [editing a project](#) you will be able to define custom annotation tools. This is useful if you want all project users to use the same indications e.g. all macrophages are marked with blue freehand annotations, or if you want quick access to particular styles of annotation with fewer clicks.

Here you can set the name of the tool by it's style e.g. "Red freehand" or it's what it's intended to annotate e.g. "Melanoma"

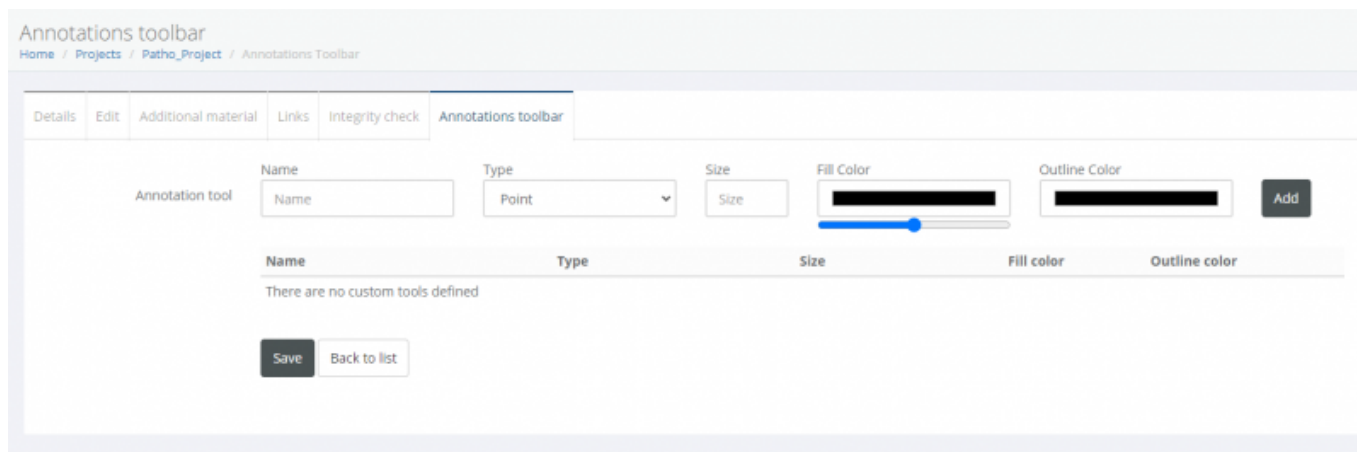

The screenshot shows the 'Annotations toolbar' interface. At the top, there are tabs: Details, Edit, Additional material, Links, Integrity check, and Annotations toolbar. Below the tabs, there are input fields for 'Name' (with a placeholder 'Name'), 'Type' (a dropdown menu set to 'Point'), 'Size' (a text input with 'Size'), 'Fill Color' (a color picker with a blue slider), and 'Outline Color' (a color picker with a black slider). An 'Add' button is to the right of the 'Outline Color' picker. Below these fields is a table with the following headers: Name, Type, Size, Fill color, and Outline color. The table body contains the text 'There are no custom tools defined'. At the bottom left, there are 'Save' and 'Back to list' buttons.

From the dropdown menu you can select what drawing behaviour the annotation tool will have

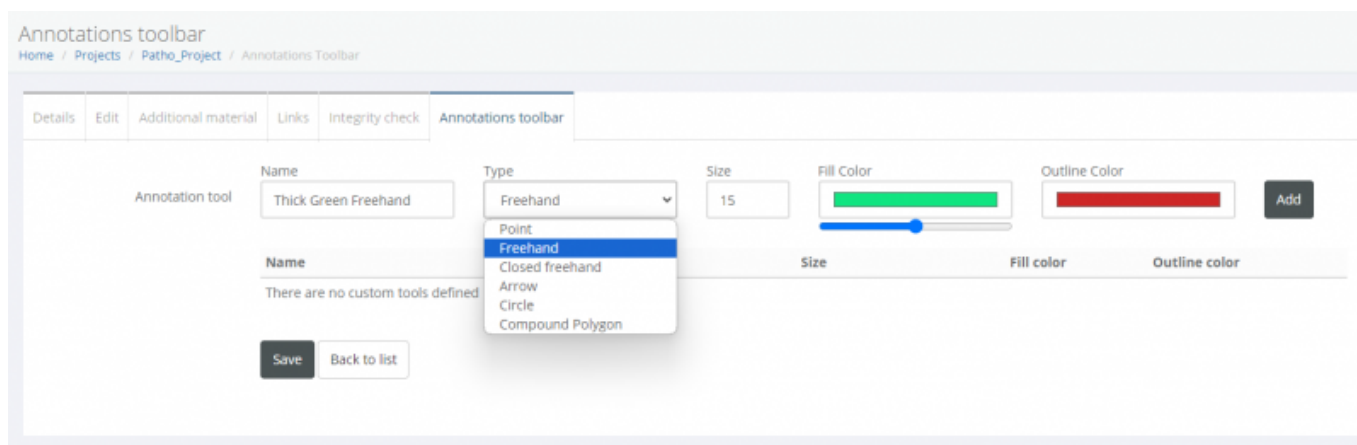

This screenshot is similar to the previous one, but the 'Type' dropdown menu is open, showing a list of options: Freehand, Point, Freehand (highlighted in blue), Closed freehand, Arrow, Circle, and Compound Polygon. The 'Name' field now contains 'Thick Green Freehand'. The 'Size' field is set to '15'. The 'Fill Color' picker shows a green color, and the 'Outline Color' picker shows a red color. The 'Add' button is still present to the right of the 'Outline Color' picker. The table below still shows 'There are no custom tools defined'.

You can select the RGB colour by using the slider + colour map, or by precisely inputting numerical RGB values

Last update:

2023/11/30 17:16 custom_annotation_tools https://docs.pathomation.com/pathotrainer/doku.php?id=custom_annotation_tools&rev=1701353792

Annotations toolbar

[Home](#) / [Projects](#) / [Patho_Project](#) / Annotations Toolbar

Details Edit Additional material Links Integrity check Annotations toolbar


Annotation tool Name Type Size Fill Color Outline Color Add

Thick Green Freehand Freehand 15

Name Type Size Fill Color Outline color

There are no custom tools defined

Save Back to list

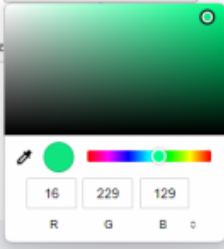

From:

<https://docs.pathomation.com/pathotrainer/> - Pathotrainer

Permanent link:

https://docs.pathomation.com/pathotrainer/doku.php?id=custom_annotation_tools&rev=1701353792

Last update: 2023/11/30 17:16

