



Edit A Project

After hitting "Create" in the [Create a Project](#) menu, or navigating to an existing project by clicking on a project's name (1) or clicking "Edit" from the dropdown menu, you will be taken to the editing page.

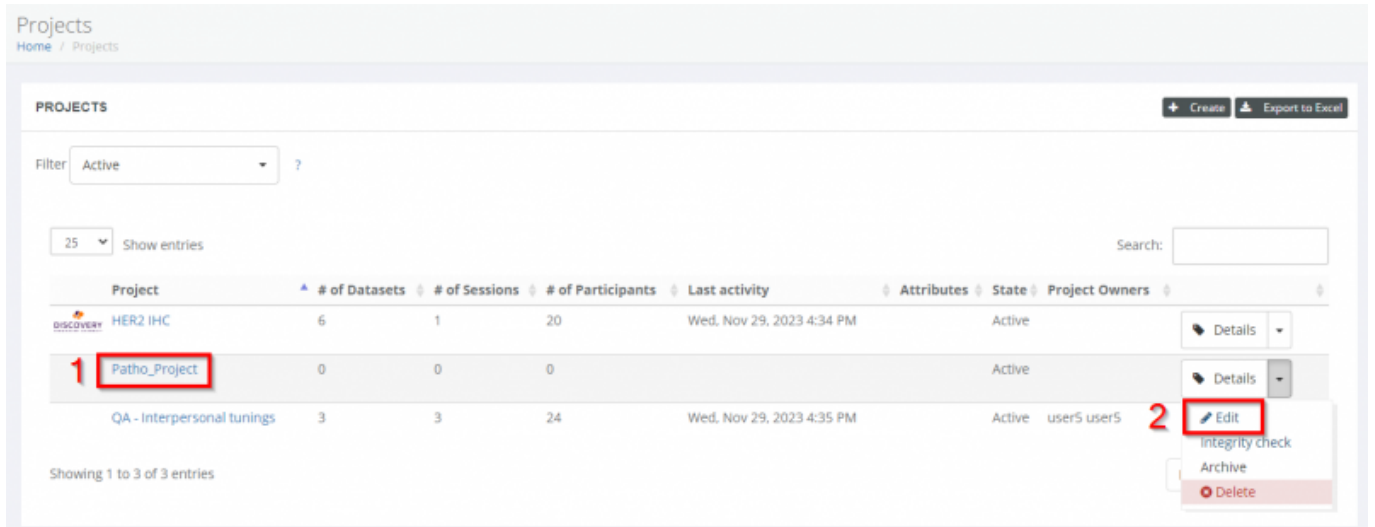

Here you have access to several tabs which allow you to:

- 1. Ordered List Item see an overview of the project Details
- 2. Edit the project options chosen during [Create a project](#)
- 3. Add additional material
- 4. Add links
- 5. Perform integrity checks
- 6. Create custom annotations

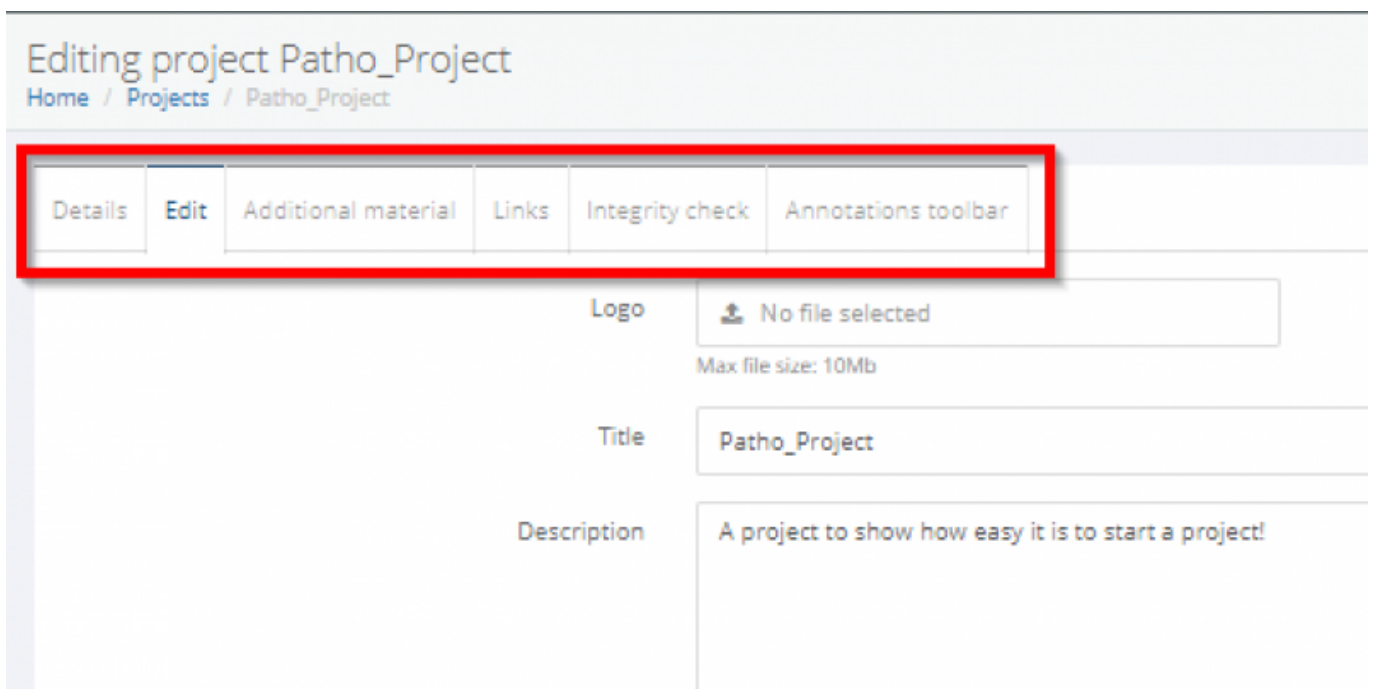

Project Details

Here you can get an overview of what content and training sessions is already in the project. If no content has been added yet, you are made aware by the orange text and can click the bluetext link to get started

Patho_Project

[Home](#) / [Projects](#) / [Patho_Project](#)

Details	Edit	Additional material	Links	Integrity check	Annotations toolbar
----------------	------	---------------------	-------	-----------------	---------------------

Title **Patho_Project**

Logo

State **Active**

Description A project to show how easy it is to start a project!

Contact info

Viewer theme **Modern**

Project Owners

Case collections There are no case collections in this project [Setup a case collection](#)

Training sessions There are no training sessions in this project [Setup a training session](#)

Allowed interaction modes ?

- Board
- Browse
- Clinical information edit
- Consensus score edit
- Consensus view
- Hidden
- Locked
- Review
- Self review
- Self test
- Test active

[Back to list](#) [Archive](#)

From:
<https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link:
https://docs.pathomation.com/pathotrainer/doku.php?id=edit_a_project&rev=1701348695

Last update: **2023/11/30 15:51**

