



Edit A Project

After hitting "Create" in the [Create a Project](#) menu, or navigating to an existing project by clicking on a project's name (1) or clicking "Edit" from the dropdown menu, you will be taken to the editing page.

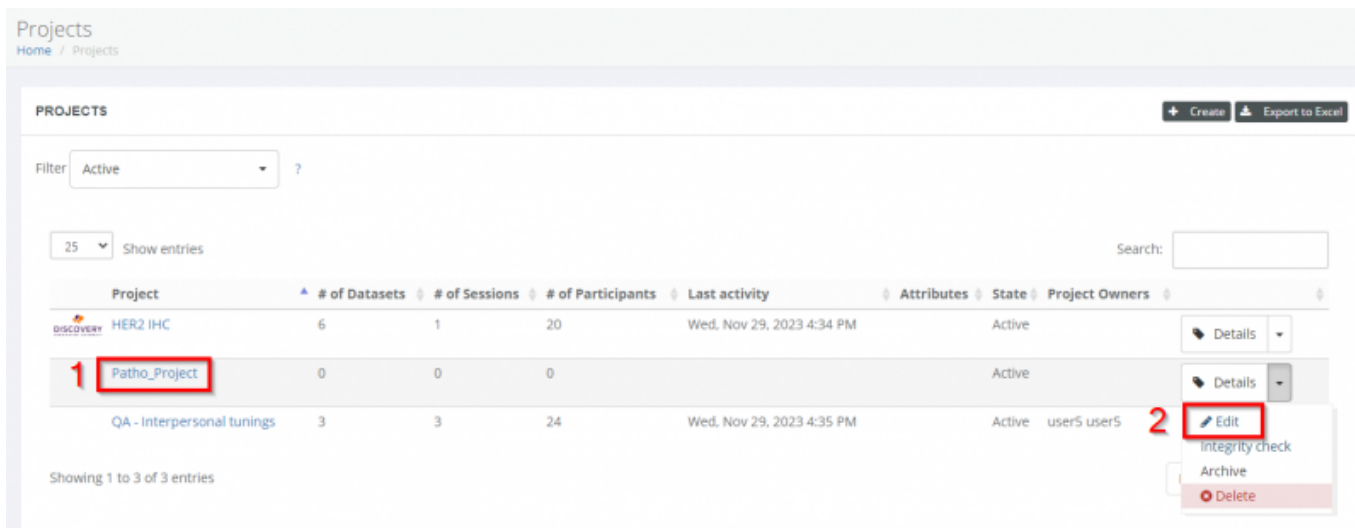

Here you have access to several tabs which allow you to:

- 1. Ordered List Item see an overview of the project Details
- 2. Edit the project options chosen during [Create a project](#)
- 3. Add additional material
- 4. Add links
- 5. Perform [integrity checks](#)
- 6. Create custom annotations

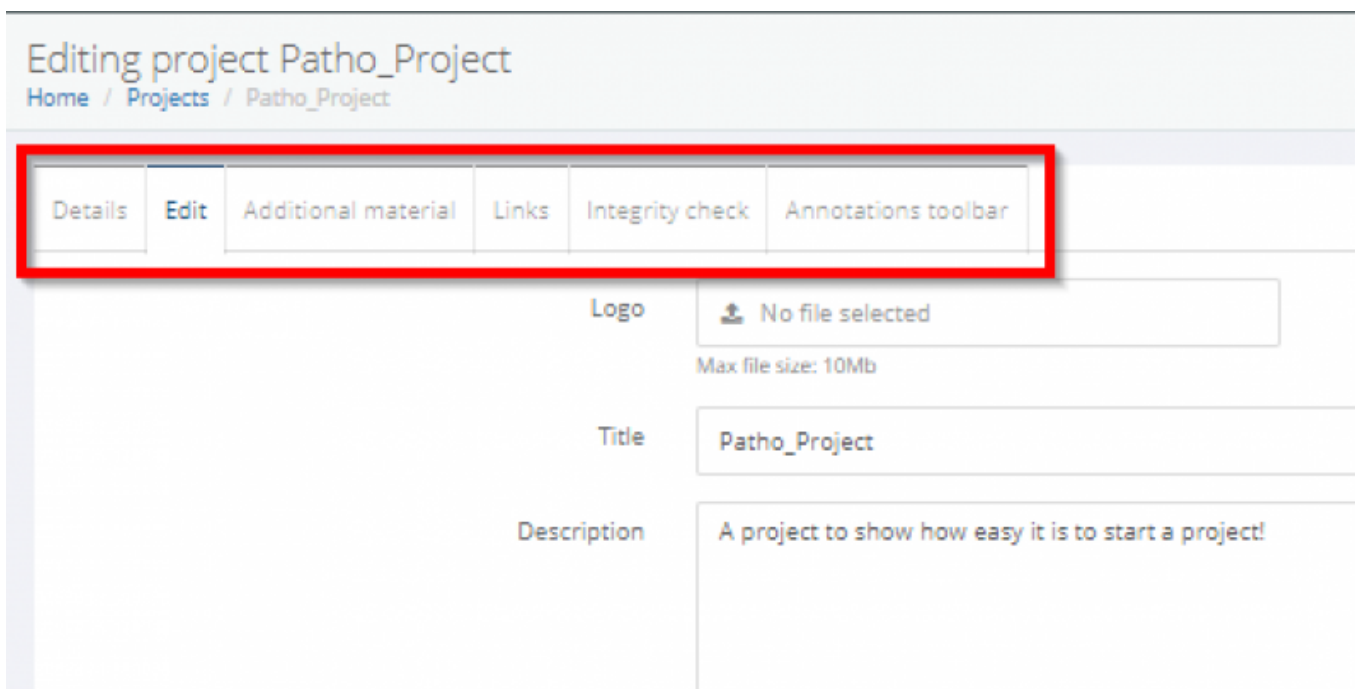

Project Details

Here you can get an overview of what content and training sessions is already in the project. If no content has been added yet, you are made aware by the orange text and can click the bluetext link to get started

Patho_Project
Home / Projects / Patho_Project

Details Edit Additional material Links Integrity check Annotations toolbar

Title **Patho_Project**

Logo

State **Active**

Description A project to show how easy it is to start a project!

Contact info

Viewer theme **Modern**

Project Owners

Case collections **There are no case collections in this project** [Setup a case collection](#)

Training sessions **There are no training sessions in this project** [Setup a training session](#)

Allowed interaction modes ?

- Board
- Browse
- Clinical information edit
- Consensus score edit
- Consensus view
- Hidden
- Locked
- Review
- Self review
- Self test
- Test active

[Back to list](#) [Archive](#)

Adding Material and Links

In these tabs you can add material and links, and see anything that has already been added

Additional material for Patho_Project

Home / Projects / Patho_Project / Additional material

Details Edit **Additional material** Links Integrity check

+ Add Recycle bin

There are no attachments

Links of Patho_Project

Home / Projects / Patho_Project / Links

Details Edit Additional material **Links** Integrity check

+ Add

Filter Patho_Project

There are no links

From:
<https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link:
https://docs.pathomation.com/pathotrainer/doku.php?id=edit_a_project&rev=1701353100

Last update: **2023/11/30 17:05**

