

- Hidden - the case collection is not visible
- Locked - the case collection is locked
- Browse - only the slides are displayed
- Test active - a form is displayed to the participant for each case
- Self test - a form is displayed to the participant for each case. The correct answers are immediately displayed upon submitting the form.
- Review - the consensus score as well as all user's individual scores are displayed
- Self review - the consensus score as well the user's own score is displayed

#### Advanced interaction modes (for supervisors only)

- Clinical information edit: enter the clinical information data of this case
- Consensus score edit: the user is presented with a form where they can enter the consensus score values. All from data from all participants is shown.

From:

<https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link:

[https://docs.pathomation.com/pathotrainer/doku.php?id=interaction\\_modes&rev=1597761321](https://docs.pathomation.com/pathotrainer/doku.php?id=interaction_modes&rev=1597761321)

Last update: **2020/08/18 17:35**

