

# Participant management

On this page a trainer/supervisor can select how the trainees can interact with the software. There are different [interaction modes](#)

[pma.control\\_interaction\\_modes.mp4](#)

This is performed through the Participant Management menu in the session overview menu. For details on where to find this see [the session page article](#).

Primarily you will interact with participants by changing the interaction mode of the slides/case collections and their associated forms.

In the screenshot below you can see how the page looks for the User (left) and the Project Owner (right) when various option

1. Locked - the content is not accessible
2. Self test - the number of cases the User has scored appears next to Test 3
3. Browse - the user can browse, but not score or see consensus/other user scores for the slides

The screenshot displays two side-by-side views of the PMA control test environment. The left view, labeled 'User', shows a 'Training Session Test' interface with a 'Test form' and 'Case collections' list. The right view, labeled 'Project Owner', shows the 'Patho\_Project - Training Session Test - Participants management' interface. In the Project Owner view, there are dropdown menus for 'Test', 'Test 3', and 'Test 2' to change the interaction mode. Red arrows and numbers 1, 2, and 3 point to these dropdowns. A red box highlights the 'Test' dropdown. A red box highlights the 'Test 3' dropdown. A red box highlights the 'Test 2' dropdown. A red box highlights the 'Apply all' button. A red box highlights the 'Test' dropdown. A red box highlights the 'Test 3' dropdown. A red box highlights the 'Test 2' dropdown. A red box highlights the 'Apply all' button. A red box highlights the 'Test' dropdown. A red box highlights the 'Test 3' dropdown. A red box highlights the 'Test 2' dropdown. A red box highlights the 'Apply all' button.

From:  
<https://docs.pathomation.com/pathotrainer/> - Pathotrainer

Permanent link:  
[https://docs.pathomation.com/pathotrainer/doku.php?id=participant\\_management&rev=1704279564](https://docs.pathomation.com/pathotrainer/doku.php?id=participant_management&rev=1704279564)

Last update: 2024/01/03 13:59

