

What's new in version 2.2.0?

Pathotrainer 2.2.0 is a release of the PathoTrainer codebase.

It is a minor release that contains several new features helpful for multi-stage training programs, extracting actionable information from global evaluation programs, and reducing the admin burden for these.

New Features

- If you have more than one PMA.core server with us e.g. in USA, LATAM, EU, Asia, Africa and Oceania, it is now possible to turn on automatic server assignment in a training session so that users will be connected to the fastest available server.
 - *It is still possible to manually assign a dedicated server per training session*
- Country tagging for users - Users can now input or be assigned to a country
- Training Session results are now searchable/filterable by country
- Users can be searched for by country when adding participants to a training session
- Progression based unlocking - It is now possible to make Case Collections in a Training Session unlock sequentially as the Trainee completes them

Older Versions

What's new in version 2.1.0?

Pathotrainer 2.1.0 is a release of the PathoTrainer codebase.

It is a minor release that expands on the annotation capabilities to allow for 'teacher' annotations.

New Features

- When using the consensus user edit mode, you are able to create 'Supervisor' level annotations that only reveal to the Trainee upon submission of their own answer.

What's new in version 2.0.0?

Pathotrainer 2.0.0 is a release of the Pathotrainer codebase (previously known as PMA.control).

It is a major release that contains multiple new features including:

- Trainees can now create annotations visible only to Supervisors within a Training Session,
- Admins can now create always-online self-study courses complete with automatic certification, quizzes,
- and much more - see the full notes below:

New Features

- CTRL-27: Made it possible to clone / duplicate forms
- CTRL-48: Enabled measurements in the training session slide viewer
- CTRL-60: Added new data export options including csv, normalized data
- CTRL-63: Documentation tab now directs users to the Pathotrainer wiki
- CTRL-64: A certification tab has been added to allow automatic certification for training sessions
- CTRL-76: API calls can now optionally filter by session name
- CTRL-82: Added shortcuts to make it easier to manage a project
- CTRL-85: Integrated changes to ensure compatibility with PMA.core 3
- CTRL-91: A pop up message has been made to increase awareness of the "Sync" function.
- CTRL-93: Course Mode - functionality to support the creation of standalone self-learning resources has been added.
- CTRL-94: The functionality to make Quizzes has been added.
- CTRL-95: Additional zoom level controls have been added at the bottom of the viewer UI
- CTRL-96: Users can now make annotations in a session outside of Live Mode. These are only visible to supervisors, and the supervisor is able to filter annotations by user(s)
- CTRL-97: Sharing of WSIs and ROIs is now possible
- CTRL-98: Project owners / Supervisors can now set how all image labels will be displayed (i.e. filenames | captions | no label) with a single click in the Training Session options.

Bug fixes

- CTRL-13: The 'Reset Score' button no longer appears when resetting the score is not permitted.
- CTRL-16: Fixed an issue that would allow scoring on the wrong case
- CTRL-17: Fixed an issue that could transfer form data from one case to another in rare circumstances
- CTRL-23: Fixed an issue causing some scores to not appear correctly in training sessions
- CTRL-51: Fixed the way the "Insert link" button was displayed in training sessions
- CTRL-88: Changed License expiry notification visibility to only admin users
- CTRL-145: Fixed bug in which WSI content would not load in Course Mode

From:

<https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link:

https://docs.pathomation.com/pathotrainer/doku.php?id=release_notes

Last update: **2025/11/14 16:59**

