

# What's new in version 2.0.0?

Pathotrainer 2.0.0 is a release of the Pathotrainer codebase (previously known as PMA.control). It is a minor that contains multiple new features including the ability to create always-on self-study courses, automatic certification, quizzes, and more.

## New Features

- CTRL-27: Made it possible to clone / duplicate forms
- CTRL-48: Enabled measurements in the training session slide viewer
- CTRL-60: Added new data export options including csv, normalized data
- CTRL-63: Documentation tab now directs users to the Pathotrainer wiki
- CTRL-64: A certification tab has been added to allow automatic certification for training sessions
- CTRL-76: API calls can now optionally filter by session name
- CTRL-82: Added shortcuts to make it easier to manage a project
- CTRL-85: Integrated changes to ensure compatibility with PMA.core 3
- CTRL-91: A pop up message has been made to increase awareness of the "Sync" function.
- CTRL-93: Course Mode - functionality to support the creation of standalone self-learning resources has been added.
- CTRL-94: The functionality to make Quizzes has been added.
- CTRL-95: Additional zoom level controls have been added at the bottom of the viewer UI
- CTRL-96: Users can now make annotations in a session outside of Live Mode. These are only visible to supervisors, and the supervisor is able to filter annotations by user(s)
- CTRL-97: Sharing of WSIs and ROIs is now possible
- CTRL-98: Project owners / Supervisors can now set how all image labels will be displayed ( i.e. filenames | captions | no label ) with a single click in the Training Session options.

## Bug fixes

- CTRL-13: The 'Reset Score' button no longer appears when resetting the score is not permitted.
- CTRL-16: Fixed an issue that would allow scoring on the wrong case
- CTRL-17: Fixed an issue that could transfer form data from one case to another in rare circumstances
- CTRL-23: Fixed an issue causing some scores to not appear correctly in training sessions
- CTRL-51: Fixed the way the "Insert link" button was displayed in training sessions
- CTRL-88: Changed License expiry notification visibility to only admin users
- CTRL-145: Fixed bug in which WSI content would not load in Course Mode

From:  
<https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link:  
[https://docs.pathomation.com/pathotrainer/doku.php?id=release\\_notes&rev=1763126398](https://docs.pathomation.com/pathotrainer/doku.php?id=release_notes&rev=1763126398)

Last update: **2025/11/14 16:19**

