

What's new in version 2.2.0?

Pathotrainer 2.2.0 is a release of the PathoTrainer codebase.

It is a minor release that contains several new features helpful for multi-stage training programs, extracting actionable information from global evaluation programs, and reducing the admin burden for these.

New Features

- If you have more than one PMA.core server with us e.g. in USA, LATAM, EU, Asia, Africa and Oceania, it is now possible to turn on automatic server assignment in a training session so that users will be connected to the fastest available server.
 - *It is still possible to manually assign a dedicated server per training session*
- Country tagging for users - Users can now input or be assigned to a country
- Training Session results are now searchable/filterable by country
- Users can be searched for by country when adding participants to a training session
- Progression based unlocking - It is now possible to make Case Collections in a Training Session unlock sequentially as the Trainee completes them

Older Versions

What's new in version 2.1.0?

Pathotrainer 2.1.0 is a release of the PathoTrainer codebase.

It is a minor release that expands on the annotation capabilities. You can now create annotations as the consensus user

New Features

- CTRL-27: Made it possible to clone / duplicate forms

What's new in version 2.0.0?

Pathotrainer 2.0.0 is a release of the Pathotrainer codebase (previously known as PMA.control).

It is a major release that contains multiple new features including:

- Trainees can now create annotations visible only to Supervisors within a Training Session,
- Admins can now create always-online self-study courses complete with automatic certification, quizzes,
- and much more - see the full notes below:

New Features

- CTRL-27: Made it possible to clone / duplicate forms
- CTRL-48: Enabled measurements in the training session slide viewer
- CTRL-60: Added new data export options including csv, normalized data
- CTRL-63: Documentation tab now directs users to the Pathotrainer wiki
- CTRL-64: A certification tab has been added to allow automatic certification for training sessions
- CTRL-76: API calls can now optionally filter by session name
- CTRL-82: Added shortcuts to make it easier to manage a project
- CTRL-85: Integrated changes to ensure compatibility with PMA.core 3
- CTRL-91: A pop up message has been made to increase awareness of the “Sync” function.
- CTRL-93: Course Mode - functionality to support the creation of standalone self-learning resources has been added.
- CTRL-94: The functionality to make Quizzes has been added.
- CTRL-95: Additional zoom level controls have been added at the bottom of the viewer UI
- CTRL-96: Users can now make annotations in a session outside of Live Mode. These are only visible to supervisors, and the supervisor is able to filter annotations by user(s)
- CTRL-97: Sharing of WSIs and ROIs is now possible
- CTRL-98: Project owners / Supervisors can now set how all image labels will be displayed (i.e. filenames | captions | no label) with a single click in the Training Session options.

Bug fixes

- CTRL-13: The 'Reset Score' button no longer appears when resetting the score is not permitted.
- CTRL-16: Fixed an issue that would allow scoring on the wrong case
- CTRL-17: Fixed an issue that could transfer form data from one case to another in rare circumstances
- CTRL-23: Fixed an issue causing some scores to not appear correctly in training sessions
- CTRL-51: Fixed the way the “Insert link” button was displayed in training sessions
- CTRL-88: Changed License expiry notification visibility to only admin users
- CTRL-145: Fixed bug in which WSI content would not load in Course Mode

From:
<https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link:
https://docs.pathomation.com/pathotrainer/doku.php?id=release_notes&rev=1763128651

Last update: **2025/11/14 16:57**

