


Scoring cases

If test active mode has been enabled for you by the supervisors, you can score each case of a case collection by filling in the form fields. Sometimes you might need to scroll down to the bottom of the right column, below the slide list to view the form:



The screenshot shows a mobile application interface for scoring a case. At the top, there is a header bar with a back arrow, the text 'case03', and a forward arrow. Below the header, there is a 'Sync' checkbox. The main content area contains two case entries: '1_HE-1.1.mrxs' and '2_CD138-1.1.mrxs'. Each entry has a small image of a cell. Below the case entries, there is a section labeled 'IC%' with a text input field containing 'ea covered by immune ce' and a dropdown arrow. Below this, there is a section labeled 'POS/NEG' with a dropdown menu. At the bottom, there are two buttons: 'Submit' and 'Reset'.

In case an error occurs, a prompt will appear. You can close the prompt, correct any errors and submit the form again:

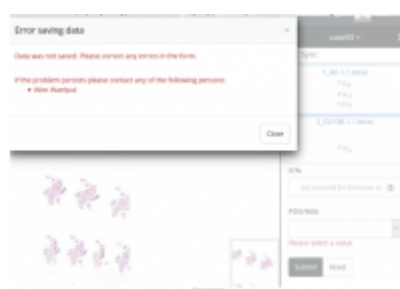

The screenshot shows an 'Error saving data' prompt. The prompt text reads: 'Data was not saved. Please correct any errors in the form. If the problem persists please contact any of the following persons: [Name] [Email]'. There is a 'Close' button. In the background, the case scoring form is visible, showing the same case entries and fields as in the previous screenshot.

You can correct any individual field errors and submit again:



IC%


extra covered by immune cells

POS/NEG

Please select a value

Submit Reset

As you complete each case, the system displays the forms that you have already scored, by expanding the list of cases:

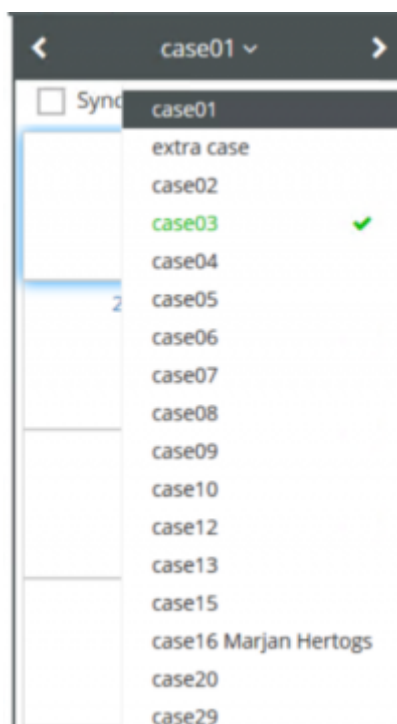

case01

extra case

case02

case03 ✓

case04

case05

case06

case07

case08

case09

case10

case12

case13

case15

case16 Marjan Hertogs

case20

case29

Finally, you can track your progress as well as overview all the cases you have scored so far, from the session main page and the score report page:



Case collection 1 1/17 cases evaluated Take the test by scoring each case

Case collection 2 0/3 cases evaluated Study the consensus score

Score report

From:

<https://docs.pathomation.com/pathotrainer/> - Pathotrainer

Permanent link:

https://docs.pathomation.com/pathotrainer/doku.php?id=scoring_cases

Last update: 2020/09/25 18:39



