


### Scoring cases

If test active mode has been enabled for you by the supervisors, you can score each case of a case collection by filling in the form fields. Sometimes you might need to scroll down to the bottom of the right column, below the slide list to view the form:

The screenshot shows a mobile application interface for scoring cases. At the top, there is a dark header with a left arrow, the text "case03" with a dropdown arrow, and a right arrow. Below the header is a white area with a "Sync" checkbox. Underneath are two case items, each in a white box with a blue border. The first item is "1\_HE-1.1.mrxs" with a star rating of 4.5. The second item is "2\_CD138-1.1.mrxs" with a star rating of 3.5. Below the cases are two input fields. The first is labeled "IC%" and contains the text "area covered by immune ce" with a dropdown arrow. The second is labeled "POS/NEG" and is empty with a dropdown arrow. At the bottom are two buttons: "Submit" and "Reset".


In case an error occurs, a prompt will appear. You can close the prompt, correct any errors and submit the form again:

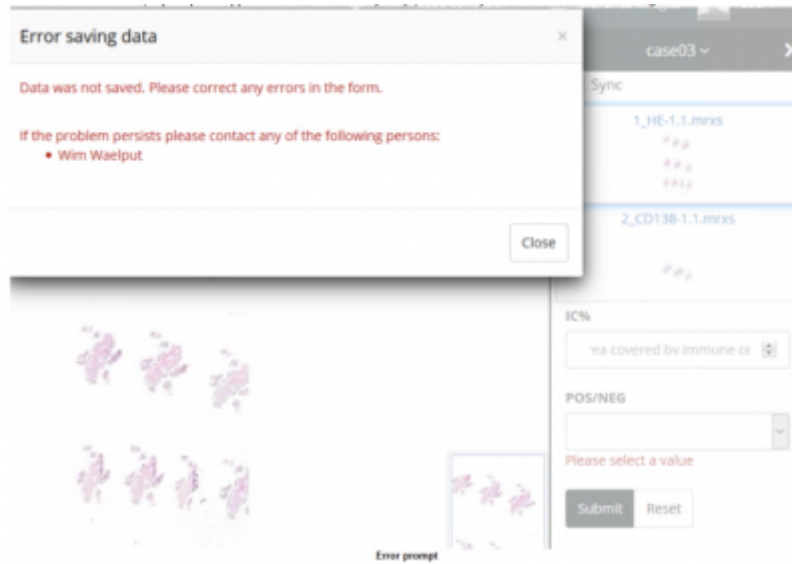


You can correct any individual field errors and submit again:



As you complete each case, the system displays the forms that you have already scored, by expanding the list of cases:

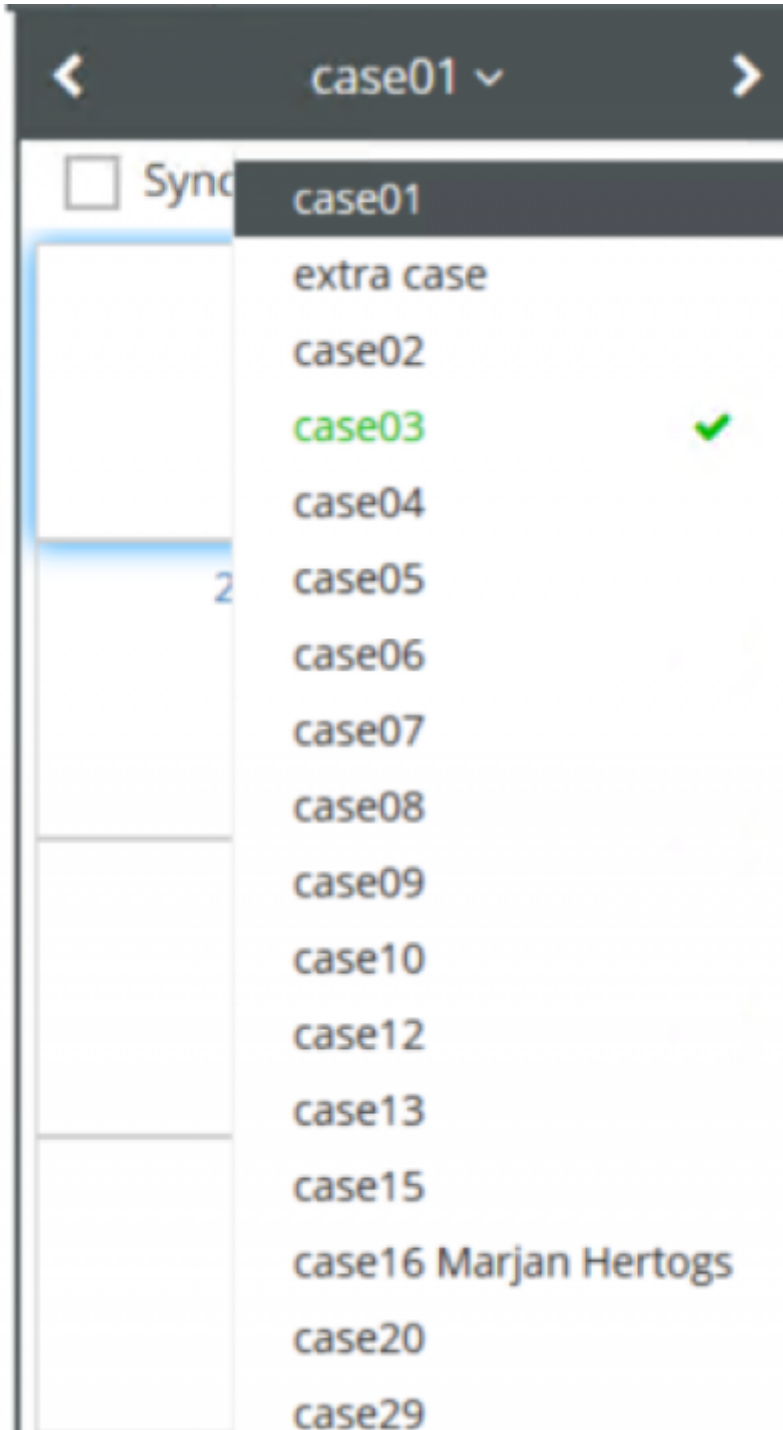

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Finally, you can track your progress as well as overview all the cases you have scored so far, from the session main page and the score report page:



From:  
<https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link:  
[https://docs.pathomation.com/pathotrainer/doku.php?id=scoring\\_cases&rev=1591362906](https://docs.pathomation.com/pathotrainer/doku.php?id=scoring_cases&rev=1591362906)

Last update: **2020/06/05 16:15**

