

## Scoring cases

If test active mode has been enabled for you by the supervisors, you can score each case of a case collection by filling in the form fields. Sometimes you might need to scroll down to the bottom of the right column, below the slide list to view the form:

The screenshot displays a mobile application interface for scoring cases. At the top, a dark header bar contains a back arrow, the text 'case03' with a dropdown arrow, and a forward arrow. Below the header, there is a 'Sync' checkbox. The main content area shows two case thumbnails. The first thumbnail, titled '1\_HE-1.1.mrxs', features a microscopic image of tissue with three red star-like markers. The second thumbnail, titled '2\_CD138-1.1.mrxs', shows a similar image with three red star-like markers. Below the thumbnails, there is an 'IC%' label followed by a dropdown menu currently displaying 'area covered by immune ce'. Underneath this is a 'POS/NEG' label followed by another dropdown menu. At the bottom of the form are two buttons: 'Submit' and 'Reset'.


In case an error occurs, a prompt will appear. You can close the prompt, correct any errors and submit the form again:

The screenshot shows a web application interface. On the left, there's a panel with histology images. On the right, a form for 'case03' is visible. The form includes a 'Sync' button, two image upload fields (1\_HE-1.1.mrxs and 2\_CD138-1.1.mrxs), a dropdown for 'IC%' with the value 'na covered by immune ce', and a dropdown for 'POS/NEG' which is currently empty. Below these is a red error message 'Please select a value' and 'Submit' and 'Reset' buttons. An 'Error saving data' modal is open in the foreground, displaying the message: 'Data was not saved. Please correct any errors in the form. If the problem persists please contact any of the following persons: • Wim Waelpuut'. The modal has a 'Close' button.

You can correct any individual field errors and submit again:

This is a close-up of the 'POS/NEG' dropdown menu. The dropdown is currently empty, and a red error message 'Please select a value' is displayed below it. The 'Submit' and 'Reset' buttons are visible at the bottom of the form.

As you complete each case, the system displays the forms that you have already scored, by expanding the list of cases:

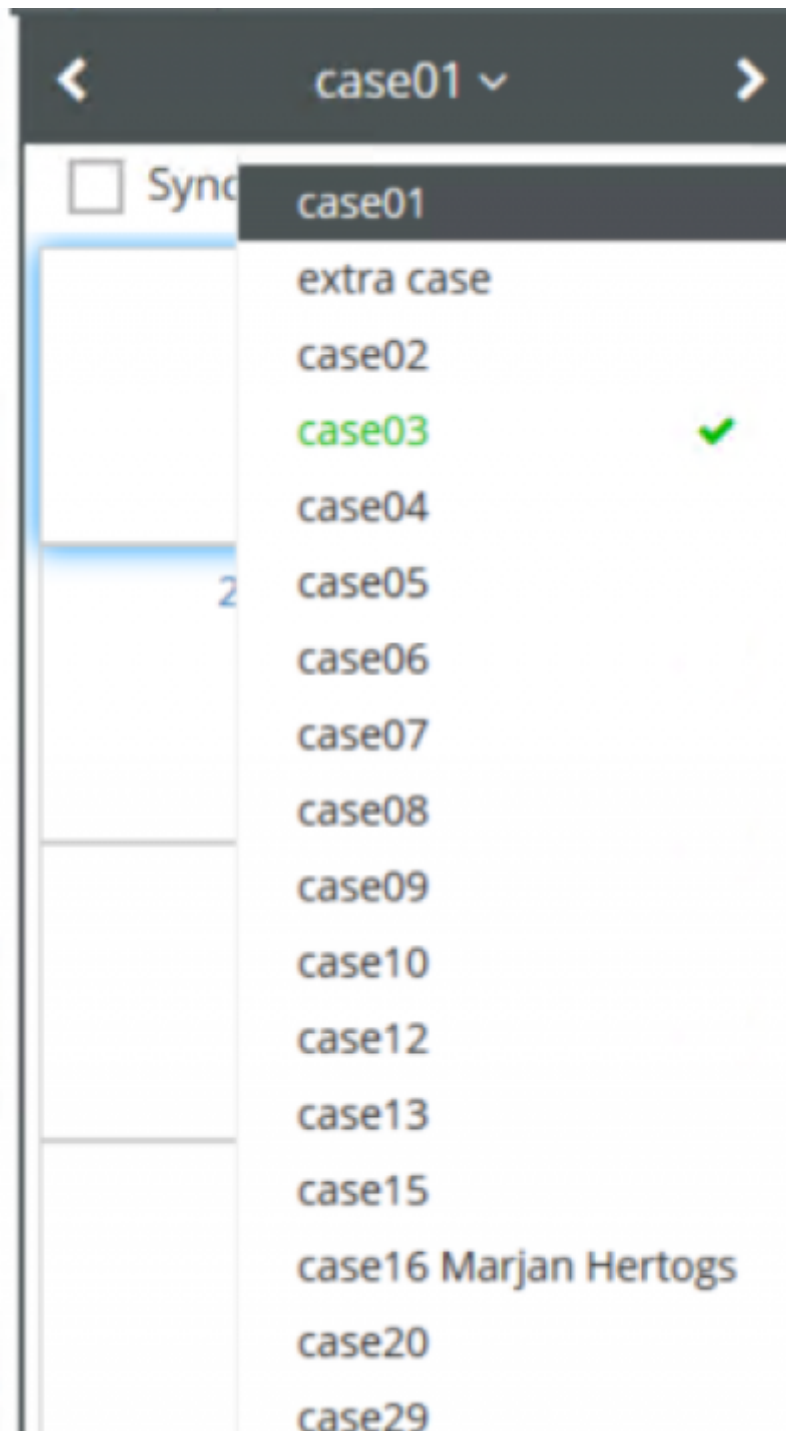

Finally, you can track your progress as well as overview all the cases you have scored so far, from the session main page and the score report page:



From:

<https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link:

[https://docs.pathomation.com/pathotrainer/doku.php?id=scoring\\_cases&rev=1591362906](https://docs.pathomation.com/pathotrainer/doku.php?id=scoring_cases&rev=1591362906)

Last update: **2020/06/05 16:15**

