


### Scoring cases

If test active mode has been enabled for you by the supervisors, you can score each case of a case collection by filling in the form fields. Sometimes you might need to scroll down to the bottom of the right column, below the slide list to view the form:



In case an error occurs, a prompt will appear. You can close the prompt, correct any errors and submit the form again:

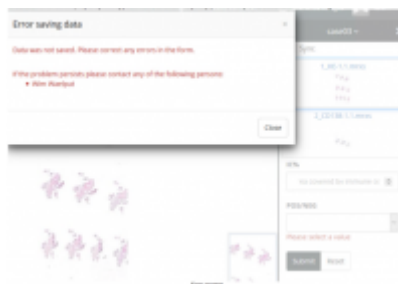

You can correct any individual field errors and submit again:

**IC%**


area covered by immune cells

**POS/NEG**


Please select a value

**Submit**    **Reset**

As you complete each case, the system displays the forms that you have already scored, by expanding the list of cases:

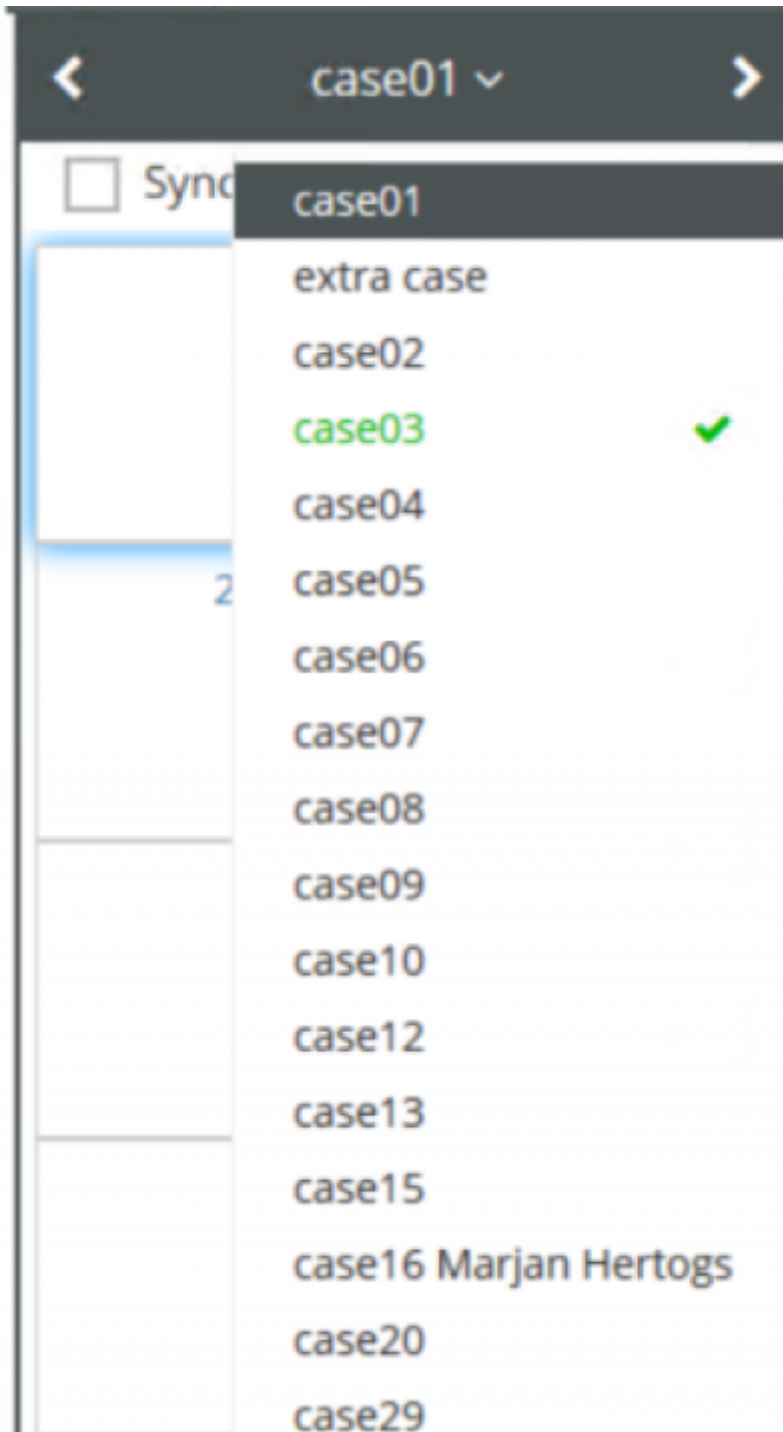

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Finally, you can track your progress as well as overview all the cases you have scored so far, from the session main page and the score report page:



From:

<https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link:

[https://docs.pathomation.com/pathotrainer/doku.php?id=scoring\\_cases&rev=1591362917](https://docs.pathomation.com/pathotrainer/doku.php?id=scoring_cases&rev=1591362917)

Last update: **2020/06/05 16:15**

