


Scoring cases

If test active mode has been enabled for you by the supervisors, you can score each case of a case collection by filling in the form fields. Sometimes you might need to scroll down to the bottom of the right column, below the slide list to view the form:



In case an error occurs, a prompt will appear. You can close the prompt, correct any errors and submit the form again:

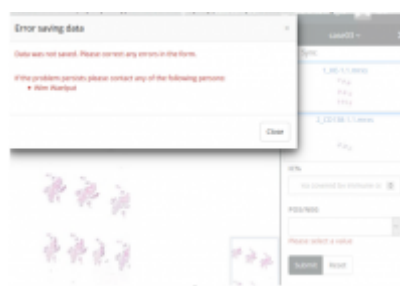

You can correct any individual field errors and submit again:

IC%

area covered by immune cells

POS/NEG

Please select a value

Submit Reset

As you complete each case, the system displays the forms that you have already scored, by expanding the list of cases:

< case01 >

☐ Sync

- case01
- extra case
- case02
- case03 ✓
- case04
- case05
- case06
- case07
- case08
- case09
- case10
- case12
- case13
- case15
- case16 Marjan Hertogs
- case20
- case29

Finally, you can track your progress as well as overview all the cases you have scored so far, from the session main page and the score report page:

Case collection 1 1/17 cases evaluated Take the test by scoring each case >

Case collection 2 0/3 cases evaluated Study the consensus score >

Score report >

From:
<https://docs.pathomation.com/pathotrainer/> - Pathotrainer

Permanent link:
https://docs.pathomation.com/pathotrainer/doku.php?id=scoring_cases&rev=1591362931

Last update: 2020/06/05 16:15

