


### Scoring cases

If test active mode has been enabled for you by the supervisors, you can score each case of a case collection by filling in the form fields. Sometimes you might need to scroll down to the bottom of the right column, below the slide list to view the form:




In case an error occurs, a prompt will appear. You can close the prompt, correct any errors and submit the form again:

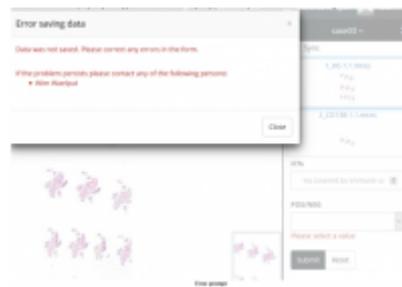

You can correct any individual field errors and submit again:

The screenshot shows a form with two main input fields. The first field is labeled 'IC%' and contains the text 'lea covered by immune ce'. The second field is labeled 'POS/NEG' and is currently empty, with a red border around it and the text 'Please select a value' below it. At the bottom of the form are two buttons: 'Submit' and 'Reset'.

As you complete each case, the system displays the forms that you have already scored, by expanding the list of cases:

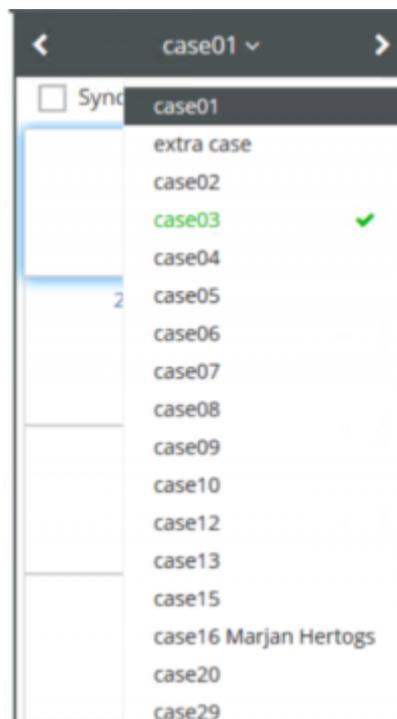

Finally, you can track your progress as well as overview all the cases you have scored so far, from the session main page and the score report page:



From:  
<https://docs.pathomation.com/pathotrainer/> - Pathotrainer

Permanent link:  
[https://docs.pathomation.com/pathotrainer/doku.php?id=scoring\\_cases&rev=1591362931](https://docs.pathomation.com/pathotrainer/doku.php?id=scoring_cases&rev=1591362931)

Last update: **2020/06/05 16:15**

