


Scoring cases

If test active mode has been enabled for you by the supervisors, you can score each case of a case collection by filling in the form fields. Sometimes you might need to scroll down to the bottom of the right column, below the slide list to view the form:



In case an error occurs, a prompt will appear. You can close the prompt, correct any errors and submit the form again:

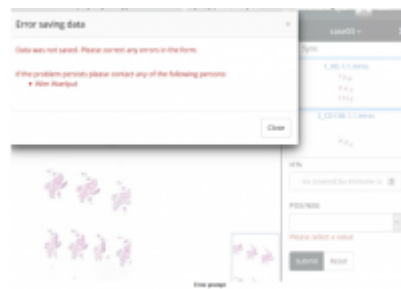


You can correct any individual field errors and submit again:

The screenshot shows a form with the following elements:

- A dropdown menu labeled "IC%" with the text "ea covered by immune ce" and a downward arrow.
- A dropdown menu labeled "POS/NEG" which is currently empty and has a red border around it. Below it is the text "Please select a value".
- Two buttons: "Submit" and "Reset".

As you complete each case, the system displays the forms that you have already scored, by expanding the list of cases:

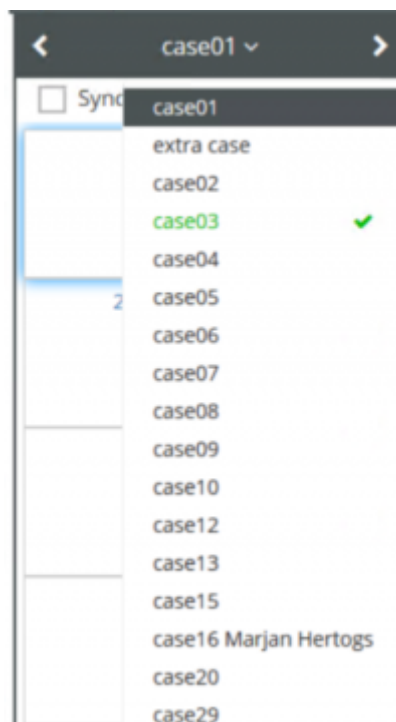

Finally, you can track your progress as well as overview all the cases you have scored so far, from the session main page and the score report page:



From:
<https://docs.pathomation.com/pathotrainer/> - Pathotrainer

Permanent link:
https://docs.pathomation.com/pathotrainer/doku.php?id=scoring_cases&rev=1591362931

Last update: **2020/06/05 16:15**

