


# Session page (supervisor)


Once you join a session, you will have access to the available case collections as well as all supplemental material related to the session. The exact layout of a session may vary depending on the configuration. A typical setup is illustrated below.

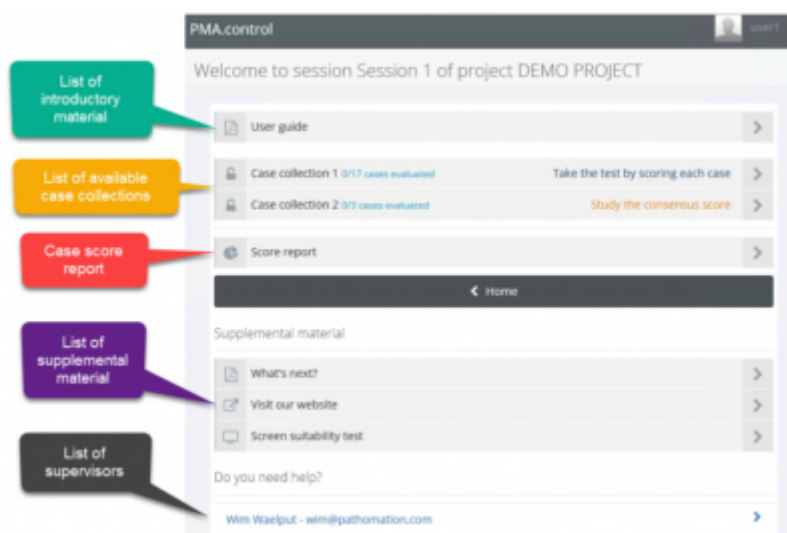


In addition to the material presented to the trainees, a Supervisor/Project Owner also sees additional options not presented to the trainees in the Supervisor Options menu (3)

This contains the:

- participants management page where slides and forms can be made available for different types of testing, or hidden/locked.
- comprehensive score report of all participants.
- questionnaire results

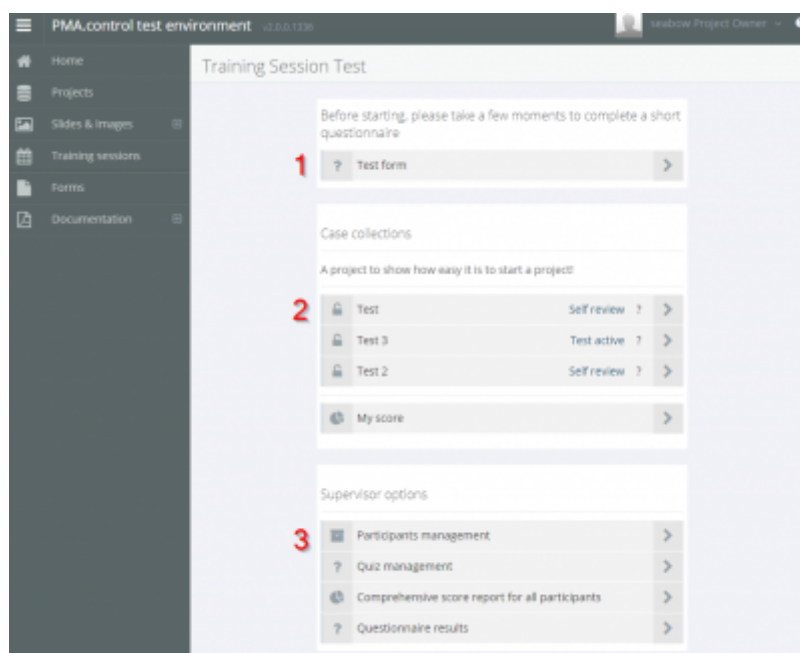

Project owners will also see the Project Owner menu (4) in which they can select Session Configuration - this will take them through to the project editing menu.

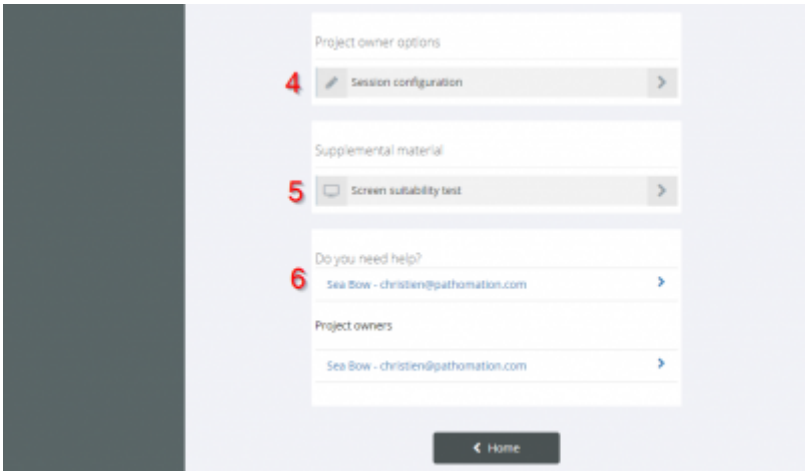

From:  
<https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link:  
[https://docs.pathomation.com/pathotrainer/doku.php?id=session\\_main\\_page\\_supervisor&rev=1704278463](https://docs.pathomation.com/pathotrainer/doku.php?id=session_main_page_supervisor&rev=1704278463)

Last update: **2024/01/03 13:41**

