


Setting Up a Session

Previously if you wanted to give a training to pathologists, you'd need a full briefcase: you'd need enough exemplar slides for all your students per indication/biomarker, exam forms, case information... and that's before you even begin to consider the microscopes!

Projects and sessions were designed to completely eradicate the need for these suitcases. A project contains everything you need to train other pathologists.

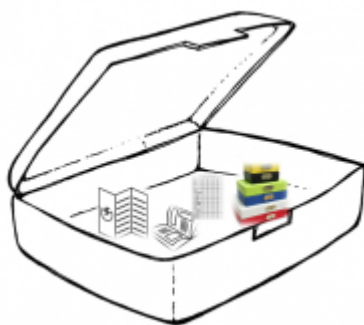

A session contains:

1. All your participants
2. The ability to invite and/or contact all of them at once
3. Tracking of email invitations and logins
4. Forms to capture the data from the participants
5. Reader manuals / interpretation guides (videos, powerpoints, pdf's, etc)
6. Cases collections ('slide boxes') which all your participants can see at once
7. Consensus scores

[trainingsession_pma.control.mp4](#)

From:

<https://docs.pathomation.com/pathotrainer/> - Pathotrainer

Permanent link:

https://docs.pathomation.com/pathotrainer/doku.php?id=setup_a_session&rev=1701771197

Last update: **2023/12/05 13:13**

