


# Sharing Content of a Training Session

Depending on how a Training Session is configured, you may be able to share WSI content from a session with other people, whether they are Pathotrainer users or not, using just a link.

The share can be password protected, and have an expiration date.


## Permission to share

The Project Owner determines who, if anyone, can share the slide content using a dropdown menu when setting up the session and by going to the 'Edit Training Session' menu. They can choose nobody, managers, managers and supervisors, or everyone.

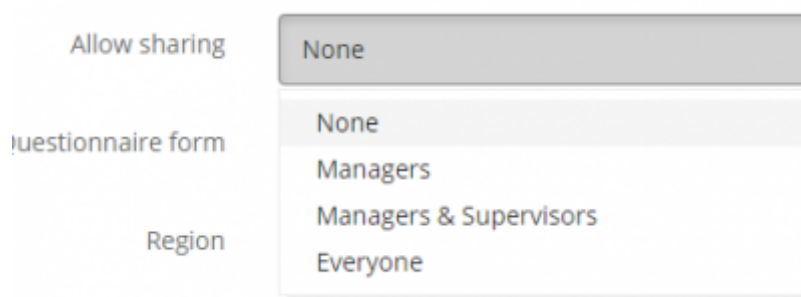

## Sharing

If you are allowed to share content, you will see the Share button in the Case menu on the right hand side:

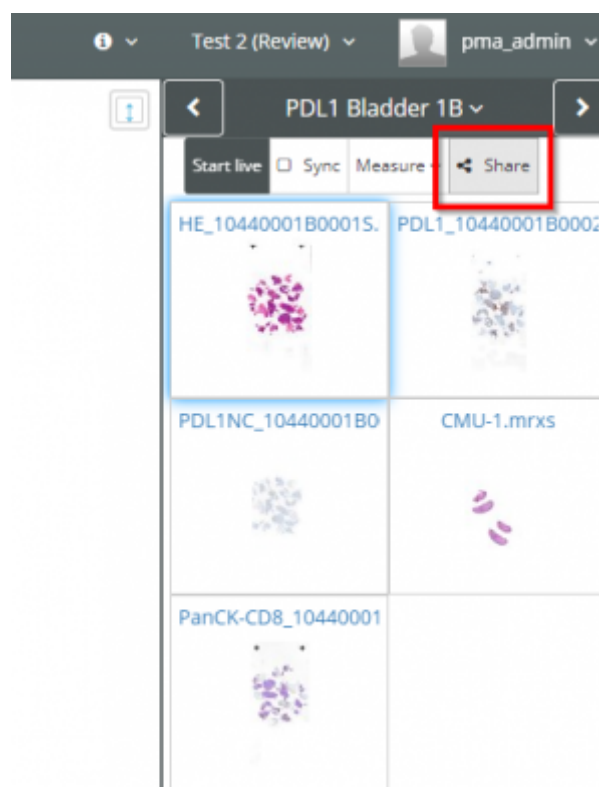

After clicking the 'Share' button, the share dialog box will open.


Here you can define how much you want to share:

- Everything - Shares all slides and case collections in the session
- Current collection (Case Collection Name) - Shares current case collection
- Current case (Case Name) - Shares current case
- Open images (# of open images) - Shares only the images you currently have open in the viewport

Once you've selected what you want to share, you can then choose to set a password, and/or an expiration date for the share.


When you're ready, click the Share button [1] to generate a link

Then copy the link [2] and send it to the recipient

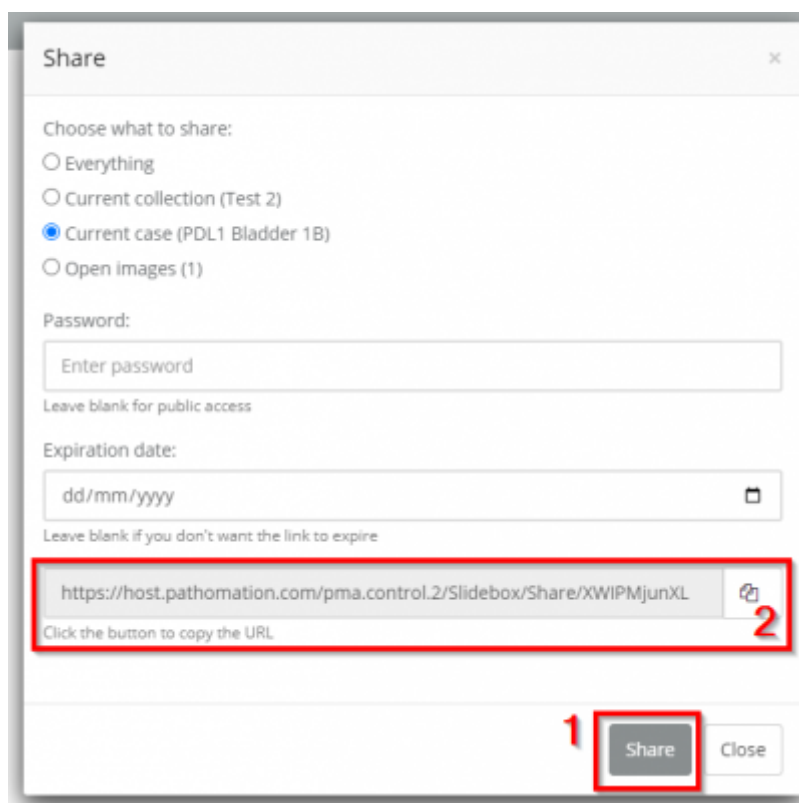

The recipient will be able to open the link and will see the slides similarly to this:

Please note, the recipient will be able to view and navigate the slide, but will not be able to score, annotate or interact with it in any other way.

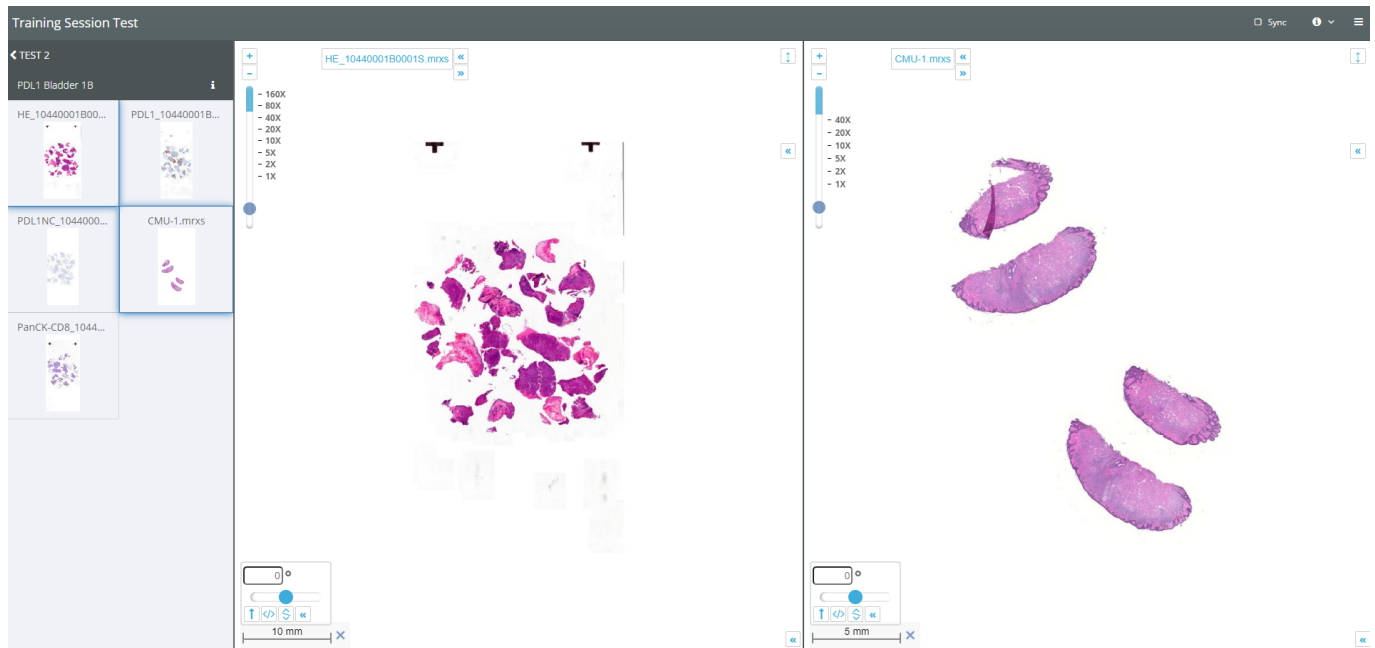

From:

<https://docs.pathomation.com/pathotrainer/> - **Pathotrainer**

Permanent link:

[https://docs.pathomation.com/pathotrainer/doku.php?id=sharing\\_content](https://docs.pathomation.com/pathotrainer/doku.php?id=sharing_content)

Last update: **2024/02/05 15:40**

