


In this page you can configure the landing page and the home page.

1. Brand name
2. Email help address
3. Logo
4. Allow user registration option
5. Password recovery option
6. Welcome message on home page
7. Item which filetypes are allowed in a project, welcome message, logo and the brand name on the landing page, email contact address, logo and automated password generation.

LANDING PAGE:

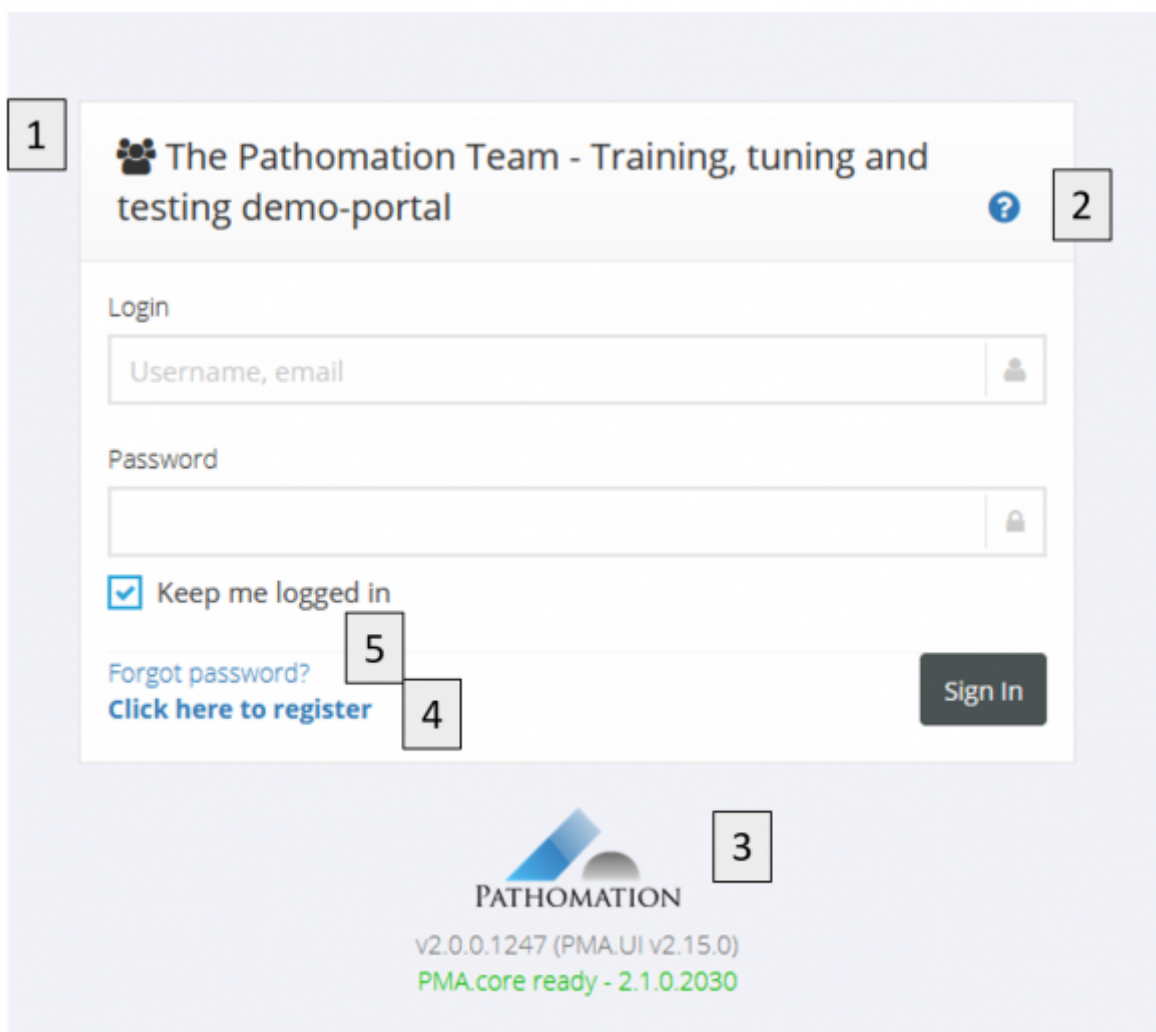

Allowed filetypes are

Brand name

Contact Us link


Allow password recovery

Enable user registration  [Set registration terms](#)

Generate passwords automatically

Allow users to be registered as participants in sessions upon registration

Logo    
Max file size: 10Mb



Welcome message 

Rich text editor toolbar: Bold, Italic, Underline, Strikethrough, Bulleted list, Numbered list, Indent, Outdent, Undo, Redo, Source code, Help.

Editor content area (empty)

From: <https://docs.pathomation.com/pathotrainer/> - Pathotrainer

Permanent link: [https://docs.pathomation.com/pathotrainer/doku.php?id=system\\_settings&rev=1638289855](https://docs.pathomation.com/pathotrainer/doku.php?id=system_settings&rev=1638289855)

Last update: **2021/11/30 19:30**

