

# Trainees vs Supervisors

Within any given Training Session that a user has been made a participant of, a user may occupy one of two roles: Trainee or Supervisor.

Trainees can engage with

Feature	Trainee	Supervisor
Join Training Sessions If Invited	Yes	Yes
Score Case Collections	Yes	Yes
View Own Score	Yes	Yes
View Others Scores	No	Yes
Create Own Annotations	Yes*	Yes
View Own Annotations	Yes*	Yes
View Other Trainees Annotations	No	Yes
Change Interaction Mode	No	Yes

  

Feature	User	Project Owner	Admin
Join Training Sessions	Yes	Yes	Yes
Create Users	No	Yes	Yes
Manage Users	No	No	Yes
Create Scoring Forms	No	No	Yes
View Projects	No	Yes*	Yes
Create Projects	No	Yes	Yes
View Image Inventory	No	Yes	Yes
View Case Collections	No	Yes*	Yes
Create Case Collections	No	Yes	Yes
View Training Sessions	No	Yes*	Yes
Create Training Sessions	No	Yes	Yes

From:

<https://docs.pathomation.com/pathotrainer/> - Pathotrainer

Permanent link:

[https://docs.pathomation.com/pathotrainer/doku.php?id=trainees\\_vs\\_supervisors&rev=1763473529](https://docs.pathomation.com/pathotrainer/doku.php?id=trainees_vs_supervisors&rev=1763473529)

Last update: 2025/11/18 16:45

