

Trainees vs Supervisors

Training Sessions are made by Content Creator account types Admin/Project Owner. Once a Training Session is created, Admin/Project Owners can invite user accounts as Participants to Training Sessions as either a Trainee or a Supervisor.

Importantly, any account type: User, Admin or Project Owner can be made into a Trainee/Supervisor in any given session.

In short: Trainees can engage with Training Sessions & content within them if they have been invited. Supervisors can do the same, and additionally can manage themselves and other trainees within the session and view scores.

A full breakdown of responsibilities is below:

Feature	Trainee	Supervisor
Join Training Sessions	Yes	Yes
Score Case Collections	Yes	Yes
View Own Score	Yes	Yes
View Others Scores	No	Yes
Create Own Annotations	Yes*	Yes*
View Own Annotations	Yes*	Yes*
View Other Trainees Annotations	No	Yes*
Change Interaction Mode	No	Yes

* = if annotations creation/review are activated in the current interaction mode

From:
<https://docs.pathomation.com/pathotrainer/> - Pathotrainer

Permanent link:
https://docs.pathomation.com/pathotrainer/doku.php?id=trainees_vs_supervisors&rev=1763475894

Last update: **2025/11/18 17:24**

