

# Uploading of slides in Pathotrainer

Before you can start to make your cases for your training session, it is necessary to upload the slides to your PathoTrainer platform. In this video, we will show you how to do this step by step.

Starting from the Home Page in PathoTrainer, navigate to Slides and Images in the sidebar and select the Inventory tab. This will open the treeview of available sources of slides, for example from the PMA.core instances you have linked to PathoTrainer.

From here, you can navigate through folders and view the slides and images you have available. You can:

Hover your cursor over image files to see a small thumbnail

Click on an image to open it in the larger viewer

From:

<https://docs.pathomation.com/pathotrainer/> - Pathotrainer

Permanent link:

[https://docs.pathomation.com/pathotrainer/doku.php?id=uploading\\_slides&rev=1763646278](https://docs.pathomation.com/pathotrainer/doku.php?id=uploading_slides&rev=1763646278)

Last update: **2025/11/20 16:44**

