

Uploading of slides in Pathotrainer

Before you can start to make your cases for your training session, it is necessary to upload the slides to your PathoTrainer platform. In this video, we will show you how to do this step by step.

Starting from the Home Page in PathoTrainer, navigate to Slides and Images in the sidebar and select the Inventory tab. This will open the treeview of available sources of slides, where you should see all the folders containing your slides in your PMA.core instances linked to PathoTrainer.

From:

<https://docs.pathomation.com/pathotrainer/> - Pathotrainer

Permanent link:

https://docs.pathomation.com/pathotrainer/doku.php?id=uploading_slides&rev=1763646666

Last update: **2025/11/20 16:51**

