

# Uploading of slides in Pathotrainer

Before you can start to make your cases for your training session, it is necessary to upload the slides to your PathoTrainer platform. In this video, we will show you how to do this step by step.

[how\\_to\\_upload\\_slides.mp4](#)

## How to upload your own slides

Starting from the Home Page in PathoTrainer, navigate to Slides and Images in the sidebar and select the Inventory tab. This will open the treeview of available sources of slides, where you should see all the folders containing your slides in your PMA.core instances linked to PathoTrainer.

Here, in the top right corner, you will have two options: you can either upload individual slides or upload entire folders containing your slides to keep a better overview.

Once you click Upload, you will see a notification appear in the bottom right corner showing the progress of your upload.

From here, you can navigate through folders and view the slides and images you have available. You can:

- Hover your cursor over image files to see a small thumbnail
- Click on an image to open it in the larger viewer

From:

<https://docs.pathomation.com/pathotrainer/> - Pathotrainer

Permanent link:

[https://docs.pathomation.com/pathotrainer/doku.php?id=uploading\\_slides&rev=1764007053](https://docs.pathomation.com/pathotrainer/doku.php?id=uploading_slides&rev=1764007053)

Last update: **2025/11/24 20:57**

