


Panels layout (administration)

This page allows us to set the default panel layout for PMA.Studio viewer. Viewer panel consists of one or more viewports organized in grid and toolbar underneath it. Almost every section of PMA.studio frontend has its own panel that allows user to easily interact and make changes accordingly.

A panel is a separate application component of specific type. There are n panels - Viewer, Tree, Tray and others, you can find more details in Panels section.

A panel stack is a group of panels organized in tabbed view each tab contains single panel and at one time only single tab can be visible in panel stack.

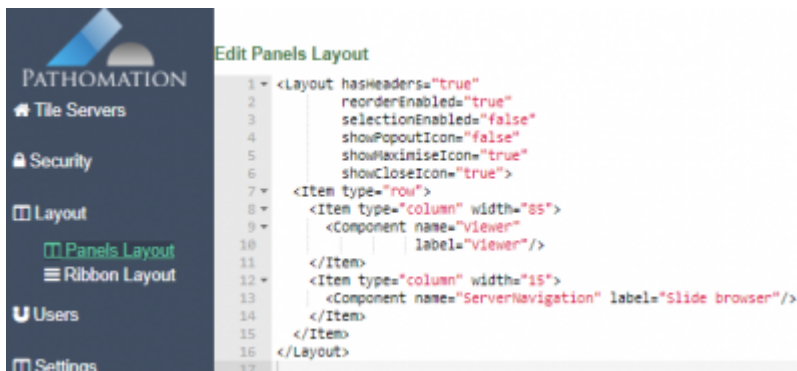

Changing panel configuration layout via the ribbon

It is possible to change the panel layout in real-time by defining action in the [ribbon layout](#).

Here's an example of how that would work:

```
<Tab label="Apsam" hint="apsam" name="tab-apsam" enabled="true"
visible="true">
  <Ribbon label="Apsam" width="10%">
    <Tool type="buttons" size="large">
      <Command name="custom-panel-algo-ichi"
        hint="Algo Ichi"
        props="https://mumbai.pathomation.com/sandbox_4143_Apsam/algo/algo1.php,Algo
        Ichi"
        icon="an_tumor.png"
      />
      <Command name="custom-panel-algo-n"
        hint="Algo N"
        props="https://mumbai.pathomation.com/sandbox_4143_Apsam/algo/algo2.php,Algo
        N"
        icon="an_macro.png"/>
    </Tool>
  </Ribbon>
  <Ribbon label="">
    <Tool type="buttons" size="large">
      <Command hint=""
```

```
        name=" "  
        icon=" " />  
    </Tool>  
</Ribbon>  
</Tab>
```

From:

<https://docs.pathomation.com/pimsls/3.0.0/> - **PIMS LS 3.0.0**

Permanent link:

https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=admin_panels&rev=1669020320Last update: **2022/11/21 11:45**