

Ribbon layout (administration)

This admin page allows you to setup default Ribbon layout that consists of different tabs and controls. Each tab consists of different controls associated to it and you can configure appearance of tabs and controls using this page.



- Tile Servers
- Security
- Layout
 - Panels Layout
 - Ribbon Layout**
- Users
- Settings
 - Global Settings
 - Conference Settings
- Logs
 - Shared Links
 - Server Visits
 - Elmah Logs
- Docs
 - Manual
 - Swagger Docs
- Viewer

Edit Ribbon Layout

[Sign Out](#)


```

51         icon="sign_out.png"/>
52     </FileTabMenu>
53     <Tab label="Home" hint="Default" name="tab-home" enabled="true" visible="true">
54         <Ribbon label="Layout" width="9%">
55             <Tool type="buttons" size="large">
56                 <Command name="simple-layout" hint="Default" icon="simple_layout.png"/>
57                 <Command name="detailed-layout" hint="With slide info and annotations" icon="detailed_layout.png"/>
58                 <Command name="simple-layout-with-tray" hint="With tray" icon="simple_layout_with_tray.png"/>
59                 <Command name="detailed-layout-with-tray" hint="Slide info, annotations and tray" icon="detailed_layout_with_tray.png"/>
60                 <Command name="panels-dialog" hint="More" icon="panels.png" />
61                 <Command name="panels-dialog" props="test" hint="More" icon="panels.png" />
62             </Tool>
63         </Ribbon>
64         <Ribbon label="viewport" width="8%">
65             <Tool type="toggle-buttons" size="small">
66                 <Command name="viewport-toggle-overview" hint="Toggle overview" icon="overview.png"/>
67                 <Command name="viewport-toggle-slide-path" hint="Toggle slide path" icon="slide_path.png"/>
68                 <Command name="viewport-toggle-barcode" hint="Toggle barcode" icon="barcode.png"/>
69                 <Command name="viewport-toggle-rotation-flip" hint="Toggle rotation and flip" icon="rotation_flip.png"/>
70             </Tool>
71             <Tool type="custom-dropdown"
72                 templateId="brightness_contrast_dropdown_template"
73                 name="brightness-contrast-dropdown"
74                 icon="bright_contr.png"/>
75             <Tool type="buttons" size="small">
76                 <Command hint="Create new Viewport panel with 1x1 grid" name="add-tab" icon="add_tab.png"/>
77                 <Command hint="Reset panel layout"
78                     name="reset-layout"
79                     icon="reset_panels_layout.png"/>
80             </Tool>
81             <Tool type="break"></Tool>
82             <Tool type="toggle-buttons" size="small">
83                 <Command name="viewport-toggle-zoom" hint="Toggle zoom controls" icon="zoom_controls.png"/>
84                 <Command name="viewport-toggle-scalebar" hint="Toggle scalebar" icon="scalebar.png"/>
85                 <Command name="viewport-toggle-label" hint="Toggle slide label" icon="slide_label.png"/>
86                 <Command name="viewport-toggle-layers" hint="Toggle channels control" icon="layers.png"/>
87                 <Command name="panel-toggle-color" hint="Toggle color settings panel" icon="panel_toggle_color.png"/>
88             </Tool>
89         </Ribbon>
90     </Tab>
91 </FileTabMenu>

```

An [exhaustive list of all available commands](#) is available elsewhere on this site.

The default sequence and behavior of the various tabs and buttons is available as [a separate ribbon tab inventory page](#).

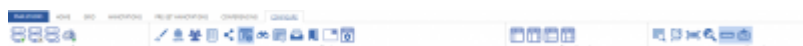

From:
<https://docs.pathomation.com/pimsls/3.0.0/> - **PIMS LS 3.0.0**

Permanent link:
https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=admin_ribbon&rev=1677512367

Last update: **2023/02/27 18:39**

