

Tile servers (administration)

In this section of PMA.studio's administrative portal, you can manage the different PMA.core instances that are used by PMA.studio to retrieve content from.

PMA.studio (PMA.view) does not function without connectivity to a PMA.core tile server. If you want to learn more about what a tile server is, [we have a YouTube video on this topic](#).

This page allows you to add different types of servers used by PMA.Studio including PMA.core, PMA.search, PMA.start, My Pathomation and PMA.live. You can press "Add New" button to add a new server.

It allows you to manage the list of created servers where you can view server status (online/offline) along with setting the order of these servers to appear in PMA.Studio viewer. Besides you can delete a server as per your convenience using the delete button.

Name	URL	PMA.core URL	PMA.search URL	PMA.start URL	PMA.live URL	PMA.core version	Status
Pathomation 1	http://www.pathomation.com/3.0.0/					3.0.0	Online
Pathomation 2	https://www.pathomation.com/3.0.0/					3.0.0	Online
Pathomation 3	https://www.pathomation.com/3.0.0/					3.0.0	Online
Pathomation 4	https://www.pathomation.com/3.0.0/					3.0.0	Offline
Pathomation 5	https://www.pathomation.com/3.0.0/					3.0.0	Offline
Pathomation 6	https://www.pathomation.com/3.0.0/					3.0.0	Offline

From:

<https://docs.pathomation.com/pimsls/3.0.0/> - PIMS LS 3.0.0

Permanent link:

https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=admin_tileservers&rev=1648584805

Last update: 2022/03/29 23:13

