


Annotations import and export

PIMS LS allows you to import and export annotations in a variety of formats, including Well-known text (the format used by PMA.core internally), Visiopharm mld files, Indica Labs HALO annotations files, Definiens/Aperio xml files, and 'plain' csv files.

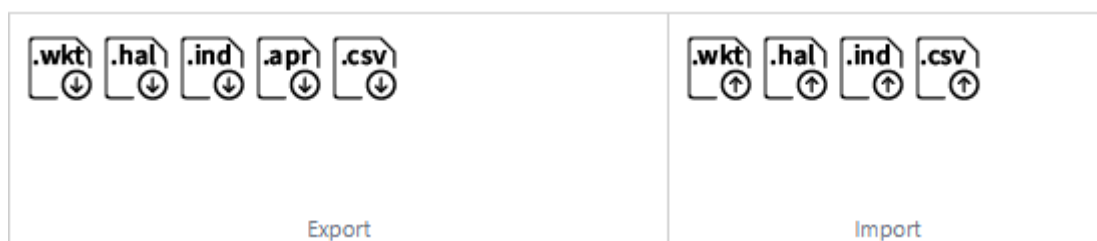


Export

You can export slide annotations in different formats including Well-known text (the format used by PMA.core internally), Visiopharm mld files, Indica Labs HALO annotations files, Definiens/Aperio xml files, and 'plain' csv files.

Import

User can also import/upload annotations through PIMS LS. Once you click on the import button for the format you want to import, a menu will appear prompting you to select the file that contains annotation data.

Once a valid file is uploaded annotations are added to the slide.

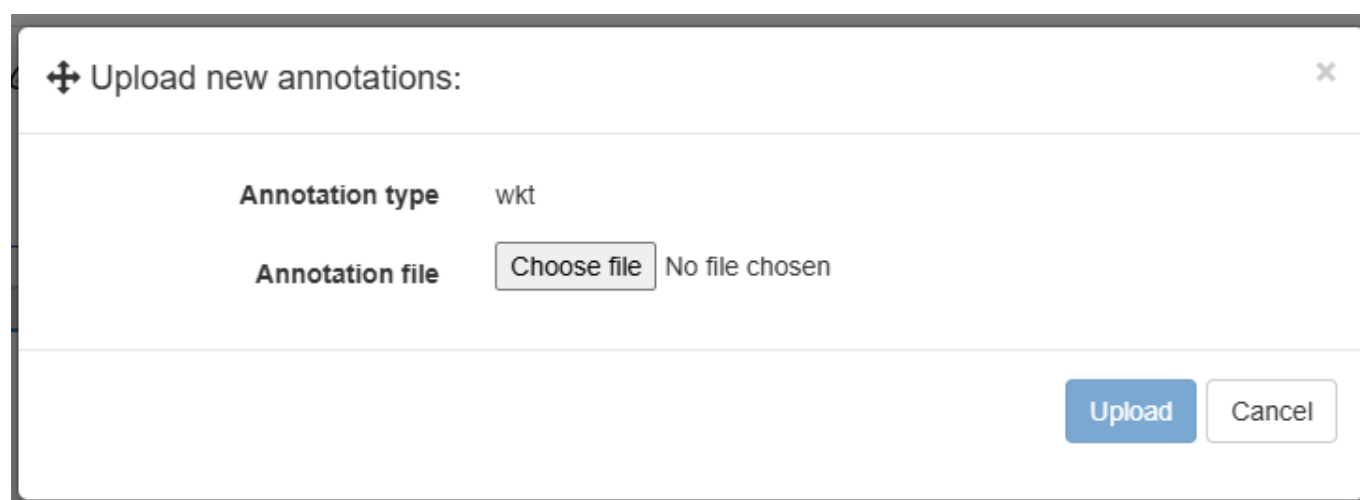

From:
<https://docs.pathomation.com/pimsls/3.0.0/> - PIMS LS 3.0.0

Permanent link:
https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=annotations_import_export&rev=1732868663

Last update: 2024/11/29 11:24

