


## Folder tools group


The Folder Tools group in the Home tab contains controls for searching & navigating folders

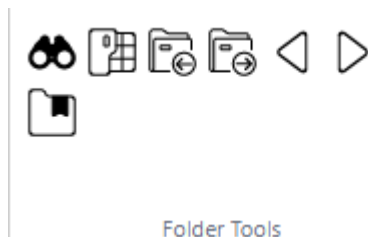

The icons, from left to right

### Top Row

- 1 Search icon - [read more about searching here](#)
- 2 Open current folder in grid - opens the menu below:

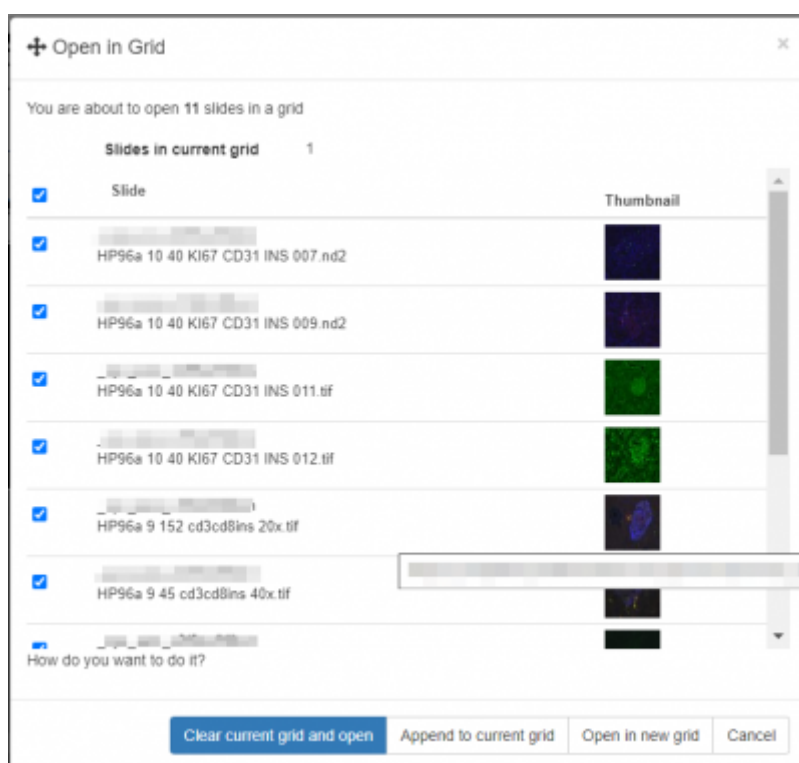

Here you can choose images from the currently highlighted folder in the slide browser to open into your current grid, or make a new grid.

- 3 Previous folder - selects and opens the previous folder
- 4 Next folder - selects and opens the previous folder

From:

<https://docs.pathomation.com/pimsls/3.0.0/> - **PIMS LS 3.0.0**

Permanent link:

[https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=folder\\_tools&rev=1731586690](https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=folder_tools&rev=1731586690)

Last update: **2024/11/14 15:18**

