


Home: Grid Tool Group

The Grid toolgroup contains icons for synchronising different viewports in the grid, clearing the grid, and opening new grid tabs.

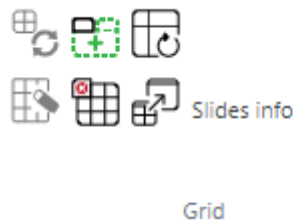

From right to left:

Top row

1. Sync button - synchronizes viewing between all images in the grid - when you zoom or move in one image, all other images will follow.
2. New viewport panel - creates a new Grid tab with a 1×1 grid
3. Reset panel layout - resets all panels to default

Bottom row

1. Clear all grid cells - clears all images from current grid
2. Clear current cell - clears image from currently selected grid cell
3. Open current cell into new viewport - creates a new 1×1 grid tab and opens the current image there

From:

<https://docs.pathomation.com/pimsls/3.0.0/> - PIMS LS 3.0.0

Permanent link:

<https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=grid&rev=1731599362>

Last update: 2024/11/14 18:49

