


Other Tools

Add/Remove Row - Add/Remove Columns



These tools allows us to add new rows to the current grid, delete rows, and add or remove new columns. This allows user to open up multiple slides in a grid view.

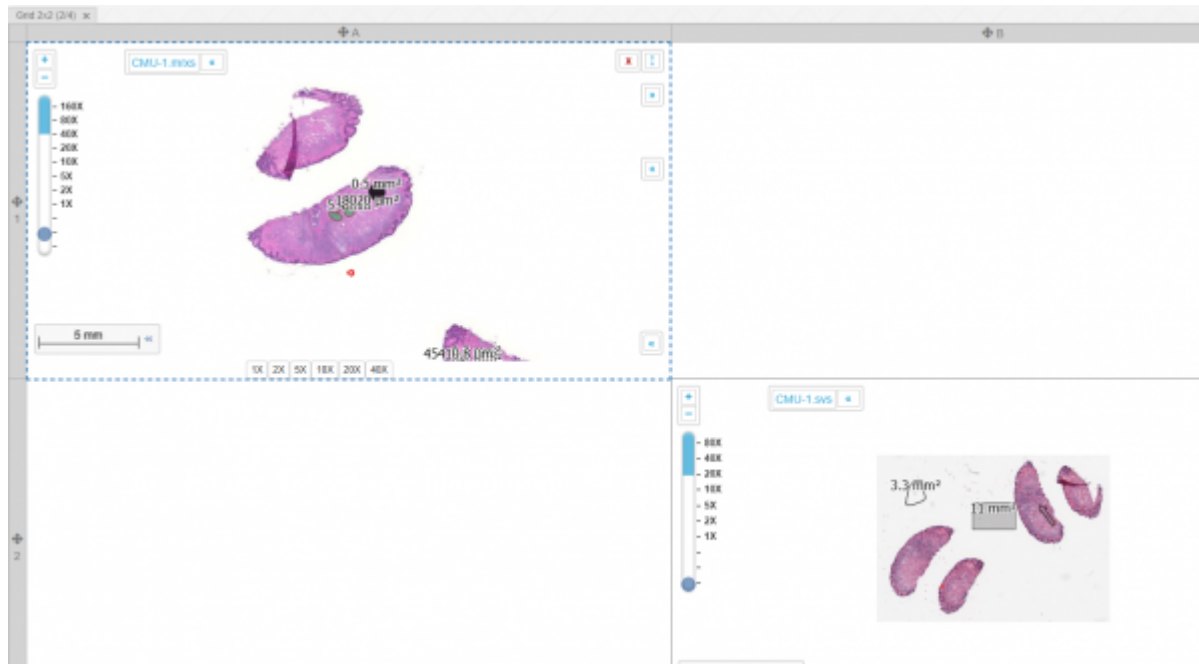

Grid snapshot



This tool allows user to take snapshot of the whole grid. On clicking the icon, a popup shows up that allows user to selected different options like annotations, barcode, labels to be part of the grid snapshot.

figure

Conference

Support

1X

2X

5X

10X

20X

Slide

CMU-1.mrxs

50X

10X

5X

2X

1X

5 mm

20X

10X

5X

2X

1X

3.3 mm²

11 mm²

Save snapshot

Select Image Format

JPEG

Filename Template

snapshot_%date%_%time%

DateTimeFolderSlide

Show Scale Bar☐

Show Annotations☐

Show Slide Overview☒

Show Barcode☐

Show Slide Label☐

Download Archive Of All Rois From The Grid☐

Server	Path	Image size
Snapshot	_sys_ref/annotations/CMU-1.mrxs	792 * 391 px
Snapshot	_sys_ref/annotations/CMU-1.svs	793 * 391 px

Save SnapshotTo ClipboardClose

_sys_ref/annotations/CMU-1.mrxs

1-nwc

0.5 mm²

18020 μm²

5.4 mm

10 mm

_sys_ref/annotations/CMU-1.svs

3.3 mm²

11 mm²

CMU-1

From:
<https://docs.pathomation.com/pimsls/3.0.0/> - PIMS LS 3.0.0

Permanent link:
https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=other_grid_tools

Last update: 2025/05/12 18:25