



Slide information panel

The Info panel displays available slide information for currently selected slide.

Info	×	□	×
File name	_sys_ref/brightfield/Grundium/hp1 (1).svs		
Last Modified	Mar 01 2021 18:11		
Width	131,072 pixels / 32.840 mm		
Height	73,728 pixels / 18.473 mm		
Max Magnification	40X		
Micrometres Per Pixel X	0.250552		
Micrometres Per Pixel Y	0.250552		
Physical Size	328.53 MB		
Tile Size	512 x 512 pixels		
Barcode			
Barcode text			
Overview			
Macro	N/A		
Layers	1		
Channels	1		

Currently selected slide is highlighted with blue dashed border.

Columns are pretty self-descriptive.

The barcode, macro and overview thumbnails are clickable. Clicking on the images will open then in a new browser tab.

From:

<https://docs.pathomation.com/pimsls/3.0.0/> - **PIMS LS 3.0.0**

Permanent link:

https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=panel_info&rev=1677836884

Last update: **2023/03/03 12:48**

